

## ***Spare Parts Catalogue***

---

Version: 2010/06/29a EN



*küschall® K-Junior*

## General Information

For any order of spare parts the following information is required:

- Type of wheelchair
- Serial Number
- Part-Number
- Part-Description

### Abbreviations

SB      Seat width

ST      Seat depth

UL      Knee-to-heel height



Left side (in direction of movement)



Right side (in direction of movement)



An assembly consisting of a right and a left part, referred to as a 'pair'.







Exploded view drawings and code numbers are not contractual and subject to modification, in relation to our continuous product improvement policy to meet end users needs.

## TABLE OF CONTENTS

---


CHASSIS .....	5
Frame .....	6
Frame for quick release .....	7
Axle tube standard .....	8
Axle tube Vario-Ax .....	10
Axle holder .....	11
Rear wheel extension standard .....	12
Rear wheel extension for Vario-Ax .....	13
SEAT .....	14
Seat module .....	15
Seat frame, cover standard .....	16
Seat cushion .....	19
BACKREST .....	22
Backrest .....	23
Bands to adjustable backrest .....	25
Cover to adjustable backrest .....	26
Push handles, height adjustable, rear set .....	27
FOOTREST .....	28
Footrest standard .....	29
Footrest angle adjustable .....	30
Footrest retracted .....	32
Footrest kit, high mounted .....	34
Calf strap .....	35
SIDERESTS .....	36
Clothes-guards .....	37
Mudguards =>31.10.2008 .....	38
Mudguards 01.11.2008=> .....	39
Tubular armrest .....	40
FRONT WHEELS .....	41
Front forks standard .....	42
Front forks quick release .....	44
Front wheels .....	45
REAR WHEELS .....	47
20" KÜSCHALL rear wheels complete .....	48
20" KÜSCHALL rear wheel accessories .....	49
22" KÜSCHALL rear wheels complete .....	50
22" KÜSCHALL rear wheel accessories .....	51
24" KÜSCHALL rear wheels complete .....	52
24" KÜSCHALL rear wheel accessories .....	53
High Performance wheel .....	55
Mountainbike wheel .....	57
Spider wheel .....	58
Spinergy wheel .....	59
BRAKES .....	60
Brakes =>14.02.2008 .....	61
Brakes 15.02.2008=> .....	62
OPTIONS & ACCESSORIES .....	63
Antitipper .....	64
Active antitipper .....	65
Transit wheels .....	66
Buggy push handle .....	67
Chair accessories .....	68



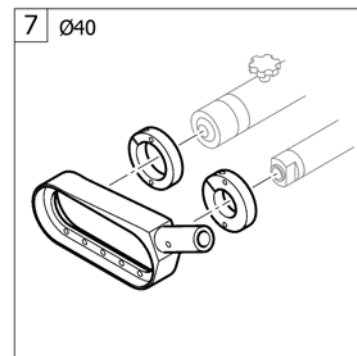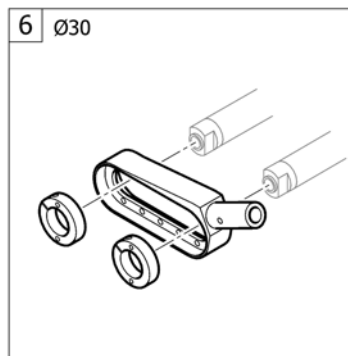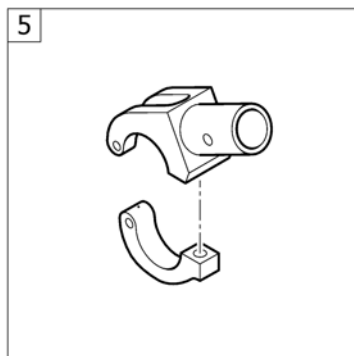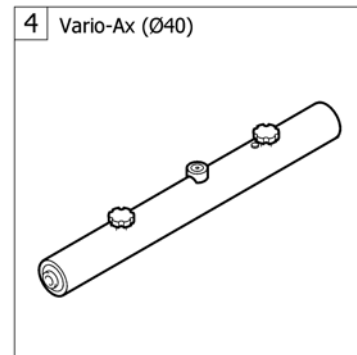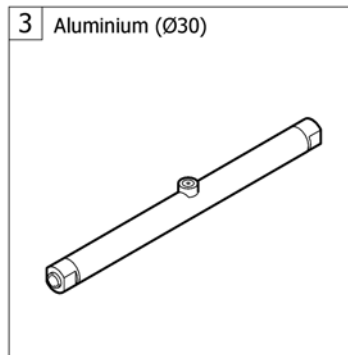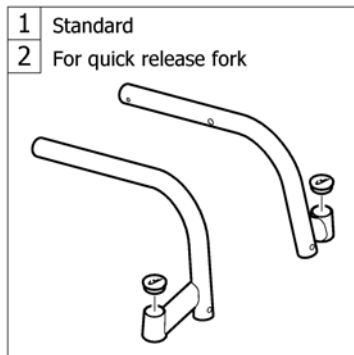


Pos.	Part number	Description	Valid from → until	Qty
1	-	Frame		1
2	-	Frame for quick release		1
3	-	Axle tube standard		1
4	-	Axle tube Vario-Ax		1
5	-	Axle holder		1
6	-	Rear wheel extension standard		1
7	-	Rear wheel extension for Vario-Ax		1

# Frame

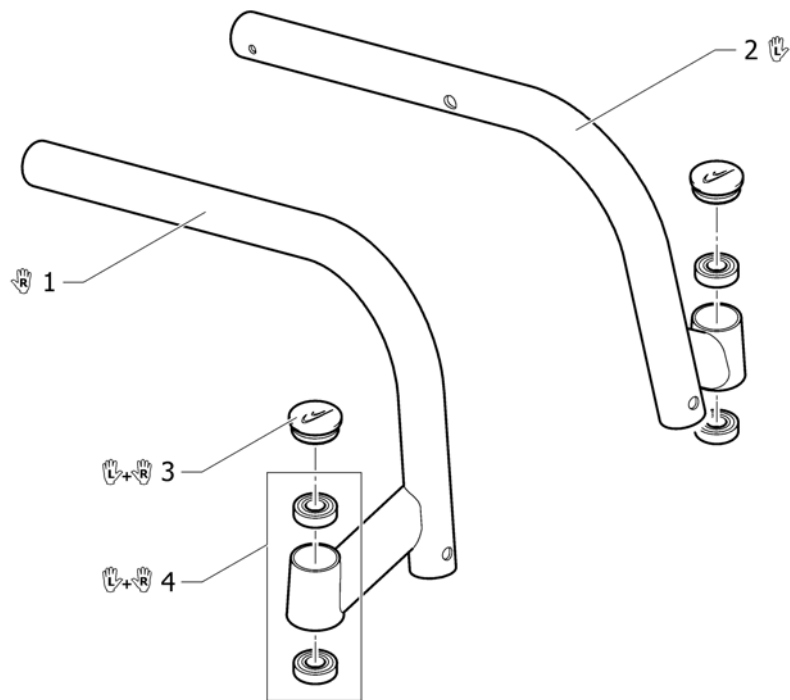


Pos.	Part number	Description	Valid from → until	Qty
1	1451932-0419	Frame Junior right, silver grey		1
1	1451932-0424	Frame Junior right, cyber green		1
1	1451932-0425	Frame Junior right, xanthic yellow	→ 20081231	1
1	1451932-0443	Frame Junior right, pearly sapphire		1
1	1451932-3524	Frame Junior right, royal red		1
1	1451932-9005	Frame Junior right, glossy black		1
2	1451933-0419	Frame Junior left, silver grey		1
2	1451933-0424	Frame Junior left, cyber green		1
2	1451933-0425	Frame Junior left, xanthic yellow	→ 20081231	1
2	1451933-0443	Frame Junior left, pearly sapphire		1
2	1451933-3524	Frame Junior left, royal red		1
2	1451933-9005	Frame Junior left, glossy black		1
3	1487961-0004	Bearing block cap pair, black anodized		1
3	1487961-0400	Bearing block cap pair, techno-anthrazit		1
3	1487961-0401	Bearing block cap pair, lemon		1
3	1487961-0402	Bearing block cap pair, apricot		1
3	1487961-0403	Bearing block cap pair, fashion violett	→ 20070630	1
3	1487961-0404	Bearing block cap pair, b-blue		1
3	1487961-0440	Bearing block cap pair, comet red	20070101 →	1
4	1451718	Bearing kit		1

# Frame for quick release

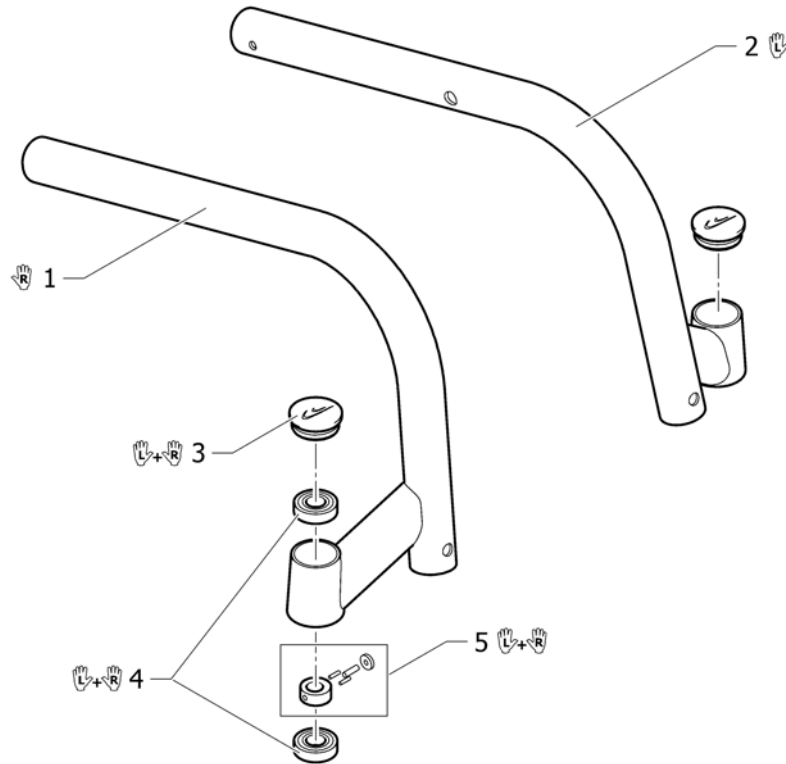

Pos.	Part number	Description	Valid from → until	Qty
1	1486013-0419	Frame Junior right for quick release fork, silver grey		1
1	1486013-0424	Frame Junior right for quick release fork, cyber green		1
1	1486013-0425	Frame Junior right for quick release fork, xanthic yellow	→ 20081231	1
1	1486013-0443	Frame Junior right for quick release fork, pearly sapphire		1
1	1486013-3524	Frame Junior right for quick release fork, royal red		1
1	1486013-9005	Frame Junior right for quick release fork, glossy black		1
2	1486014-0419	Frame Junior left for quick release fork, silver grey		1
2	1486014-0424	Frame Junior left for quick release fork, cyber green		1
2	1486014-0425	Frame Junior left for quick release fork, xanthic yellow	→ 20081231	1
2	1486014-0443	Frame Junior left for quick release fork, pearly sapphire		1
2	1486014-3524	Frame Junior left for quick release fork, royal red		1
2	1486014-9005	Frame Junior left for quick release fork, glossy black		1
3	1487961-0004	Bearing block cap pair, black anodized		1
3	1487961-0400	Bearing block cap pair, techno-anthrazit		1
3	1487961-0401	Bearing block cap pair, lemon		1
3	1487961-0402	Bearing block cap pair, apricot		1
3	1487961-0403	Bearing block cap pair, fashion violett	→ 20070630	1
3	1487961-0404	Bearing block cap pair, b-blue		1
3	1487961-0440	Bearing block cap pair, comet red	20070101 →	1
4	1451718	Bearing kit		1
5	1486015	Quick release, kit		1

# Axle tube standard

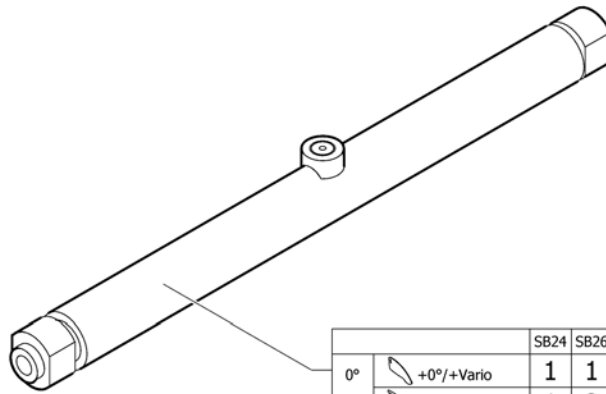

		SB24	SB26	SB28	SB30	SB32	SB34	SB36
0°	+0°/+Vario	1	1	2	3	4	5	6
	+0°/+3°/+Vario	1	2	3	4	5	6	7
3°	+0°/+3°/+Vario	8	8	9	10	11	12	13
	+6°	9	9	10	11	12	13	14
6°	+3°/+6°	15	15	16	17	18	19	20
	+9°	16	16	17	18	19	20	21
9°	+6°/+9°	22	22	23	24	25	26	27


Pos.	Part number	Description	Valid from → until	Qty
1	1507507	Axle tube Aluminium 0° complete, 218mm <i>Replaces 1416810</i>		1
2	1507508	Axle tube Aluminium 0° complete, 238mm <i>Replaces 1487990</i>		1
3	1507509	Axle tube Aluminium 0° complete, 258mm <i>Replaces 1487991</i>		1
4	1507510	Axle tube Aluminium 0° complete, 278mm <i>Replaces 1416822</i>		1
5	1507511	Axle tube Aluminium 0° complete, 298mm <i>Replaces 1487992</i>		1
6	1507512	Axle tube Aluminium 0° complete, 318mm <i>Replaces 1487993</i>		1
7	1507513	Axle tube Aluminium 0° complete, 338mm <i>Replaces 1416834</i>		1
8	1507522	Axle tube Aluminium 3° complete, 244mm <i>Replaces 1416811</i>		1
9	1507523	Axle tube Aluminium 3° complete, 264mm <i>Replaces 1420809</i>		1
10	1507524	Axle tube Aluminium 3° complete, 284mm <i>Replaces 1487994</i>		1
11	1507525	Axle tube Aluminium 3° complete, 304mm <i>Replaces 1416823</i>		1
12	1507526	Axle tube Aluminium 3° complete, 324mm <i>Replaces 1420817</i>		1
13	1507527	Axle tube Aluminium 3° complete, 344mm <i>Replaces 1438710</i>		1
14	1507528	Axle tube Aluminium 3° complete, 364mm <i>Replaces 1416835</i>		1
15	1507536	Axle tube Aluminium 6° complete, 274mm <i>Replaces 1416812</i>		1
16	1507537	Axle tube Aluminium 6° complete, 294mm <i>Replaces 1420810</i>		1
17	1507538	Axle tube Aluminium 6° complete, 314mm <i>Replaces 1487995</i>		1
18	1507539	Axle tube Aluminium 6° complete, 334mm <i>Replaces 1416824</i>		1




**Axle tube standard**

<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
19	1507540	Axle tube Aluminium 6° complete, 354mm <i>Replaces 1420818</i>		1
20	1507541	Axle tube Aluminium 6° complete, 374mm <i>Replaces 1438717</i>		1
21	1507542	Axle tube Aluminium 6° complete, 394mm <i>Replaces 1416836</i>		1
22	1507550	Axle tube Aluminium 9° complete, 304mm <i>Replaces 1416813</i>		1
23	1507551	Axle tube Aluminium 9° complete, 324mm <i>Replaces 1420811</i>		1
24	1507552	Axle tube Aluminium 9° complete, 344mm <i>Replaces 1487996</i>		1
25	1507553	Axle tube Aluminium 9° complete, 364mm <i>Replaces 1416825</i>		1
26	1507554	Axle tube Aluminium 9° complete, 384mm <i>Replaces 1420819</i>		1
27	1507555	Axle tube Aluminium 9° complete, 404mm <i>Replaces 1438724</i>		1

# Axle tube Vario-Ax

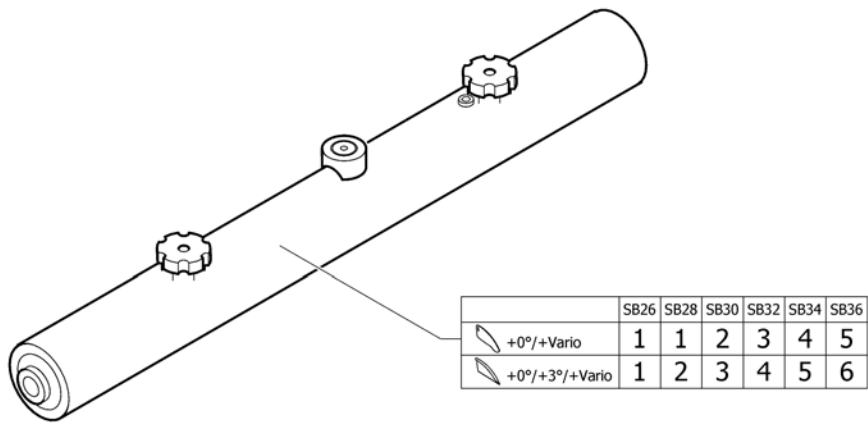


Pos.	Part number	Description	Valid from → until	Qty
1	1507605	Vario-Ax complete 275mm <i>Replaces 1487980</i>		1
2	1507607	Vario-Ax complete 295mm <i>Replaces 1487981</i>		1
3	1507608	Vario-Ax complete 315mm <i>Replaces 1416989</i>		1
4	1507609	Vario-Ax complete 335mm <i>Replaces 1487982</i>		1
5	1507611	Vario-Ax complete 355mm <i>Replaces 1487983</i>		1
6	1507612	Vario-Ax complete 375mm <i>Replaces 907.1161</i>		1

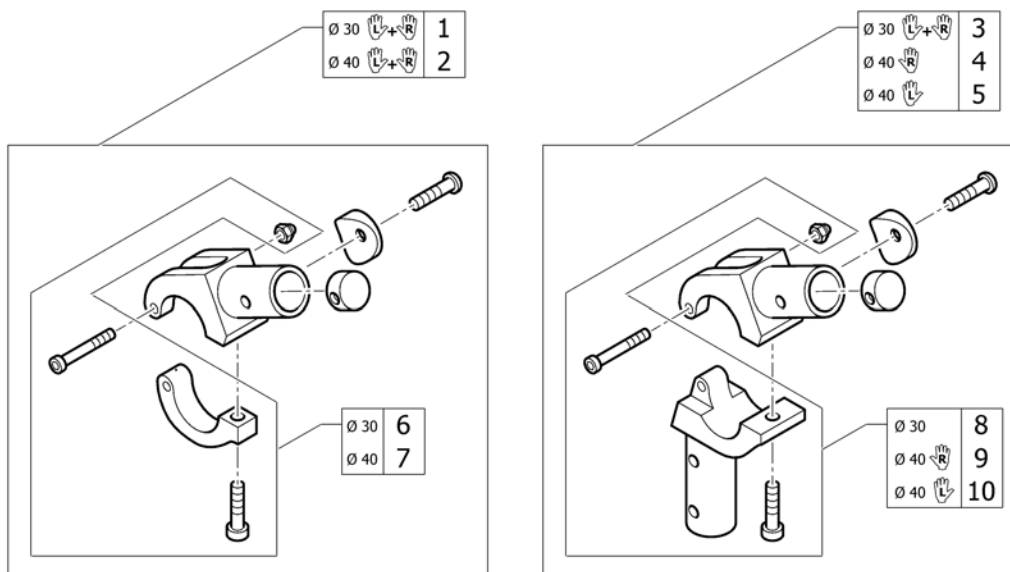**Axle holder**


Pos.	Part number	Description	Valid from → until	Qty
1	1487954	Axle holder complete Ø30, pair		1
2	1487955	Axle holder complete Ø40, pair		1
3	1451738	Axle holder for adaptations Ø30, pair		1
4	1451739	Axle holder for adaptations Ø40, right		1
5	1451740	Axle holder for adaptations Ø40, left		1
6	1487956	Axle holder latch Ø30		2
7	1487957	Axle holder latch Ø40		2
8	1487958	Axle holder latch for Adaptations, Ø30		2
9	1487959	Axle holder latch for Adaptations right, Ø40		1
10	1487960	Axle holder latch for Adaptations left, Ø40		1

## Rear wheel extension standard

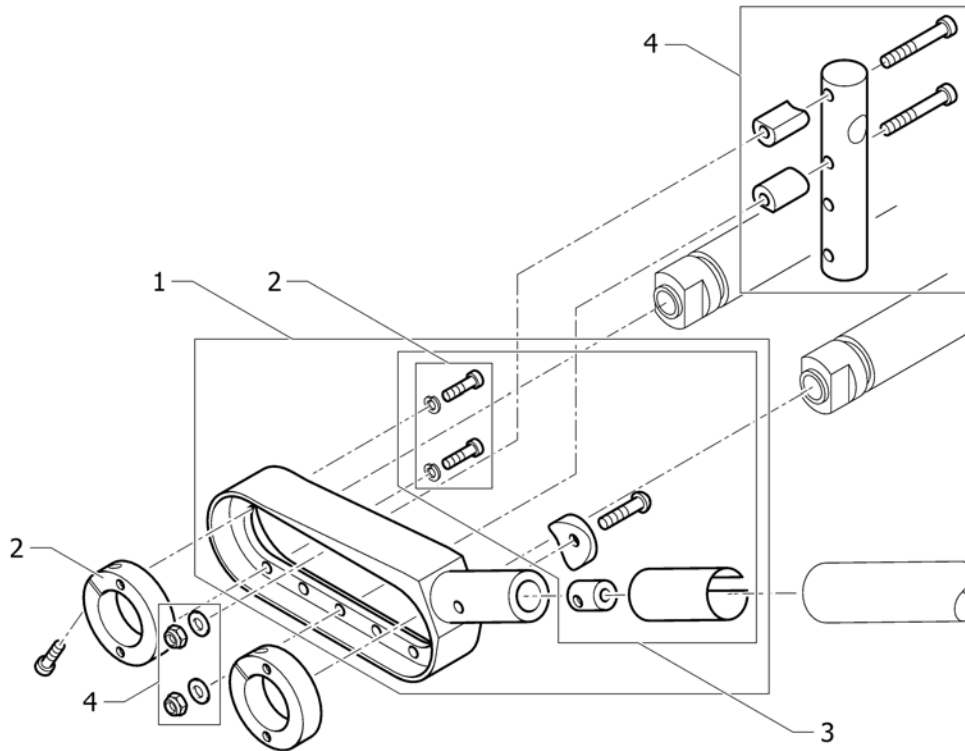



Pos.	Part number	Description	Valid from → until	Qty
1	1451741	Rear wheel extension standard complete		2
2	1451743	Clamping ring Ø50/30 incl. screw kit		2
3	1451746	Assembly kit for pair rear wheel extension		1
4	1451747	Adaptation holder, kit for rear wheel extension standard		2

## Rear wheel extension for Vario-Ax

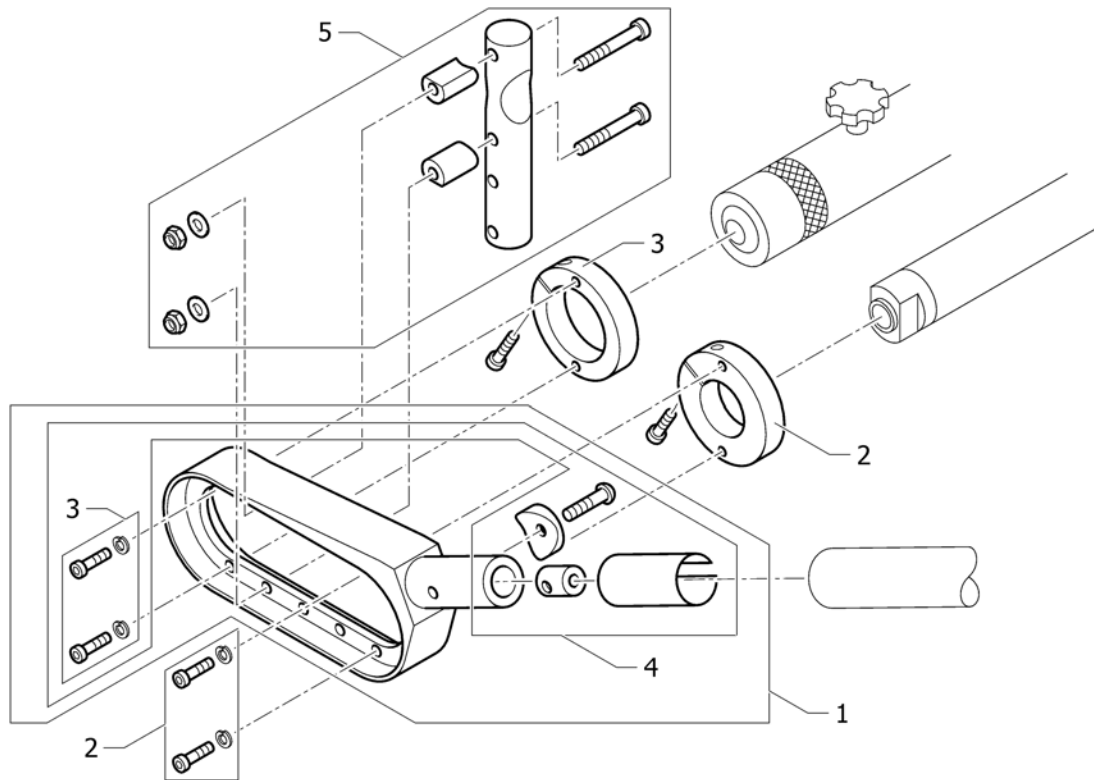


Pos.	Part number	Description	Valid from → until	Qty
1	1451742	Rear wheel extension Vario-Ax complete		1
2	1451744	Clamping ring Ø60/30 incl. screw kit		1
3	1451745	Clamping ring Ø60/40 incl. screw kit		1
4	1451746	Assembly kit for pair rear wheel extension		1
5	1451748	Adaptation holder, kit for rear wheel extension Vario-Ax		1

# SEAT

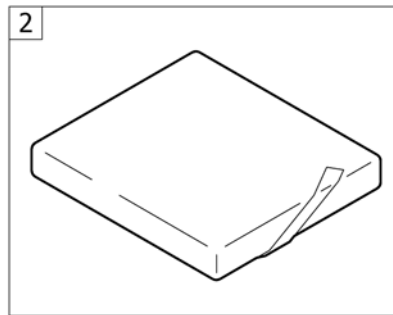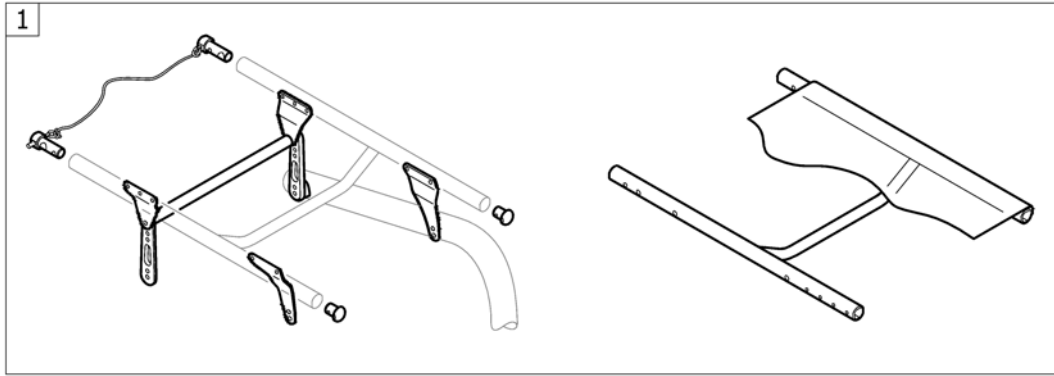


Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat module		1
2	-	Seat cushion		1

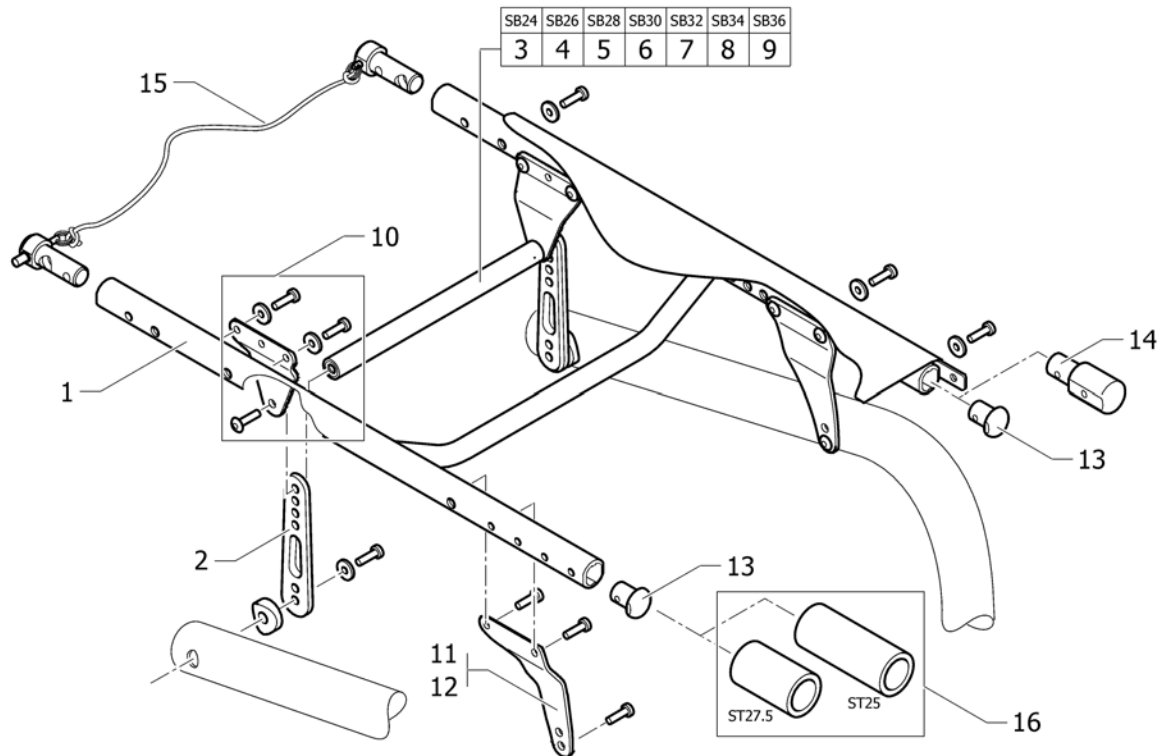

Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat frame, cover standard		1
2	1486528	Seat brace-Set vertical (S, M, L)		1
3	1485733	Seat brace horizontal SB24		1
4	1485734	Seat brace horizontal SB26		1
5	1485735	Seat brace horizontal SB28		1
6	1485736	Seat brace horizontal SB30		1
7	1485737	Seat brace horizontal SB32		1
8	1451842	Seat brace horizontal SB34		1
9	1451843	Seat brace horizontal SB36		1
10	1451921	Hinge bracket-Set Seat Frame, rear		1
11	1486535	Hinge bracket-Set Front Seat Frame S		1
12	1486536	Hinge bracket-Set Front Seat Frame L		1
13	1486539-0004	Seatmodule plug, pair, black anodized		1
13	1486539-0400	Seatmodule plug, pair, techno-anthrazit		1
13	1486539-0401	Seatmodule plug, pair, lemon		1
13	1486539-0402	Seatmodule plug, pair, apricot		1
13	1486539-0403	Seatmodule plug, pair, fashion violett	→ 20070630	1
13	1486539-0404	Seatmodule plug, pair, b-blue		1
13	1486539-0440	Seatmodule plug, pair, comet red	20070101 →	1
13	1485738-0004	Seat extension, pair, black		1
14	1485738-0400	Seat extension, pair, techno-anthracit		1
14	1485738-0401	Seat extension, pair, lemon		1
14	1485738-0402	Seat extension, pair, apricot		1
14	1485738-0403	Seat extension, pair, fashion violett	→ 20070630	1
14	1485738-0404	Seat extension, pair, b-blue		1
14	1485738-0440	Seat extension, pair, comet red	20070101 →	1
15	1486540	Backrest-Bolt Set		1
16	1506103	Protection cover for seat frame		1

# Seat frame, cover standard

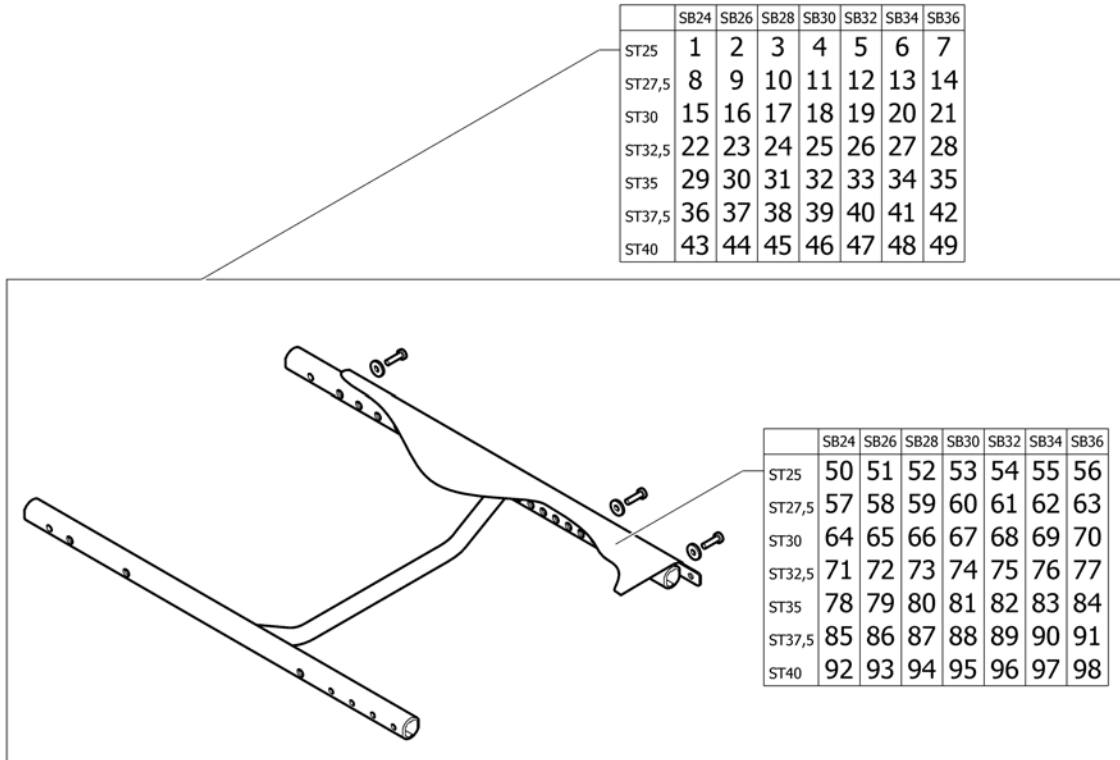

Pos.	Part number	Description	Valid from → until	Qty
1	1484608	Seat module complete for frame Junior SB24 ST25		1
2	1484615	Seat module complete for frame Junior SB26 ST25		1
3	1484622	Seat module complete for frame Junior SB28 ST25		1
4	1484629	Seat module complete for frame Junior SB30 ST25		1
5	1484860	Seat module complete for frame Junior SB32 ST25		1
6	1484867	Seat module complete for frame Junior SB34 ST25		1
7	1485050	Seat module complete for frame Junior SB36 ST25		1
8	1484609	Seat module complete for frame Junior SB24 ST27.5		1
9	1484616	Seat module complete for frame Junior SB26 ST27.5		1
10	1484623	Seat module complete for frame Junior SB28 ST27.5		1
11	1484630	Seat module complete for frame Junior SB30 ST27.5		1
12	1484861	Seat module complete for frame Junior SB32 ST27.5		1
13	1484868	Seat module complete for frame Junior SB34 ST27.5		1
14	1485051	Seat module complete for frame Junior SB36 ST27.5		1
15	1484610	Seat module complete for frame Junior SB24 ST30		1
16	1484617	Seat module complete for frame Junior SB26 ST30		1
17	1484624	Seat module complete for frame Junior SB28 ST30		1
18	1484631	Seat module complete for frame Junior SB30 ST30		1
19	1484862	Seat module complete for frame Junior SB32 ST30		1
20	1485048	Seat module complete for frame Junior SB34 ST30		1
21	1485052	Seat module complete for frame Junior SB36 ST30		1
22	1484611	Seat module complete for frame Junior SB24 ST32.5		1
23	1484618	Seat module complete for frame Junior SB26 ST32.5		1
24	1484625	Seat module complete for frame Junior SB28 ST32.5		1
25	1484856	Seat module complete for frame Junior SB30 ST32.5		1
26	1484863	Seat module complete for frame Junior SB32 ST32.5		1
27	1485049	Seat module complete for frame Junior SB34 ST32.5		1
28	1485053	Seat module complete for frame Junior SB36 ST32.5		1
29	1484612	Seat module complete for frame Junior SB24 ST35		1
30	1484619	Seat module complete for frame Junior SB26 ST35		1
31	1484626	Seat module complete for frame Junior SB28 ST35		1




**Seat frame, cover standard**

Pos.	Part number	Description	Valid from → until	Qty
32	1484857	Seat module complete for frame Junior SB30 ST35		1
33	1484864	Seat module complete for frame Junior SB32 ST35		1
34	1487584	Seat module complete for frame short SB34 ST35		1
35	1487585	Seat module complete for frame short SB36 ST35		1
36	1484613	Seat module complete for frame Junior SB24 ST37.5		1
37	1484620	Seat module complete for frame Junior SB26 ST37.5		1
38	1484627	Seat module complete for frame Junior SB28 ST37.5		1
39	1484858	Seat module complete for frame Junior SB30 ST37.5		1
40	1484865	Seat module complete for frame Junior SB32 ST37.5		1
41	1487590	Seat frame to frame Short cover standard SB34 ST37.5		1
42	1487591	Seat frame to frame Short cover standard SB36 ST37.5		1
43	1484614	Seat module complete for frame Junior SB24 ST40		1
44	1484621	Seat module complete for frame Junior SB26 ST40		1
45	1484628	Seat module complete for frame Junior SB28 ST40		1
46	1484859	Seat module complete for frame Junior SB30 ST40		1
47	1484866	Seat module complete for frame Junior SB32 ST40		1
48	1487596	Seat frame to frame Short cover standard SB34 ST40		1
49	1487535	Seat frame to frame Short Bezug standard SB36 ST40		1
50	1485054	Seat cover complete SB24 ST25 for seat frame Junior		1
51	1485061	Seat cover complete SB26 ST25 for seat frame Junior		1
52	1485068	Seat cover complete SB28 ST25 for seat frame Junior		1
53	1485075	Seat cover complete SB30 ST25 for seat frame Junior		1
54	1485186	Seat cover complete SB32 ST25 for seat frame Junior		1
55	1485571	Seat cover complete SB34 ST25 for seat frame Junior		1
56	1485729	Seat cover complete SB36 ST25 for seat frame Junior		1
57	1485055	Seat cover complete SB24 ST27.5 for seat frame Junior		1
58	1485062	Seat cover complete SB26 ST27.5 for seat frame Junior		1
59	1485069	Seat cover complete SB28 ST27.5 for seat frame Junior		1
60	1485076	Seat cover complete SB30 ST27.5 for seat frame Junior		1
61	1485187	Seat cover complete SB32 ST27.5 for seat frame Junior		1
62	1485572	Seat cover complete SB34 ST27.5 for seat frame Junior		1
63	1485730	Seat cover complete SB36 ST27.5 for seat frame Junior		1
64	1485056	Seat cover complete SB24 ST30 for seat frame Junior		1
65	1485063	Seat cover complete SB26 ST30 for seat frame Junior		1
66	1485070	Seat cover complete SB28 ST30 for seat frame Junior		1
67	1485077	Seat cover complete SB30 ST30 for seat frame Junior		1
68	1485553	Seat cover complete SB32 ST30 for seat frame Junior		1
69	1485573	Seat cover complete SB34 ST30 for seat frame Junior		1
70	1485731	Seat cover complete SB36 ST30 for seat frame Junior		1
71	1485057	Seat cover complete SB24 ST32.5 for seat frame Junior		1
72	1485064	Seat cover complete SB26 ST32.5 for seat frame Junior		1
73	1485071	Seat cover complete SB28 ST32.5 for seat frame Junior		1
74	1485078	Seat cover complete SB30 ST32.5 for seat frame Junior		1
75	1485567	Seat cover complete SB32 ST32.5 for seat frame Junior		1
76	1485728	Seat cover complete SB34 ST32.5 for seat frame Junior		1
77	1485732	Seat cover complete SB36 ST32.5 for seat frame Junior		1
78	1485058	Seat cover complete SB24 ST35 for seat frame Junior		1
79	1485065	Seat cover complete SB26 ST35 for seat frame Junior		1
80	1485072	Seat cover complete SB28 ST35 for seat frame Junior		1
81	1485079	Seat cover complete SB30 ST35 for seat frame Junior		1
82	1485568	Seat cover complete SB32 ST35 for seat frame Junior		1
83	1486573	Seatcover complete for frame short SB34 ST35		1
84	1486513	Seatcover complete for frame short SB36 ST35		1
85	1485059	Seat cover complete SB24 ST37.5 for seat frame Junior		1
86	1485066	Seat cover complete SB26 ST37.5 for seat frame Junior		1

**Seat frame, cover standard**

Pos.	Part number	Description	Valid from → until	Qty
87	1485073	Seat cover complete SB28 ST37.5 for seat frame Junior		1
88	1485080	Seat cover complete SB30 ST37.5 for seat frame Junior		1
89	1485569	Seat cover complete SB32 ST37.5 for seat frame Junior		1
90	1486574	Seat cover to Standard-Seat frame, kit Short SB34 ST37.5		1
91	1486514	Seat cover to Standard-Seat frame Short SB36 ST37.5		1
92	1485060	Seat cover complete SB24 ST40 for seat frame Junior		1
93	1485067	Seat cover complete SB26 ST40 for seat frame Junior		1
94	1485074	Seat cover complete SB28 ST40 for seat frame Junior		1
95	1485081	Seat cover complete SB30 ST40 for seat frame Junior		1
96	1485570	Seat cover complete SB32 ST40 for seat frame Junior		1
97	1486575	Seat cover to Standard-Seat frame, kit Short SB34 ST40		1
98	1486515	Seat cover to Standard-Seat frame Short SB36 ST40		1

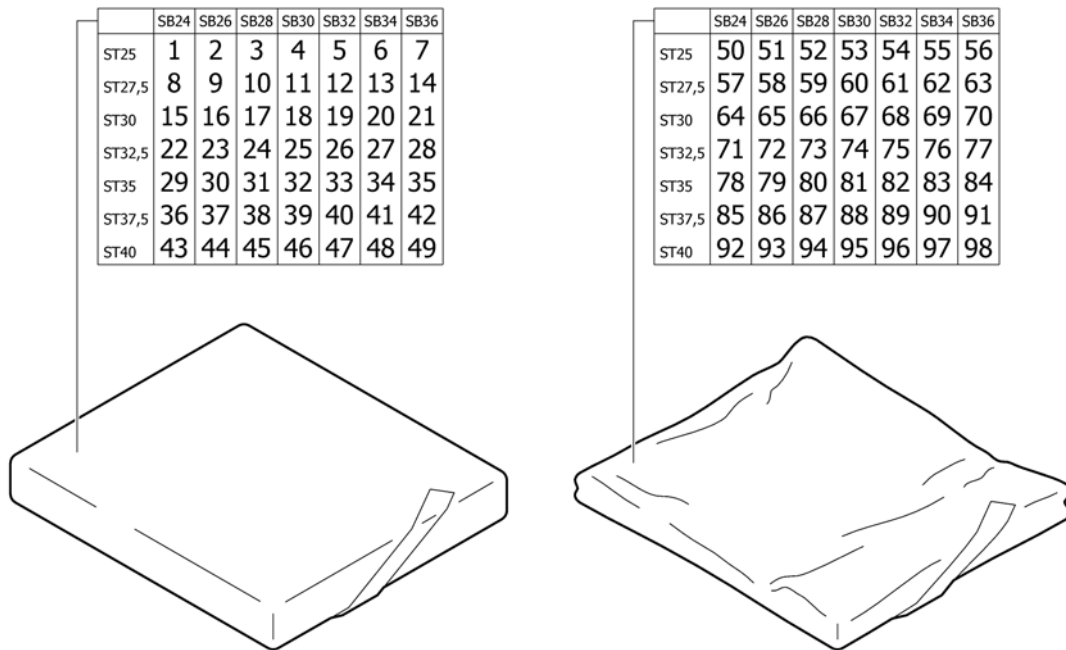**Seat cushion**

Pos.	Part number	Description	Valid from → until	Qty
1	1516406	Seat cushion SB24 ST25 black complete		1
2	1516407	Seat cushion SB26 ST25 black complete		1
3	1516408	Seat cushion SB28 ST25 black complete		1
4	1516409	Seat cushion SB30 ST25 black complete		1
5	1516410	Seat cushion SB32 ST25 black complete		1
6	1516411	Seat cushion SB34 ST25 black complete		1
7	1516412	Seat cushion SB36 ST25 black complete		1
8	1516420	Seat cushion SB24 ST27,5 black complete		1
9	1516421	Seat cushion SB26 ST27,5 black complete		1
10	1516422	Seat cushion SB28 ST27,5 black complete		1
11	1516423	Seat cushion SB30 ST27,5 black complete		1
12	1516424	Seat cushion SB32 ST27,5 black complete		1
13	1516425	Seat cushion SB34 ST27,5 black complete		1
14	1516426	Seat cushion SB36 ST27,5 black complete		1
15	1516434	Seat cushion SB24 ST30 black complete		1
16	1516435	Seat cushion SB26 ST30 black complete		1
17	1516436	Seat cushion SB28 ST30 black complete		1
18	1516437	Seat cushion SB30 ST30 black complete		1
19	1516438	Seat cushion SB32 ST30 black complete		1
20	1516439	Seat cushion SB34 ST30 black complete		1
21	1516440	Seat cushion SB36 ST30 black complete		1
22	1516490	Seat cushion SB24 ST32,5 black complete		1
23	1516491	Seat cushion SB26 ST32,5 black complete		1
24	1516492	Seat cushion SB28 ST32,5 black complete		1
25	1516493	Seat cushion SB30 ST32,5 black complete		1
26	1516518	Seat cushion SB32 ST32,5 black complete		1
27	1516519	Seat cushion SB34 ST32,5 black complete		1
28	1516520	Seat cushion SB36 ST32,5 black complete		1
29	1516547	Seat cushion SB24 ST35 black complete		1
30	1516548	Seat cushion SB26 ST35 black complete		1
31	1516560	Seat cushion SB28 ST35 black complete		1


**Seat cushion**

Pos.	Part number	Description	Valid from → until	Qty
32	1516561	Seat cushion SB30 ST35 black complete		1
33	1516562	Seat cushion SB32 ST35 black complete		1
34	1516574	Seat cushion SB34 ST35 black complete		1
35	1516575	Seat cushion SB36 ST35 black complete		1
36	1516770	Seat cushion SB24 ST37,5 black complete		1
37	1516774	Seat cushion SB26 ST37,5 black complete		1
38	1516779	Seat cushion SB28 ST37,5 black complete		1
39	1516783	Seat cushion SB30 ST37,5 black complete		1
40	1516787	Seat cushion SB32 ST37,5 black complete		1
41	1516792	Seat cushion SB34 ST37,5 black complete		1
42	1516796	Seat cushion SB36 ST37,5 black complete		1
43	1516504	Seat cushion SB24 ST40 black complete		1
44	1516505	Seat cushion SB26 ST40 black complete		1
45	1516506	Seat cushion SB28 ST40 black complete		1
46	1516507	Seat cushion SB30 ST40 black complete		1
47	1516508	Seat cushion SB32 ST40 black complete		1
48	1516509	Seat cushion SB34 ST40 black complete		1
49	1516510	Seat cushion SB36 ST40 black complete		1
50	1516588	Cover to seat cushion SB24 ST25 black		1
51	1516589	Cover to seat cushion SB26 ST25 black		1
52	1516590	Cover to seat cushion SB28 ST25 black		1
53	1516591	Cover to seat cushion SB30 ST25 black		1
54	1516592	Cover to seat cushion SB32 ST25 black		1
55	1516593	Cover to seat cushion SB34 ST25 black		1
56	1516594	Cover to seat cushion SB36 ST25 black		1
57	1516602	Cover to seat cushion SB24 ST27,5 black		1
58	1516603	Cover to seat cushion SB26 ST27,5 black		1
59	1516604	Cover to seat cushion SB28 ST27,5 black		1
60	1516605	Cover to seat cushion SB30 ST27,5 black		1
61	1516606	Cover to seat cushion SB32 ST27,5 black		1
62	1516607	Cover to seat cushion SB34 ST27,5 black		1
63	1516608	Cover to seat cushion SB36 ST27,5 black		1
64	1516616	Cover to seat cushion SB24 ST30 black		1
65	1516617	Cover to seat cushion SB26 ST30 black		1
66	1516618	Cover to seat cushion SB28 ST30 black		1
67	1516619	Cover to seat cushion SB30 ST30 black		1
68	1516620	Cover to seat cushion SB32 ST30 black		1
69	1516621	Cover to seat cushion SB34 ST30 black		1
70	1516622	Cover to seat cushion SB36 ST30 black		1
71	1516672	Cover to seat cushion SB24 ST32,5 black		1
72	1516673	Cover to seat cushion SB26 ST32,5 black		1
73	1516674	Cover to seat cushion SB28 ST32,5 black		1
74	1516675	Cover to seat cushion SB30 ST32,5 black		1
75	1516700	Cover to seat cushion SB32 ST32,5 black		1
76	1516701	Cover to seat cushion SB34 ST32,5 black		1
77	1516702	Cover to seat cushion SB36 ST32,5 black		1
78	1516729	Cover to seat cushion SB24 ST35 black		1
79	1516730	Cover to seat cushion SB26 ST35 black		1
80	1516742	Cover to seat cushion SB28 ST35 black		1
81	1516743	Cover to seat cushion SB30 ST35 black		1
82	1516744	Cover to seat cushion SB32 ST35 black		1
83	1516756	Cover to seat cushion SB34 ST35 black		1
84	1516757	Cover to seat cushion SB36 ST35 black		1
85	1516823	Cover to seat cushion SB24 ST37,5 black		1
86	1516827	Cover to seat cushion SB26 ST37,5 black		1


**Seat cushion**

<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
87	1516832	Cover to seat cushion SB28 ST37,5 black		1
88	1516836	Cover to seat cushion SB30 ST37,5 black		1
89	1516840	Cover to seat cushion SB32 ST37,5 black		1
90	1516975	Cover to seat cushion SB34 ST37,5 black		1
91	1516979	Cover to seat cushion SB36 ST37,5 black		1
92	1516686	Cover to seat cushion SB24 ST40 black		1
93	1516687	Cover to seat cushion SB26 ST40 black		1
94	1516688	Cover to seat cushion SB28 ST40 black		1
95	1516689	Cover to seat cushion SB30 ST40 black		1
96	1516690	Cover to seat cushion SB32 ST40 black		1
97	1516691	Cover to seat cushion SB34 ST40 black		1
98	1516692	Cover to seat cushion SB36 ST40 black		1

# BACKREST

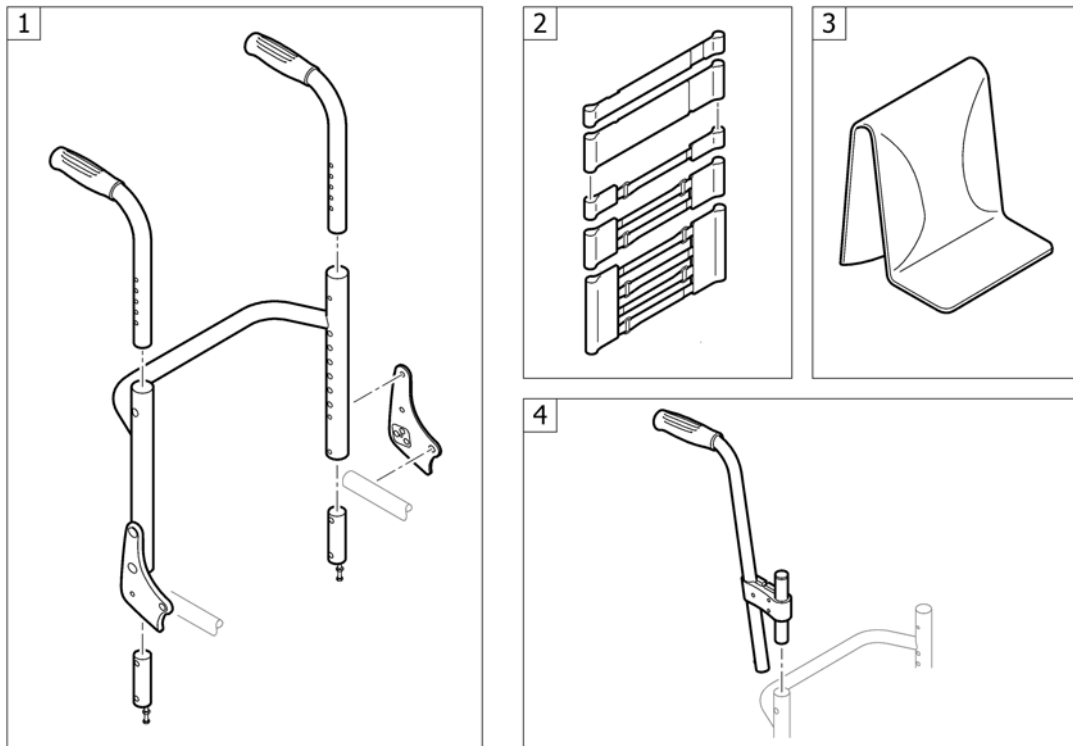

Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrest		1
2	-	Bands to adjustable backrest		1
3	-	Cover to adjustable backrest		1
4	-	Push handles, height adjustable, rear set		1

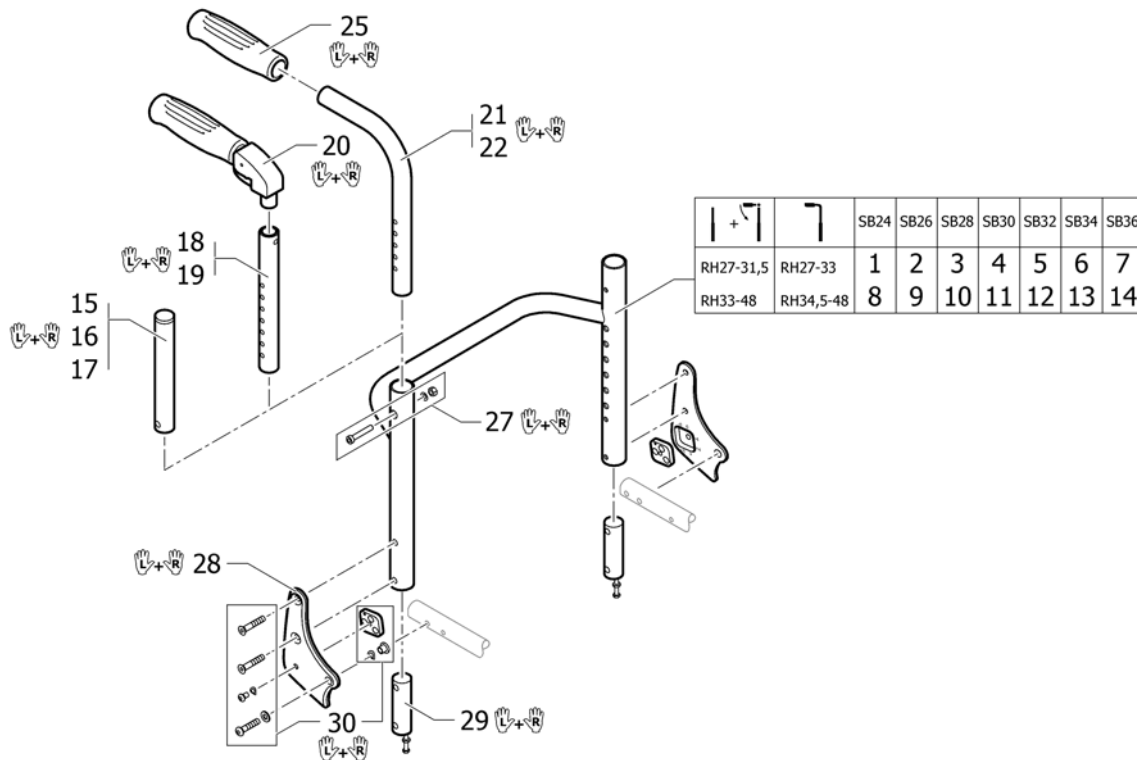


Pos.	Part number	Description	Valid from → until	Qty
1	1451936	Backrest tube Alu SB24 short		1
2	1451937	Backrest tube Alu SB26 short		1
3	1451938	Backrest tube Alu SB28 short		1
4	1451939	Backrest tube Alu SB30 short		1
5	1483905	Backrest tube Alu SB32 short		1
6	1491119	Backrest tube Alu SB34 short		1
7	1491120	Backrest tube Alu SB36 short		1
8	1484392	Backrest tube Alu SB24 long		1
9	1484393	Backrest tube Alu SB26 long		1
10	1484605	Backrest tube Alu SB28 long		1
11	1484606	Backrest tube Alu SB30 long		1
12	1484607	Backrest tube Alu SB32 long		1
13	1491091	Backrest tube Alu SB34 Long		1
14	1491092	Backrest tube Alu SB36 Long		1
15	1511524	Telescopic back tube standard short, pair		1
16	1511525	Telescopic back tube standard medium, pair		1
17	1511526	Telescopic back tube standard long, pair		1
18	1524956	Telescopic tube short for pushhandle foldable, pair		1
19	1525121	Telescopic tube medium for pushhandle foldable, pair		1
20	1507733	Push handles foldable, pair		1
21	1525122	Backrest tube standard, short, pair		1
22	1525123	Backrest tube standard, long, pair		1
25	1507734	Handle black, pair		1
27	1509136	Screw kit for backrest tube		1
28	1487963-0004	Backrest joint plate, pair, black anodized		1
28	1487963-0400	Backrest joint plate, pair, techno-anthrazit		1
28	1487963-0401	Backrest joint plate, pair, lemon		1
28	1487963-0402	Backrest joint plate, pair, apricot		1
28	1487963-0403	Backrest joint plate, pair, fashion violett	→ 20070630	1
28	1487963-0404	Backrest joint plate, pair, b-blue		1
28	1487963-0440	Backrest joint plate, pair, comet red	20070101 →	1

**Backrest**

<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
29	1487962	Sleeves for backrest tube alu, pair complete		1
30	1487964	Fixation kit to backrest joint		1




## Bands to adjustable backrest

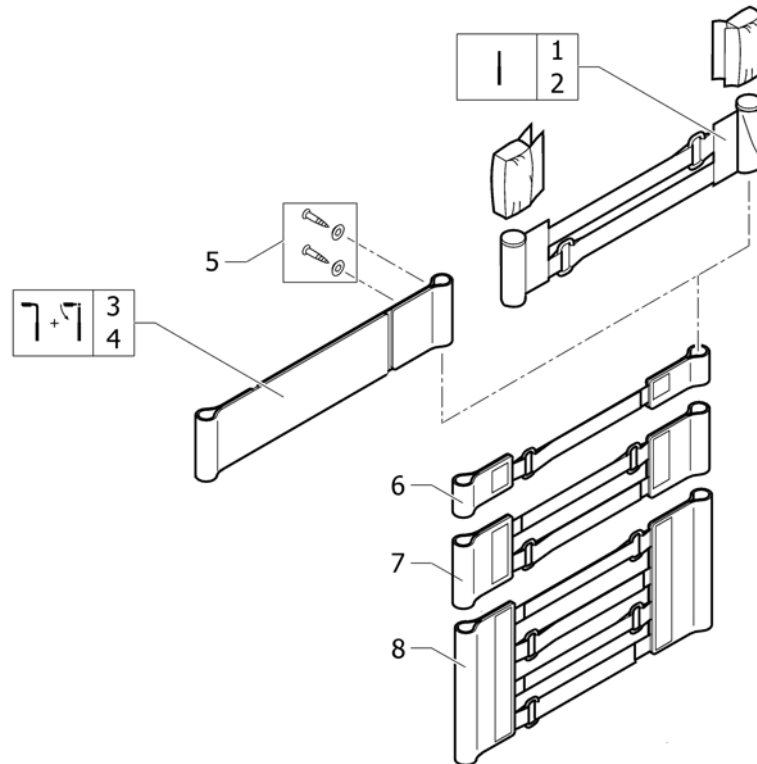


Pos.	Part number	Description	Valid from → until	Qty
1	1513172	Velcro end part, 5 cm <i>Replaces 1416975</i>		1
2	1513173	Velcro end part, 10cm <i>Replaces 1416974</i>		1
3	1513170	Backrest band 2 parts, width: 5cm <i>Replaces 709.1009</i>		1
4	1513171	Backrest band 2 parts, width: 10cm <i>Replaces 709.1018</i>		1
5	1432083	Fixation kit for backrest band		1
6	1513167	Velcro adjustment part with 1 strap, pair <i>Replaces 1439291</i>		1
7	1513168	Velcro adjustment strap with 2 straps, pair <i>Replaces 707.1088</i>		1
8	1513169	Velcro adjustment strap with 4 straps, pair <i>Replaces 707.1087</i>		1

# Cover to adjustable backrest

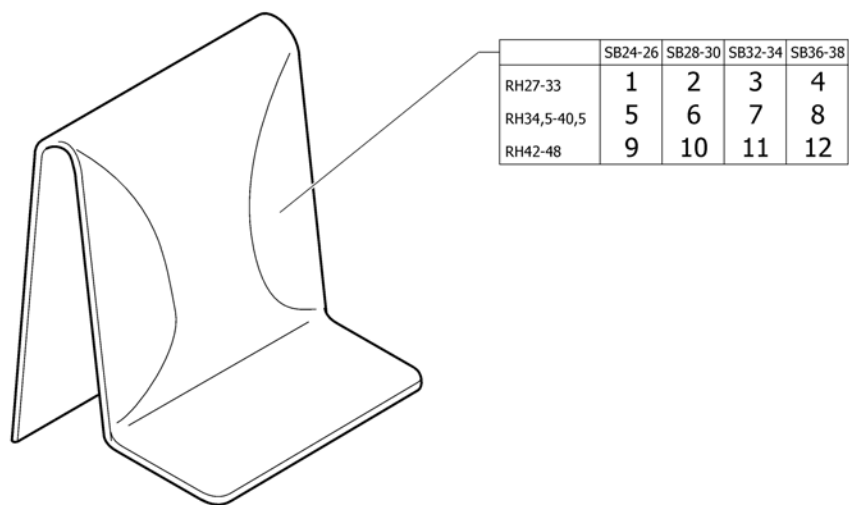






Pos.	Part number	Description	Valid from → until	Qty
1	1512135	Cover to adjustable backrest, SB24-26, short <i>Replaces 1485319</i>		1
2	1512136	Cover to adjustable backrest, SB28-30, short <i>Replaces 1485321</i>		1
3	1512137	Cover to adjustable backrest, SB32-34, short <i>Replaces 1438884</i>		1
4	1512138	Cover to adjustable backrest, SB36-38, short <i>Replaces 1438886</i>		1
5	1512142	Cover to adjustable backrest, SB24-26, medium <i>Replaces 1485324</i>		1
6	1512143	Cover to adjustable backrest, SB28-30, medium <i>Replaces 1485326</i>		1
7	1512144	Cover to adjustable backrest, SB32-34, medium <i>Replaces 1432010</i>		1
8	1512145	Cover to adjustable backrest, SB36-38, medium <i>Replaces 1432012</i>		1
9	1512149	Cover to adjustable backrest, SB24-26, long <i>Replaces 1485614</i>		1
10	1512150	Cover to adjustable backrest, SB28-30, long <i>Replaces 1485616</i>		1
11	1512151	Cover to adjustable backrest, SB32-34, long <i>Replaces 1432016</i>		1
12	1512152	Cover to adjustable backrest, SB36-38, long <i>Replaces 1432018</i>		1

## Push handles, height adjustable, rear set

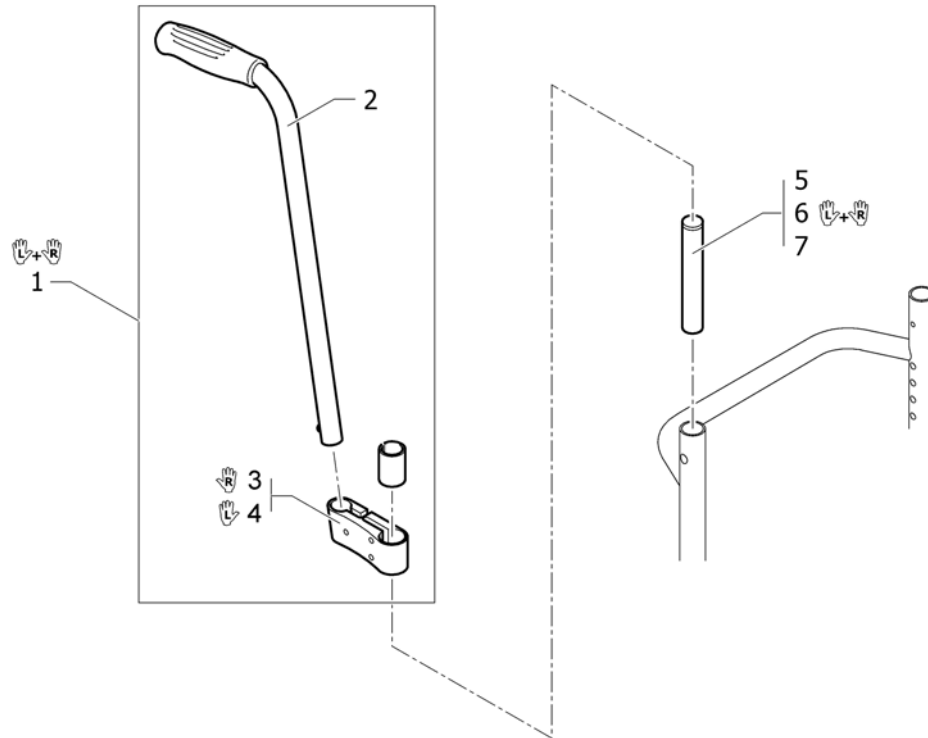


Pos.	Part number	Description	Valid from → until	Qty
1	1517876	Push handles, height adjustable, rear set, pair <i>Replaces 907.1157</i>		1
2	1438938	Push handle tube complete, rearset		2
3	1438939	Mounting fixture right complete, for rearset push handles		1
4	1438940	Mounting fixture left complete, for rearset push handles		1
5	1511524	Telescopic back tube standard short, pair		1
6	1511525	Telescopic back tube standard medium, pair		1
7	1511526	Telescopic back tube standard long, pair		1

# FOOTREST

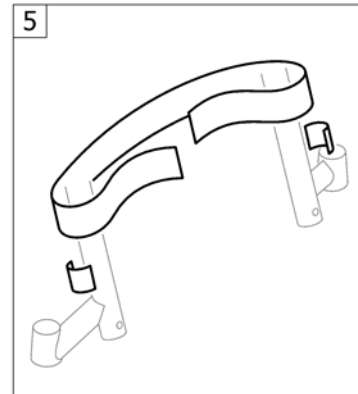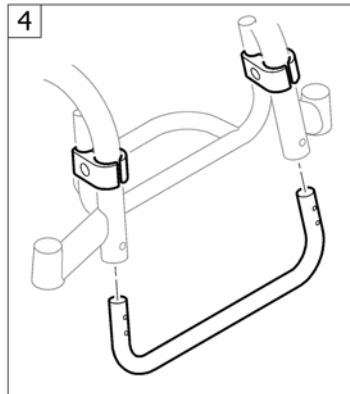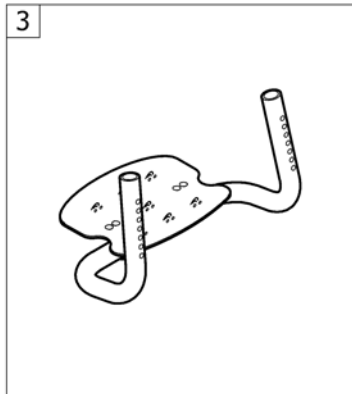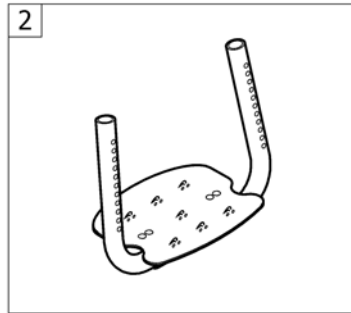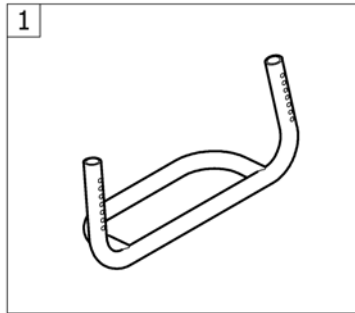


Pos.	Part number	Description	Valid from → until	Qty
1	-	Footrest standard		1
2	-	Footrest angle adjustable		1
3	-	Footrest retracted		1
4	-	Footrest kit, high mounted		1
5	-	Calf strap		1

# Footrest standard

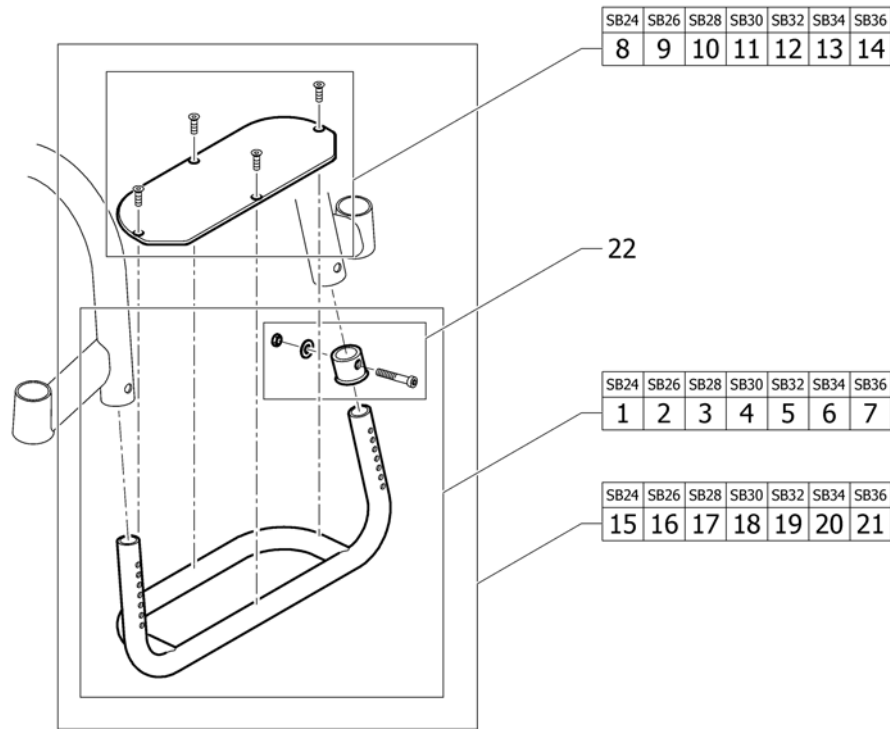

Pos.	Part number	Description	Valid from → until	Qty
1	1485739	Footrest Alu complete SB24 Junior		1
2	1485740	Footrest Alu complete SB26 Junior		1
3	1485741	Footrest Alu complete SB28 Junior		1
4	1485742	Footrest Alu complete SB30 Junior		1
5	1485743	Footrest Alu complete SB32 Junior		1
6	1485744	Footrest Alu complete SB34 Junior		1
7	1485745	Footrest Alu complete SB36 Junior		1
8	1485746	Plate SB24 for Alu Footrest Junior		1
9	1485747	Plate SB26 for Alu Footrest Junior		1
10	1485748	Plate SB28 for Alu Footrest Junior		1
11	1485749	Plate SB30 for Alu Footrest Junior		1
12	1485750	Plate SB32 for Alu Footrest Junior		1
13	1485751	Plate SB34 for Alu Footrest Junior		1
14	1485752	Plate SB36 for Alu Footrest Junior		1
15	1517810	Footrest Alu with plate complete SB24 Junior		1
16	1517811	Footrest Alu with plate complete SB26 Junior		1
17	1517812	Footrest Alu with plate complete SB28 Junior		1
18	1517813	Footrest Alu with plate complete SB30 Junior		1
19	1517814	Footrest Alu with plate complete SB32 Junior		1
20	1517815	Footrest Alu with plate complete SB34 Junior		1
21	1517816	Footrest Alu with plate complete SB36 Junior		1
22	1451716	Footplate sleeve Alu		1

# Footrest angle adjustable

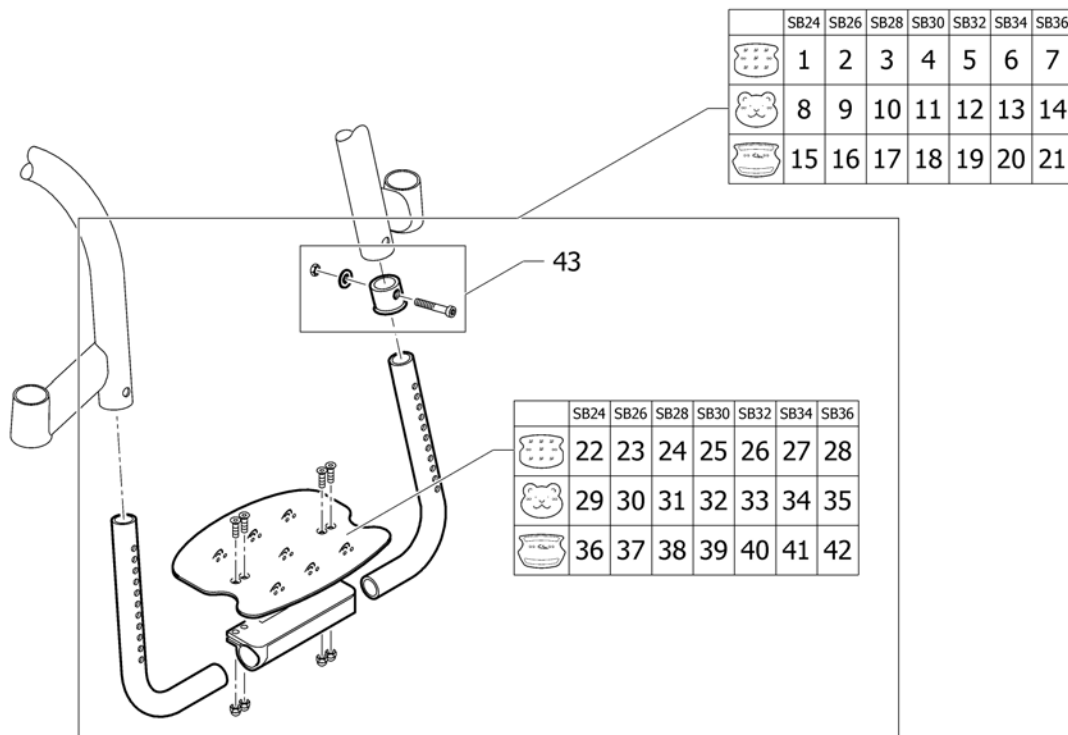


Pos.	Part number	Description	Valid from → until	Qty
1	1486035	Footrest smiley SB24 complete		1
2	1487925	Footrest smiley SB26 complete		1
3	1487926	Footrest smiley SB28 complete		1
4	1487927	Footrest smiley SB30 complete		1
5	1487928	Footrest smiley SB32 complete		1
6	1487929	Footrest smiley SB34 complete		1
7	1487930	Footrest smiley SB36 complete		1
8	1486028	Footrest SB24 Teddy Bear complete		1
9	1486029	Footrest SB26 Teddy Bear complete		1
10	1486030	Footrest SB28 Teddy Bear complete		1
11	1486031	Footrest SB30 Teddy Bear complete		1
12	1486032	Footrest SB32 Teddy Bear complete		1
13	1486033	Footrest SB34 Teddy Bear complete		1
14	1486034	Footrest SB36 Teddy Bear complete		1
15	1487931	Footrest Carbon SB24 complete		1
16	1487932	Footrest Carbon SB26 complete		1
17	1487933	Footrest Carbon SB28 complete		1
18	1487934	Footrest Carbon SB30 complete		1
19	1487935	Footrest Carbon SB32 complete		1
20	1487936	Footrest Carbon SB34 complete		1
21	1487937	Footrest Carbon SB36 complete		1
22	1485760	Plate smiley SB24		1
23	1485761	Plate smiley SB26		1
24	1485762	Plate smiley SB28		1
25	1485763	Plate smiley SB30		1
26	1485764	Plate smiley SB32		1
27	1485765	Plate smiley SB34		1
28	1485766	Plate smiley SB36		1
29	1485753	Plate SB24 Teddy bear		1
30	1485754	Plate SB26 Teddy bear		1
31	1485755	Plate SB28 Teddy bear		1

**Footrest angle adjustable**

<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
32	1485756	Plate SB30 Teddy bear		1
33	1485757	Plate SB32 Teddy bear		1
34	1485758	Plate SB34 Teddy bear		1
35	1485759	Plate SB36 Teddy bear		1
36	1485767	Plate Carbon SB24		1
37	1485768	Plate Carbon SB26		1
38	1485769	Plate Carbon SB28		1
39	1485770	Plate Carbon SB30		1
40	1485771	Plate Carbon SB32		1
41	1485772	Plate Carbon SB34		1
42	1485773	Plate Carbon SB36		1
43	1451716	Footplate sleeve Alu		1

# Footrest retracted

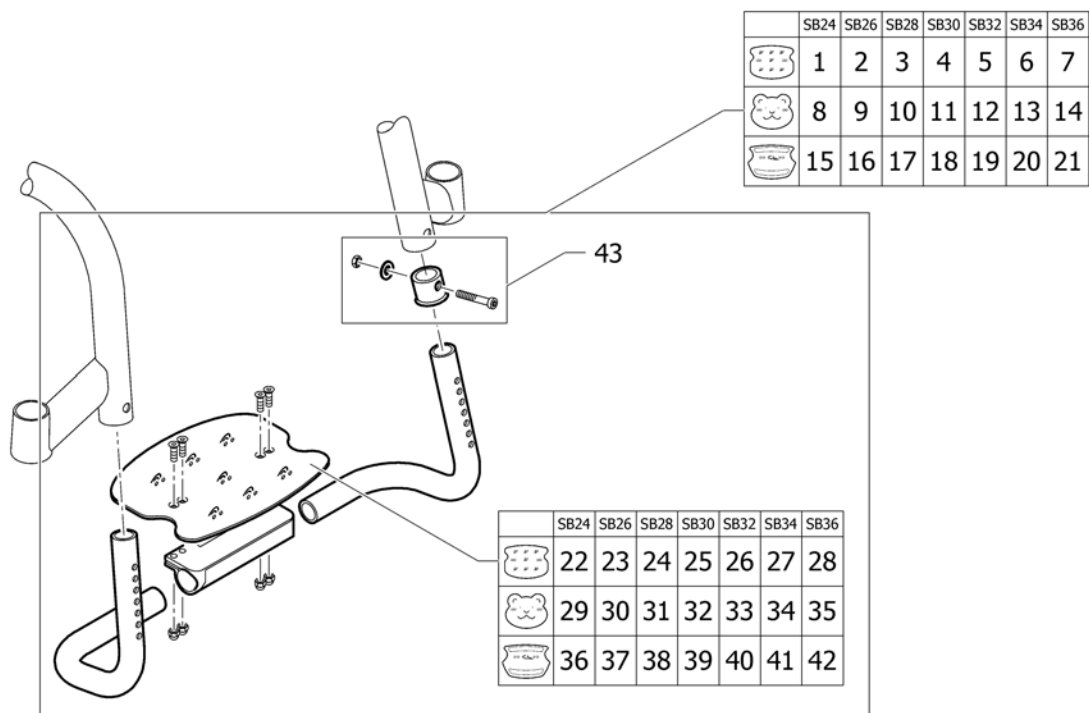


Pos.	Part number	Description	Valid from → until	Qty
1	1488088	Footrest smiley SB24 inclined complete		1
2	1488089	Footrest smiley SB26 inclined complete		1
3	1488090	Footrest smiley SB28 inclined complete		1
4	1488091	Footrest smiley SB30 inclined complete		1
5	1488092	Footrest smiley SB32 inclined complete		1
6	1488093	Footrest smiley SB34 inclined complete		1
7	1488094	Footrest smiley SB36 inclined complete		1
8	1488081	Footrest Teddy Bear SB24 inclined complete		1
9	1488082	Footrest Teddy Bear SB26 inclined complete		1
10	1488083	Footrest Teddy Bear SB28 inclined complete		1
11	1488084	Footrest Teddy Bear SB30 inclined complete		1
12	1488085	Footrest Teddy Bear SB32 inclined complete		1
13	1488086	Footrest Teddy Bear SB34 inclined complete		1
14	1488087	Footrest Teddy Bear SB36 inclined complete		1
15	1488095	Footrest Carbon SB24 inclined complete		1
16	1488162	Footrest Carbon SB26 inclined complete		1
17	1488163	Footrest Carbon SB28 inclined complete		1
18	1488164	Footrest Carbon SB30 inclined complete		1
19	1488165	Footrest Carbon SB32 inclined complete		1
20	1488166	Footrest Carbon SB34 inclined complete		1
21	1488167	Footrest Carbon SB36 inclined complete		1
22	1485760	Plate smiley SB24		1
23	1485761	Plate smiley SB26		1
24	1485762	Plate smiley SB28		1
25	1485763	Plate smiley SB30		1
26	1485764	Plate smiley SB32		1
27	1485765	Plate smiley SB34		1
28	1485766	Plate smiley SB36		1
29	1485753	Plate SB24 Teddy bear		1
30	1485754	Plate SB26 Teddy bear		1
31	1485755	Plate SB28 Teddy bear		1




**Footrest retracted**

<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
32	1485756	Plate SB30 Teddy bear		1
33	1485757	Plate SB32 Teddy bear		1
34	1485758	Plate SB34 Teddy bear		1
35	1485759	Plate SB36 Teddy bear		1
36	1485767	Plate Carbon SB24		1
37	1485768	Plate Carbon SB26		1
38	1485769	Plate Carbon SB28		1
39	1485770	Plate Carbon SB30		1
40	1485771	Plate Carbon SB32		1
41	1485772	Plate Carbon SB34		1
42	1485773	Plate Carbon SB36		1
43	1451716	Footplate sleeve Alu		1

# Footrest kit, high mounted

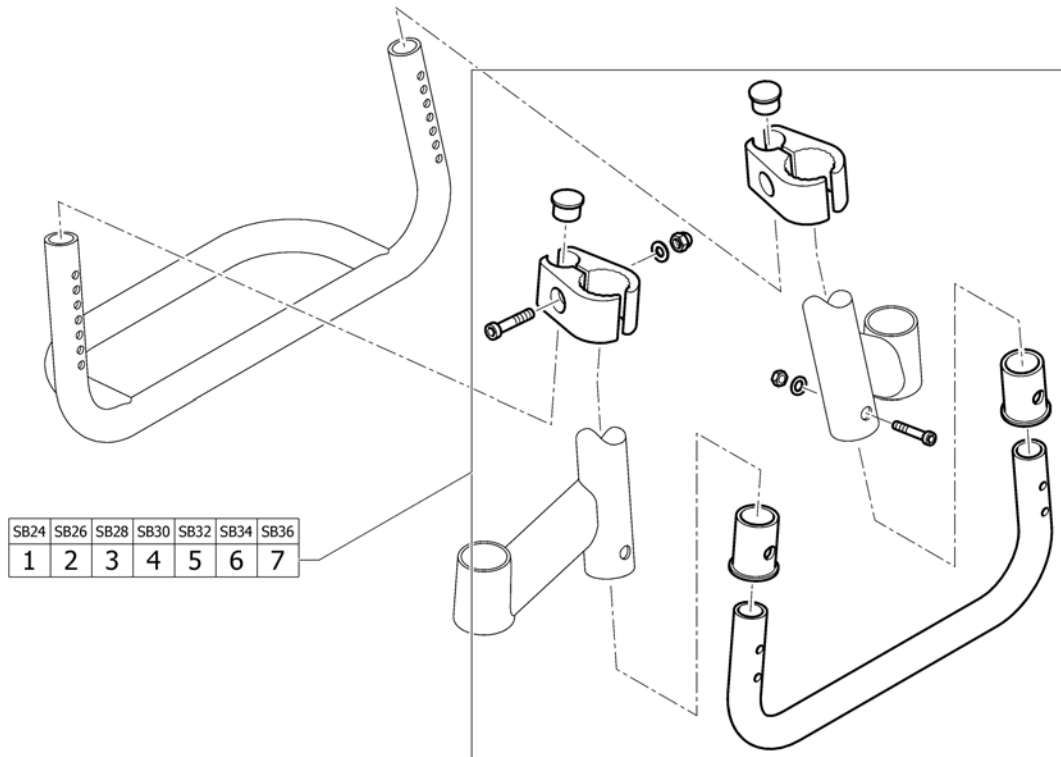


Pos.	Part number	Description	Valid from → until	Qty
1	1486006	Fixation kit for footrest mounted in high position SB24		1
2	1486007	Fixation kit for footrest mounted in high position SB26		1
3	1486008	Fixation kit for footrest mounted in high position SB28		1
4	1486009	Fixation kit for footrest mounted in high position SB30		1
5	1486010	Fixation kit for footrest mounted in high position SB32		1
6	1486011	Fixation kit for footrest mounted in high position SB34		1
7	1486012	Fixation kit for footrest mounted in high position SB36		1

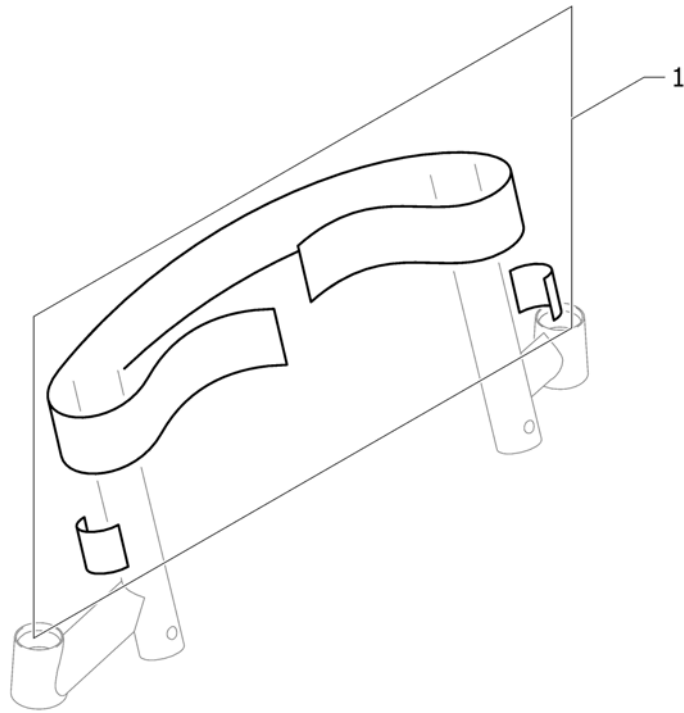


Pos.	Part number	Description	Valid from → until	Qty
1	1517817	Calfstrap black <i>Replaces 709.1008</i>		1

# SIDERESTS

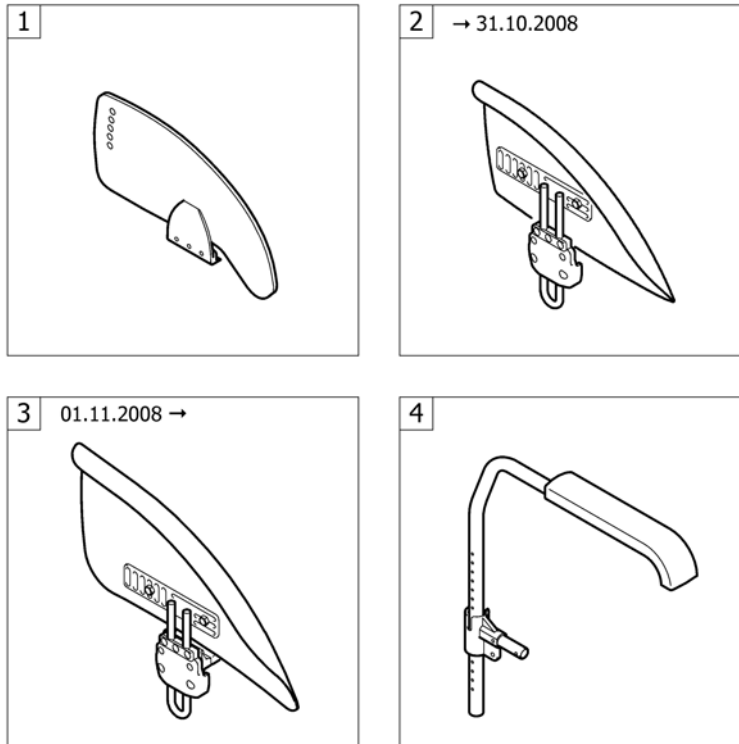


Pos.	Part number	Description	Valid from → until	Qty
1	-	Clothes-guards		1
2	-	Mudguards =>31.10.2008	→ 20081031	1
3	-	Mudguards 01.11.2008=>	20081101 →	1
4	-	Tubular armrest		1

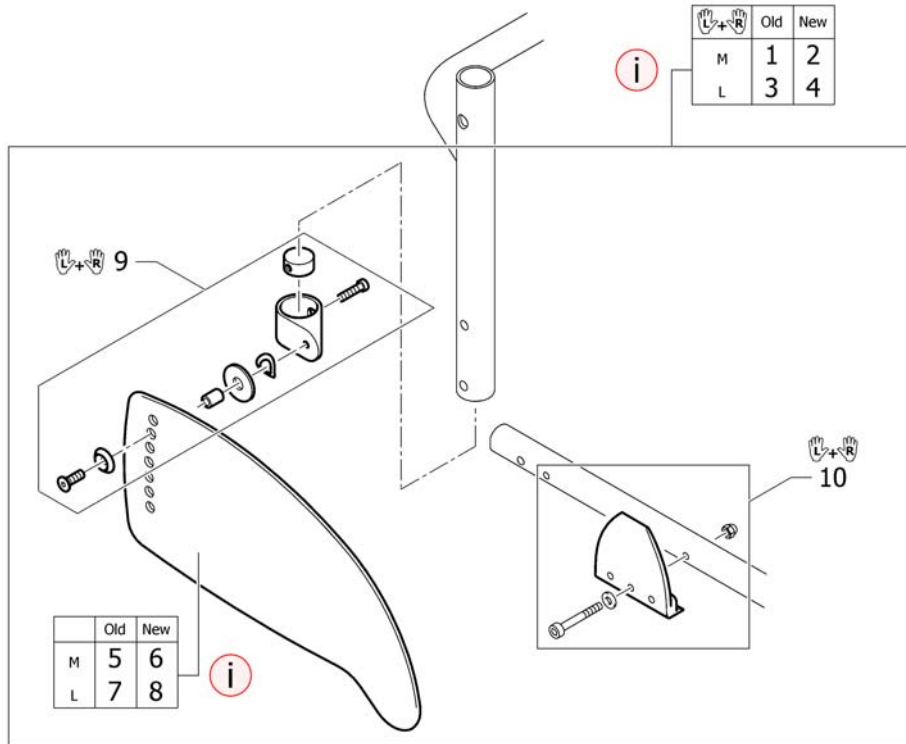


Pos.	Part number	Description	Valid from → until	Qty
i	-	New design with improved profile ensures better coverage. Recommended as a pair.		0
1	1517833	Clothes-guards Carbon M with fixation kit to backrest Aluminium, pair	20060101 → 20090228	1
2	1527104	Clothes-guards Carbon M with fixation kit to backrest Aluminium, pair	20090301 →	1
3	1517834	Clothes-guards Carbon L with fixation kit to backrest Aluminium, pair	20060101 → 20081210	1
4	1527105	Clothes-guards Carbon L with fixation kit to backrest Aluminium, pair	20081211 →	1
5	1451719	Clothes-guard Carbon M	20060101 → 20090228	2
6	1527100	Clothes-guard Carbon M	20090301 →	2
7	1451720	Clothes-guard Carbon L	20060101 → 20081210	2
8	1527101	Clothes-guard Carbon L	20081211 →	2
9	1451722	Fixation kit clothes-guard to backrest Aluminium		1
10	1486016	Fixation clothes-guard to frame, pair		1

# Mudguards =>31.10.2008

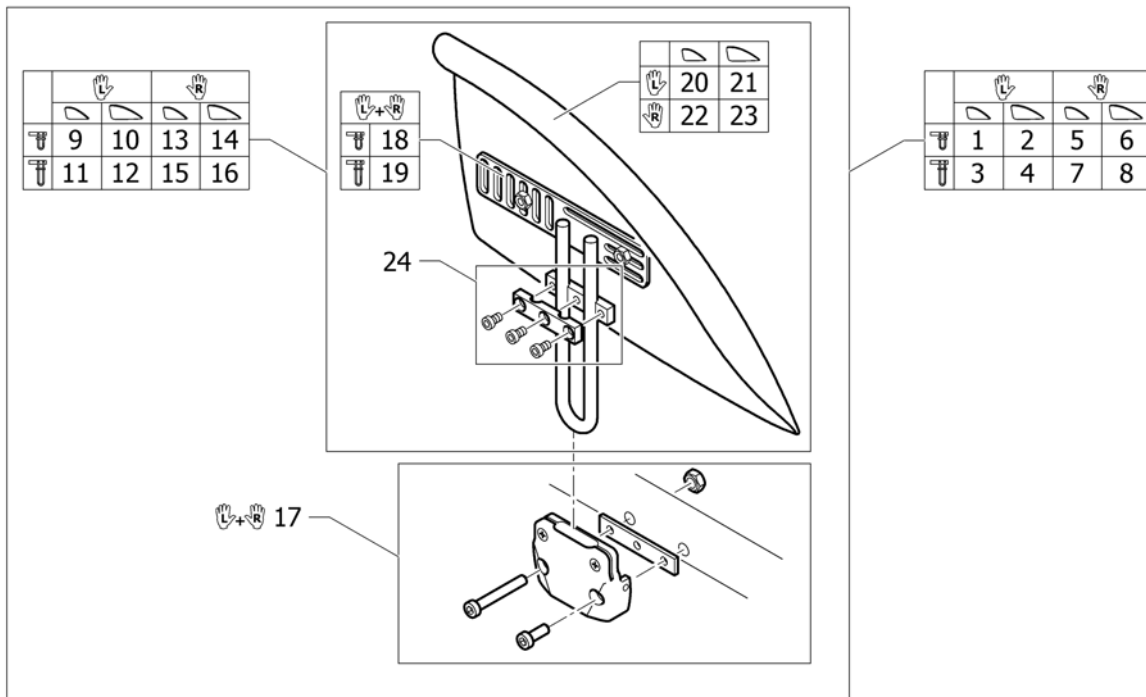


Pos.	Part number	Description	Valid from → until	Qty
1	1488460	Mudguard Carbon left M short complete	→ 20081031	1
2	1488461	Mudguard Carbon left L short complete	→ 20081031	1
3	1488464	Mudguard Carbon left M long complete	→ 20081031	1
4	1488465	Mudguard Carbon left L long complete	→ 20081031	1
5	1488462	Mudguard Carbon right M short complete	→ 20081031	1
6	1488463	Mudguard Carbon right L short complete	→ 20081031	1
7	1488873	Mudguard Carbon right M long complete	→ 20081031	1
8	1488728	Mudguard Carbon right L long complete	→ 20081031	1
9	1451723	Mudguard Carbon left M short	→ 20081031	1
10	1451724	Mudguard Carbon left L short	→ 20081031	1
11	1488729	Mudguard Carbon left M long	→ 20081031	1
12	1488730	Mudguard Carbon left L long	→ 20081031	1
13	1451725	Mudguard Carbon right M short	→ 20081031	1
14	1451726	Mudguard Carbon right L short	→ 20081031	1
15	1488731	Mudguard Carbon right M long	→ 20081031	1
16	1488732	Mudguard Carbon right L long	→ 20081031	1
17	1451727	Fixationset mudguard frame		1
18	1451728	Mudguard holder short, pair		1
19	1488459	Mudguard holder long, pair		1
20	1451749	Mudguard Carbon left M	→ 20081031	1
21	1451750	Mudguard Carbon left L	→ 20081031	1
22	1451751	Mudguard Carbon right M	→ 20081031	1
23	1451752	Mudguard Carbon right L	→ 20081031	1
24	1437465	Siderests height clamp kit, pair		1

## Mudguards 01.11.2008=&gt;

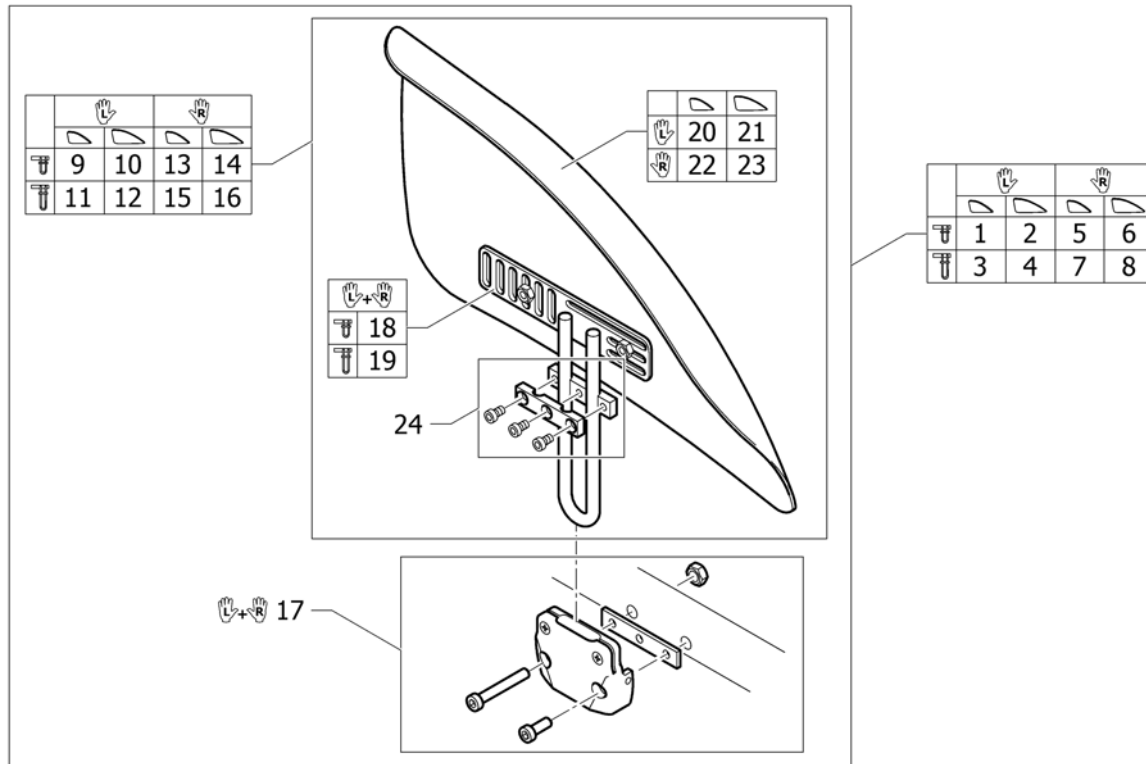




Pos.	Part number	Description	Valid from → until	Qty
1	1529308	Mudguard Carbon left M short complete	20081101 →	1
2	1529309	Mudguard Carbon left L short complete	20081101 →	1
3	1529312	Mudguard Carbon left M long complete	20081101 →	1
4	1529313	Mudguard Carbon left L long complete	20081101 →	1
5	1529310	Mudguard Carbon right M short complete	20081101 →	1
6	1529311	Mudguard Carbon right L short complete	20081101 →	1
7	1529314	Mudguard Carbon right M long complete	20081101 →	1
8	1529315	Mudguard Carbon right L long complete	20081101 →	1
9	1529300	Mudguard Carbon left M short	20081101 →	1
10	1529301	Mudguard Carbon left L short	20081101 →	1
11	1529304	Mudguard Carbon left M long	20081101 →	1
12	1529305	Mudguard Carbon left L long	20081101 →	1
13	1529302	Mudguard Carbon right M short	20081101 →	1
14	1529303	Mudguard Carbon right L short	20081101 →	1
15	1529306	Mudguard Carbon right M long	20081101 →	1
16	1529307	Mudguard Carbon right L long	20081101 →	1
17	1451727	Fixationset mudguard frame		1
18	1451728	Mudguard holder short, pair		1
19	1488459	Mudguard holder long, pair		1
20	1510275	Mudguard Carbon left M	20081101 →	1
21	1510277	Mudguard Carbon left L	20081101 →	1
22	1510276	Mudguard Carbon right M	20081101 →	1
23	1510278	Mudguard Carbon right L	20081101 →	1
24	1437465	Siderests height clamp kit, pair		1

# Tubular armrest

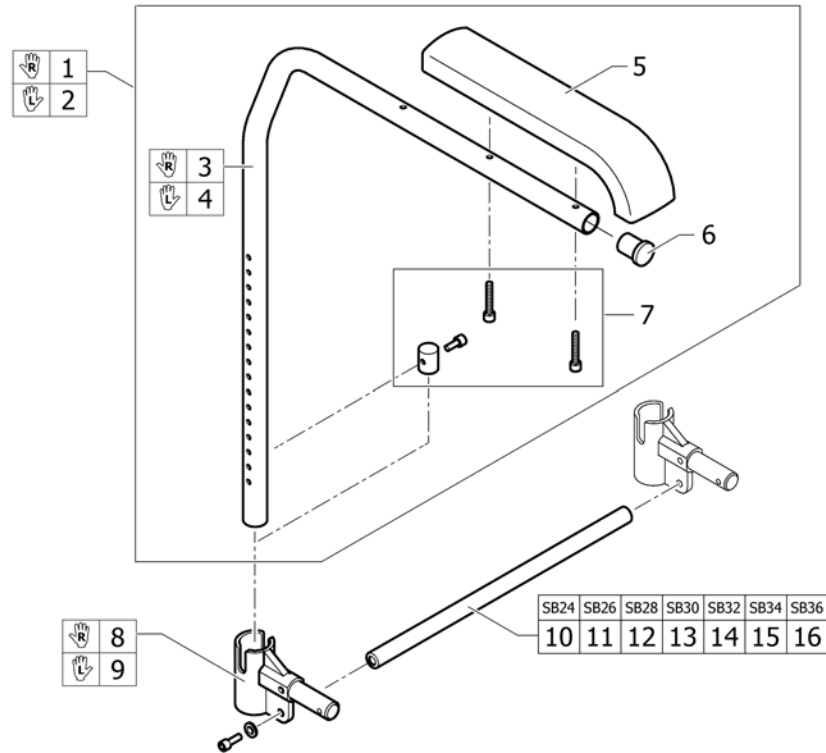


Pos.	Part number	Description	Valid from → until	Qty
1	913.1012	Armrest with standard cushion, right		1
2	913.1013	Armrest, with standard cushion, left		1
3	456.1111.956	Armrest tube, right		1
4	456.1112.956	Armrest tube, left		1
5	702.1004	Standard cushion, bent		1
6	220.1013	Tube plug Ø24mm		1
7	1451729	Boltset Armrest		1
8	1451730	Armrest adapter right incl. Fixation and cord		1
9	1451731	Armrest adapter left incl. Fixation and cord		1
10	1486017	Connection tube for armrest SB24		1
11	1486018	Connection tube for armrest SB26		1
12	1486019	Connection tube for armrest SB28		1
13	1486020	Connection tube for armrest SB30		1
14	1486021	Connection tube for armrest SB32		1
15	1451732	Connection tube for armrest SB34		1
16	1451733	Connection tube for armrest SB36		1



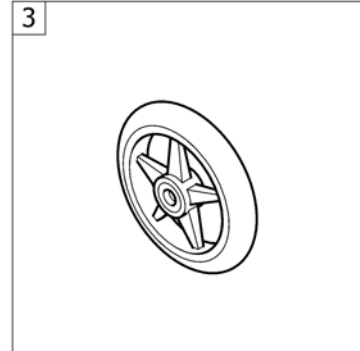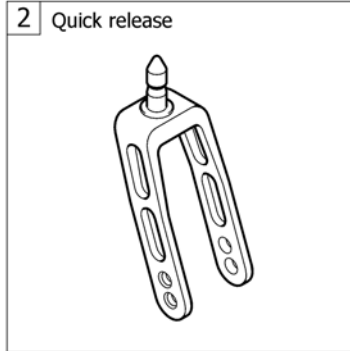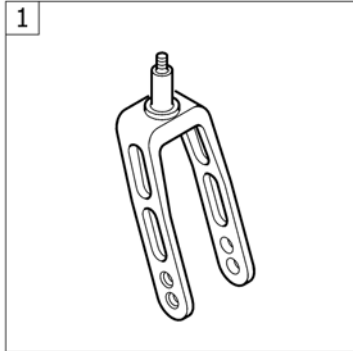

Pos.	Part number	Description	Valid from → until	Qty
1	-	Front forks standard		1
2	-	Front forks quick release		1
3	-	Front wheels		1

## Front forks standard

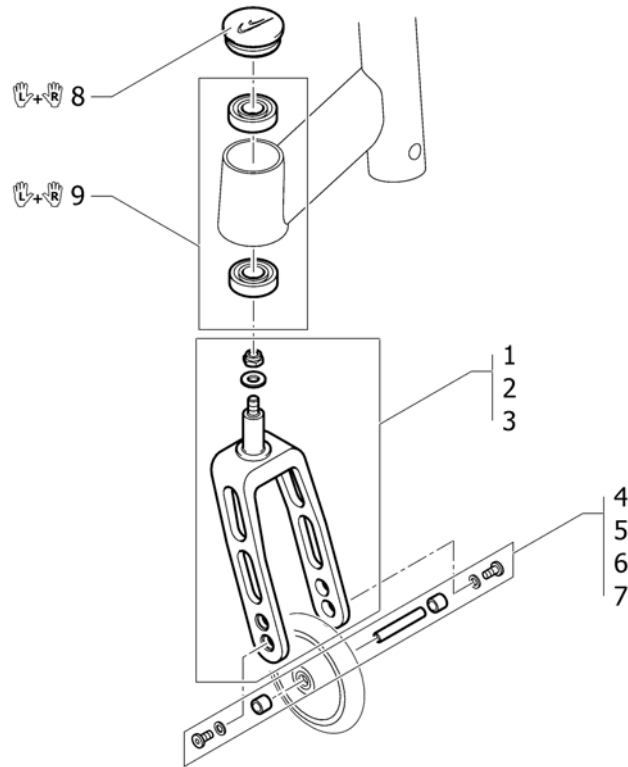


Pos.	Part number	Description	Valid from → until	Qty
1	1432004-0004	Front fork 5" with bolt, black anodized		2
1	1432004-0400	Front fork 5" with bolt, techno-anthrazit		2
1	1432004-0401	Front fork 5" with bolt, lemon		2
1	1432004-0402	Front fork 5" with bolt, apricot		2
1	1432004-0403	Front fork 5" with bolt, fashion violett	→ 20070630	2
1	1432004-0404	Front fork 5" with bolt, b-blue		2
1	1432004-0440	Front fork 5" with bolt, comet red	20070101 →	2
2	1432005-0004	Front fork 6" with bolt, black anodized		2
2	1432005-0400	Front fork 6" with bolt, techno-anthrazit		2
2	1432005-0401	Front fork 6" with bolt, lemon		2
2	1432005-0402	Front fork 6" with bolt, apricot		2
2	1432005-0403	Front fork 6" with bolt, fashion violett	→ 20070630	2
2	1432005-0404	Front fork 6" with bolt, b-blue		2
2	1432005-0440	Front fork 6" with bolt, comet red	20070101 →	2
3	1432006-0004	Front fork 8" with bolt, black anodized		2
3	1432006-0400	Front fork 8" with bolt, techno-anthrazit		2
3	1432006-0401	Front fork 8" with bolt, lemon		2
3	1432006-0402	Front fork 8" with bolt, apricot		2
3	1432006-0403	Front fork 8" with bolt, fashion violett	→ 20070630	2
3	1432006-0404	Front fork 8" with bolt, b-blue		2
3	1432006-0440	Front fork 8" with bolt, comet red	20070101 →	2
4	1420972	Axle set complete Starec/Skater wheel		2
5	1420973	Axle set complete 4" sports wheel		2
6	1420974	Axle set complete 5"/6" sports wheel		2
7	1420975	Axle set complete 5" soft roll wheel		2
8	1487961-0004	Bearing block cap pair, black anodized		1
8	1487961-0400	Bearing block cap pair, techno-anthrazit		1
8	1487961-0401	Bearing block cap pair, lemon		1
8	1487961-0402	Bearing block cap pair, apricot		1
8	1487961-0403	Bearing block cap pair, fashion violett	→ 20070630	1
8	1487961-0404	Bearing block cap pair, b-blue		1


**Front forks standard**

<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
8	1487961-0440	Bearing block cap pair, comet red	20070101 →	1
9	1451718	Bearing kit		1

## Front forks quick release

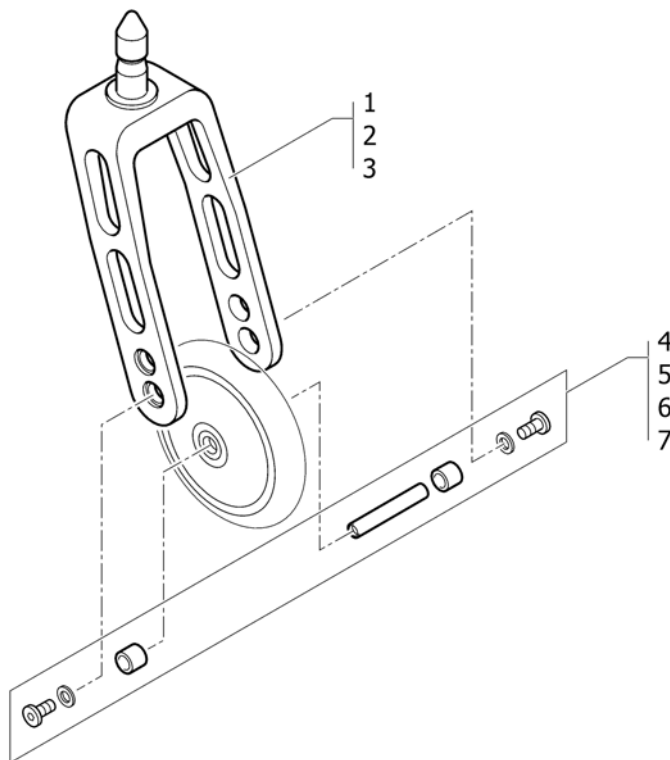







Pos.	Part number	Description	Valid from → until	Qty
1	1486025-0004	Front fork 5", quick release, complete K05, black anodized		2
1	1486025-0400	Front fork 5", quick release, complete K05, techno-anthrazit		2
1	1486025-0401	Front fork 5", quick release, complete K05, lemon		2
1	1486025-0402	Front fork 5", quick release, complete K05, apricot		2
1	1486025-0403	Front fork 5", quick release, complete K05, fashion violett	→ 20070630	2
1	1486025-0404	Front fork 5", quick release, complete K05, b-blue		2
1	1486025-0440	Front fork 5", quick release, complete, comet red	20070101 →	2
2	1486026-0004	Front fork 6", quick release, complete K05, black anodized		2
2	1486026-0400	Front fork 6", quick release, complete K05, techno-anthrazit		2
2	1486026-0401	Front fork 6", quick release, complete K05, lemon		2
2	1486026-0402	Front fork 6", quick release, complete K05, apricot		2
2	1486026-0403	Front fork 6", quick release, complete K05, fashion violett	→ 20070630	2
2	1486026-0404	Front fork 6", quick release, complete K05, b-blue		2
2	1486026-0440	Front fork 6", quick release, complete, comet red	20070101 →	2
3	1486027-0004	Front fork 8", quick release, complete K05, black anodized		2
3	1486027-0400	Front fork 8", quick release, complete K05, techno-anthrazit		2
3	1486027-0401	Front fork 8", quick release, complete K05, lemon		2
3	1486027-0402	Front fork 8", quick release, complete K05, apricot		2
3	1486027-0403	Front fork 8", quick release, complete K05, fashion violett	→ 20070630	2
3	1486027-0404	Front fork 8", quick release, complete K05, b-blue		2
3	1486027-0440	Front fork 8", quick release, complete, comet red	20070101 →	2
4	1420972	Axle set complete Starec/Skater wheel		2
5	1420973	Axle set complete 4" sports wheel		2
6	1420974	Axle set complete 5"/6" sports wheel		2
7	1420975	Axle set complete 5" soft roll wheel		2

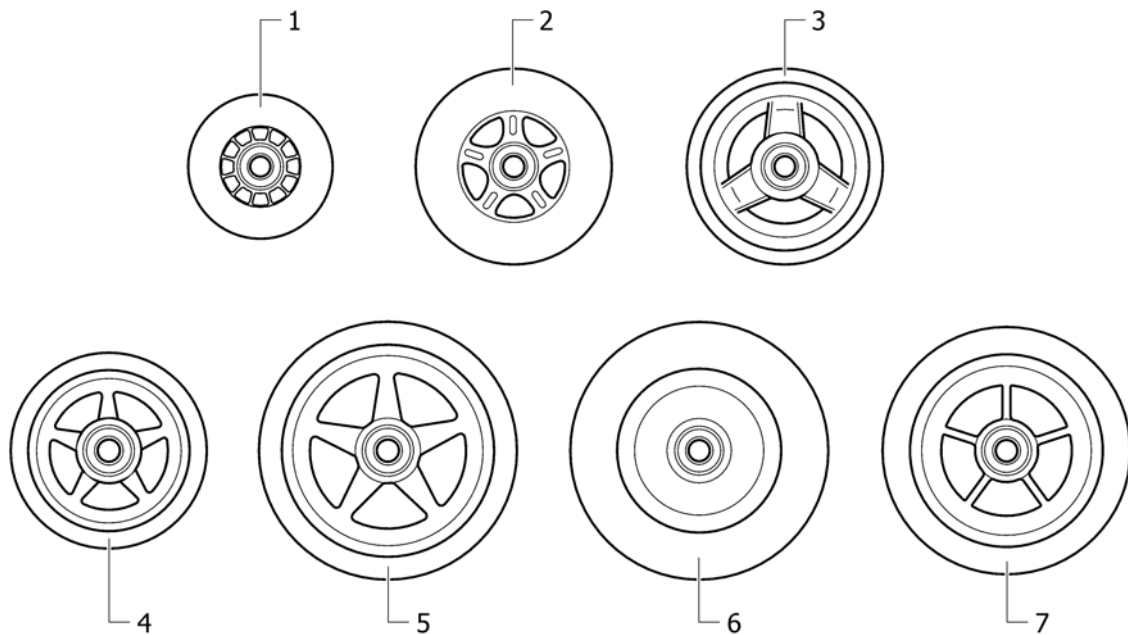

Pos.	Part number	Description	Valid from → until	Qty
1	1513180	3" Sports wheel, transparent <i>Replaces 732.1026</i>		2
2	1513181	4" Skater wheel, transparent <i>Replaces 1416805</i>		2
3	1513182	4" Everyday wheel, 3 spokes <i>Replaces 732.1033</i>		2
4	1513183-0408	4" Starec wheel, glossy black <i>Replaces 1420968-0408</i>		2
4	1513183-0400	4" Starec wheel, techno-anthrazit <i>Replaces 1420968-0400</i>		2
4	1513183-0401	4" Starec wheel, lemon <i>Replaces 1420968-0401</i>		2
4	1513183-0402	4" Starec wheel, apricot <i>Replaces 1420968-0402</i>		2
4	1513183-0403	4" Starec wheel, fashion violett <i>Replaces 1420968-0403</i>	→ 20070630	2
4	1513183-0404	4" Starec wheel, b-blue <i>Replaces 1420968-0404</i>		2
4	1513183-0407	4" Starec wheel, colourless <i>Replaces 1420968-0407</i>		2
4	1513183-0440	4" Starec wheel, comet red <i>Replaces 1420968-0440</i>	20070101 →	2
5	1513187-0408	5" Starec wheel, black <i>Replaces 732.1032.961</i>		2
5	1513187-0400	5" Starec wheel, techno-anthrazit <i>Replaces 732.1032.950</i>		2
5	1513187-0401	5" Starec wheel, lemon <i>Replaces 732.1032.951</i>		2
5	1513187-0402	5" Starec wheel, apricot <i>Replaces 732.1032.954</i>		2
5	1513187-0403	5" Starec wheel, fashion violett <i>Replaces 732.1032.955</i>	→ 20070630	2
5	1513187-0404	5" Starec wheel, b-blue <i>Replaces 732.1032.957</i>		2
5	1513187-0407	5" Starec wheel, colourless <i>Replaces 732.1032.960</i>		2


**Front wheels**

<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
5	1513187-0440	5" Starec wheel, comet red <i>Replaces 732.1032-0440</i>	20070101 →	2
6	1513186	5" Soft roll wheel <i>Replaces 732.1011</i>		2
7	1513185	5" Light wheel <i>Replaces 732.1010</i>		2

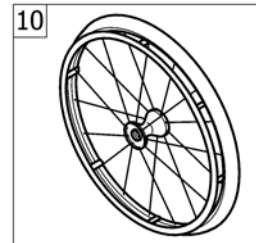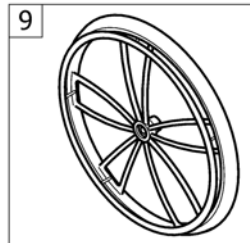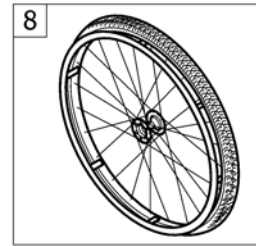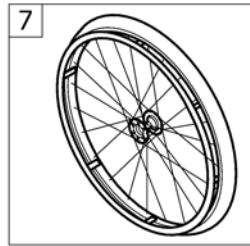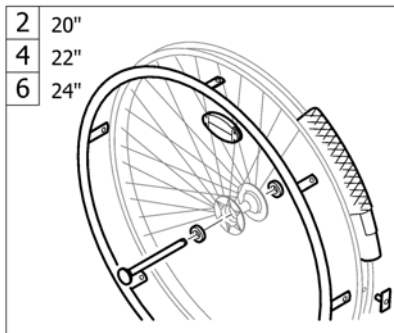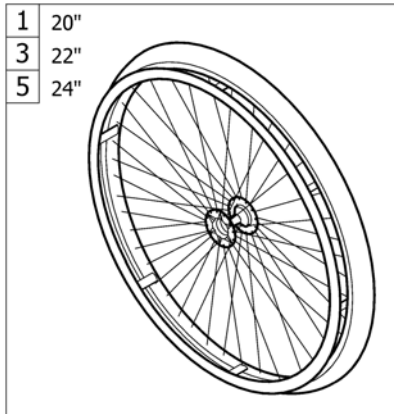
**REAR WHEELS**

Pos.	Part number	Description	Valid from → until	Qty
1	-	20" Küschall rear wheels complete		1
2	-	20" Küschall rear wheel accessories		1
3	-	22" Küschall rear wheels complete		1
4	-	22" Küschall rear wheel accessories		1
5	-	24" Küschall rear wheels complete		1
6	-	24" Küschall rear wheel accessories		1
7	-	High Performance wheel		1
8	-	Mountainbike wheel		1
9	-	Spider wheel		1
10	-	Spinergy wheel		1

## 20" Küschall rear wheels complete

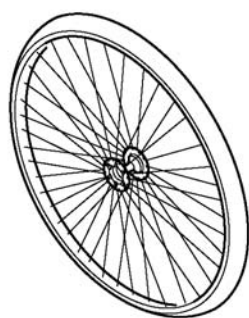

	no tyre			
	1	2	3	4
	5	6	7	8

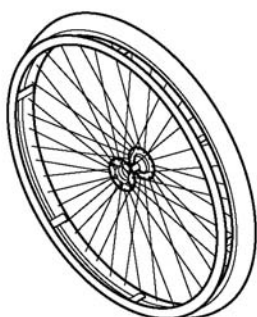


	no tyre			
		9	10	11
		12	13	14

Pos.	Part number	Description	Valid from → until	Qty
1	904.1080.950	Rear wheel 20" Starec R, tecno, without handrim, without tyre		2
1	904.1080.951	Rear wheel 20" Starec R, lemon, without handrim, without tyre		2
1	904.1080.954	Rear wheel 20" Starec R, apricot, without handrim, without tyre		2
1	904.1080.955	Rear wheel 20" Starec R, purple, without handrim, without tyre	→ 20070630	2
1	904.1080.957	Rear wheel 20" Starec R, b-blue, without handrim, without tyre		2
1	904.1080-0440	Rear wheel 20" Starec R, comet red, without handrim, without tyre	20070101 →	2
2	904.1194	Rear wheel 20", Starec hub R, without handrim, light wall		2
3	904.1156	Rear wheel 20", Starec hub R, without handrim, Tread		2
4	904.1157	Rear wheel 20", Starec hub R, without handrim, Indoor sports		2
5	904.1130	Rear wheel 20", Starec hub X, without handrim, without tyre		2
5	904.1130.950	Rear wheel 20" Starec X Tecno without handrim, without tyre		2
5	904.1130.951	Rear wheel 20" Starec X Lemon without handrim, without tyre		2
5	904.1130.954	Rear wheel 20" Starec X Apricot without handrim, without tyre		2
5	904.1130.955	Rear wheel 20" Starec X purple without handrim, without tyre	→ 20070630	2
5	904.1130.957	Rear wheel 20" Starec X B-blue without handrim, without tyre		2
5	904.1130-0440	Rear wheel 20" Starec X Comet red without handrim, without tyre	20070101 →	2
6	904.1188	Rear wheel 20", Starec hub X, without handrim, light wall		2
7	904.1136	Rear wheel 20", Starec hub X, without handrim, Tread		2
8	904.1137	Rear wheel 20", Starec hub X, without handrim, Indoor sports		2
9	904.1191	Rear wheel 20", Starec hub R, handrim, light wall		2
10	904.1147	Rear wheel 20", Starec hub R, handrim, Tread		2
11	904.1148	Rear wheel 20", Starec hub R, handrim, Indoor sports		2
12	904.1185	Rear wheel 20", Starec hub X, handrim, light wall		2
13	904.1121	Rear wheel 20", Starec hub X, handrim, Tread		2
14	904.1122	Rear wheel 20", Starec hub X, handrim, Indoor sports		2




## 20" Küschall rear wheel accessories

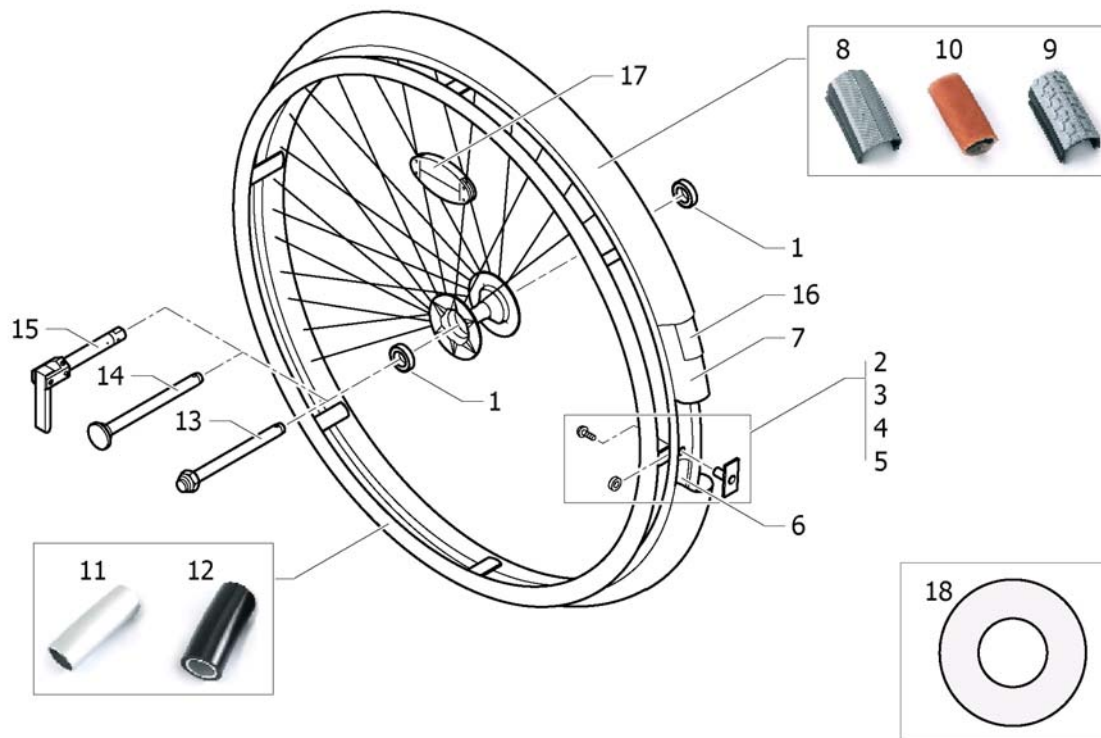


Pos.	Part number	Description	Valid from → until	Qty
1	270.1003	R8-2RS 1/2"/1.1/8"x5/16"		4
2	1495958	Fixation kit handrim, 3 cm		2
3	1495959	Fixation kit handrim, 4 cm		2
4	1495960	Fixation kit handrim, 3 cm with spokes guard		2
5	1495961	Fixation kit handrim, 4 cm with spokes guard		2
6	726.1001	Rim protection 20"		2
7	727.1005	Inner Tube 20" x 1"		2
8	728.1026	Tyre Lightwall 20"		2
9	728.1005	Treaded Tyre 20"		2
10	729.1008	Indoor sports tyre 20" / 10 bar		2
11	457.1024	Handrim 20" Alu		2
12	457.1044	Handrim 20" Super-Grip		2
13	1512031	Quick release axles long, pair		1
14	1509137-0407	Quick release axles long with cap, pair, colourless		1
14	1509137-0400	Quick release axles long with cap, pair, tecno anthracite		1
14	1509137-0401	Quick release axles long with cap, pair, lemon		1
14	1509137-0402	Quick release axles long with cap, pair, apricot		1
14	1509137-0404	Quick release axles long with cap, pair, b-blue		1
14	1509137-0440	Quick release axles long with cap, pair, comet red		1
15	1493755	Tetra clip with quick release axle, pair, long		1
16	1424917	Rear wheel inset anti-flat for 20,22,24,26" tyre		2
17	220.1033	Spokereflection		2
18	736.1006	Spokes guard Transparent 20"		2

## 22" Küschall rear wheels complete

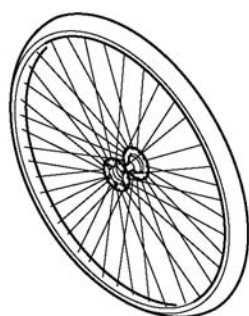


	no tyre			
	1	2	3	4
	5	6	7	8

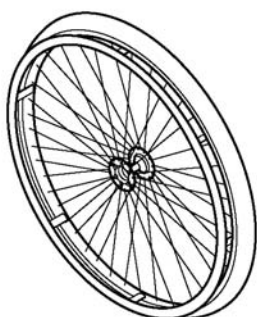

	no tyre			
		9	10	11
		12	13	14


Pos.	Part number	Description	Valid from → until	Qty
1	904.1081	Rear wheel 22", Starec hub R, without handrim, without tyre		2
1	904.1081.950	Rear wheel 22" Starec R, tecno, without handrim, without tyre		2
1	904.1081.951	Rear wheel 22" Starec R, lemon, without handrim, without tyre		2
1	904.1081.954	Rear wheel 22" Starec R, apricot, without handrim, without tyre		2
1	904.1081.955	Rear wheel 22" Starec R, purple, without handrim, without tyre	→ 20070630	2
1	904.1081.957	Rear wheel 22" Starec R, b-blue, without handrim, without tyre		2
1	904.1081-0440	Rear wheel 22" Starec R, comet red, without handrim, without tyre		2
2	904.1195	Rear wheel 22 ", Starec hub R, without handrim, light wall		2
3	904.1158	Rear wheel 22", Starec hub R, without handrim, Tread		2
4	904.1159	Rear wheel 22", Starec hub R, without handrim, Indoor sports		2
5	904.1131	Rear wheel 22", Starec hub X, without handrim, without tyre		2
5	904.1131.950	Rear wheel 22" Starec X Tecno without handrim, without tyre		2
5	904.1131.951	Rear wheel 22" Starec X Lemon without handrim, without tyre		2
5	904.1131.954	Rear wheel 22" Starec X Apricot without handrim, without tyre		2
5	904.1131.955	Rear wheel 22" Starec X purple without handrim, without tyre	→ 20070630	2
5	904.1131.957	Rear wheel 22" Starec X B-blue without handrim, without tyre		2
5	904.1131-0440	Rear wheel 22" Starec X Comet red without handrim, without tyre		2
6	904.1189	Rear wheel 22", Starec hub X, without handrim, light wall		2
7	904.1139	Rear wheel 22", Starec hub X, without handrim, Tread		2
8	904.1140	Rear wheel 22", Starec hub X, without handrim, Indoor sports		2
9	904.1192	Rear wheel 22", Starec hub R, handrim, light wall		2
10	904.1149	Rear wheel 22", Starec hub R, handrim, Tread		2
11	904.1150	Rear wheel 22", Starec hub R, handrim, Indoor sports		2
12	904.1186	Rear wheel 22", Starec hub X, handrim, light wall		2
13	904.1123	Rear wheel 22", Starec hub X, handrim, Tread		2
14	904.1124	Rear wheel 22", Starec hub X, handrim, Indoor sports		2

## 22" Küschall rear wheel accessories

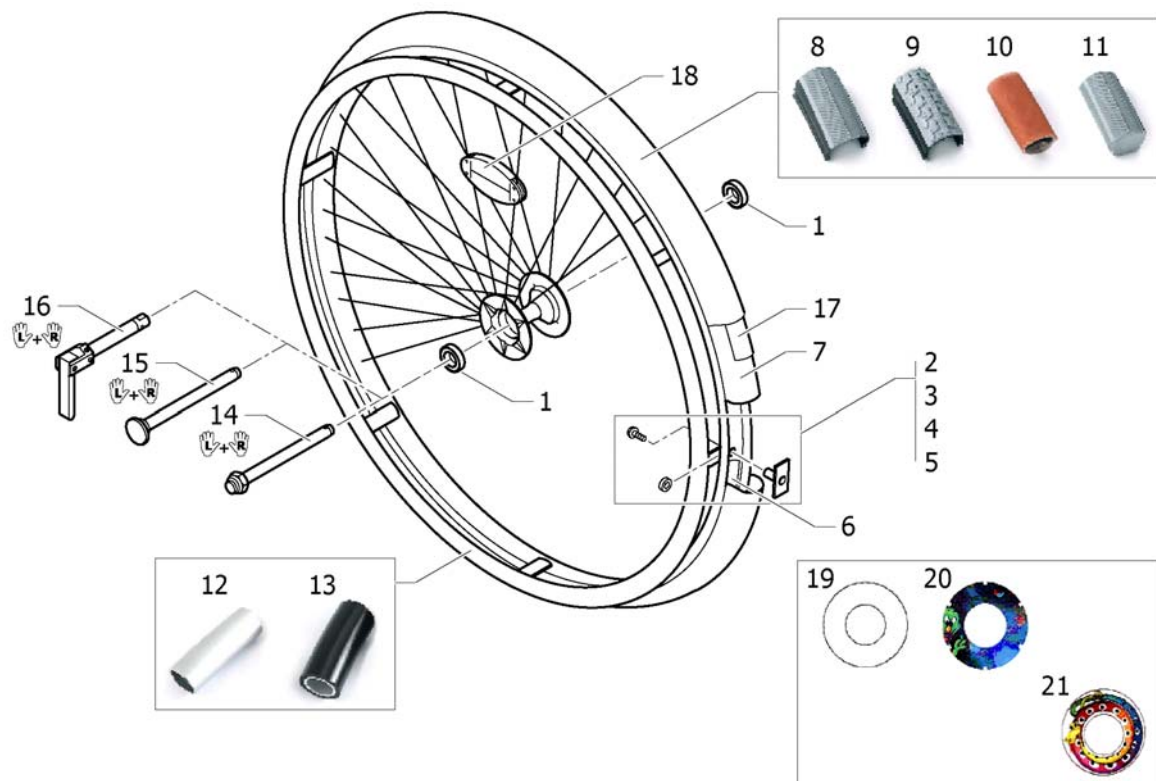


Pos.	Part number	Description	Valid from → until	Qty
1	270.1003	R8-2RS 1/2"/1.1/8"x5/16"		4
2	1495958	Fixation kit handrim, 3 cm		2
3	1495959	Fixation kit handrim, 4 cm		2
4	1495960	Fixation kit handrim, 3 cm with spokes guard		2
5	1495961	Fixation kit handrim, 4 cm with spokes guard		2
6	726.1005	Rim protection 22"		2
7	727.1007	Inner Tube 22" x 1"		2
8	728.1027	Tyre Lightwall 22"		2
9	1493768	Treaded tyre 22"		2
10	729.1006	Indoor sports tyre 22" / 10 bar		2
11	728.1031	Solid tyre 22", light, grey		2
12	457.1023	Handrim 22" Alu		2
13	457.1045.429	Handrim 22" Super-Grip		2
14	1512031	Quick release axles long, pair		1
15	1509137-0407	Quick release axles long with cap, pair, colourless	20080101 →	1
15	1509137-0400	Quick release axles long with cap, pair, tecno anthracite	20080101 →	1
15	1509137-0401	Quick release axles long with cap, pair, lemon	20080101 →	1
15	1509137-0402	Quick release axles long with cap, pair, apricot	20080101 →	1
15	1509137-0404	Quick release axles long with cap, pair, b-blue	20080101 →	1
15	1509137-0440	Quick release axles long with cap, pair, comet red	20080101 →	1
16	1493755	Tetra clip with quick release axle, pair, long		1
17	1424917	Rear wheel inset anti-flat for 20,22,24,26" tyre		2
18	220.1033	Spokereflection		2
19	736.1005	Spokes guard Transparent 22"		2
20	736.1010	Spokes guard Cosmos Brownie 22"	→ 20081231	2
21	736.1019.000	Spokes guard Boa Boa 22"		2

## 24" Küschall rear wheels complete

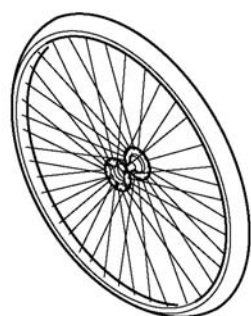


	∅ no tyre			
	1	2	3	4
	5	6	7	8

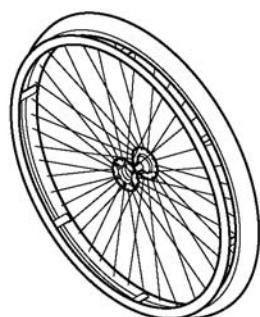

	∅ no tyre			
		9	10	
		11	12	13

Pos.	Part number	Description	Valid from → until	Qty
1	904.1082	Rear wheel 24", Starec hub R, without handrim, without tyre		2
1	904.1082.950	Rear wheel 24" Starec R, tecno, without handrim, without tyre		2
1	904.1082.951	Rear wheel 24" Starec R, lemon, without handrim, without tyre		2
1	904.1082.954	Rear wheel 24" Starec R, apricot, without handrim, without tyre		2
1	904.1082.955	Rear wheel 24" Starec R, purple, without handrim, without tyre	→ 20070630	2
1	904.1082.957	Rear wheel 24" Starec R, b-blue, without handrim, without tyre		2
1	904.1082-0440	Rear wheel 24" Starec R, comet red, without handrim, without tyre	20070101 →	2
2	904.1161	Rear wheel 24", Starec hub R, without handrim, light wall		2
3	904.1160	Rear wheel 24", Starec hub R, without handrim, Tread		2
4	904.1163	Rear wheel 24", Starec hub R, without handrim, Indoor sports		2
5	1416885	Rear wheel 24", Starec hub X, without handrim, without tyre		2
5	904.1132.950	Rear wheel 24" Starec X Tecno without handrim, without tyre		2
5	904.1132.951	Rear wheel 24" Starec X Lemon without handrim, without tyre		2
5	904.1132.954	Rear wheel 24" Starec X Apricot without handrim, without tyre		2
5	904.1132.955	Rear wheel 24" Starec X purple without handrim, without tyre	→ 20070630	2
5	904.1132.957	Rear wheel 24" Starec X B-blue without handrim, without tyre		2
5	904.1132-0440	Rear wheel 24" Starec X Comet red without handrim, without tyre	20070101 →	2
6	1416886	Rear wheel 24", Starec hub X, without handrim, light wall		2
7	904.1141	Rear wheel 24", Starec hub X, without handrim, Tread		2
8	904.1144	Rear wheel 24", Starec hub X, without handrim, Indoor sports		2
9	904.1152	Rear wheel 24", Starec hub R, handrim, light wall		2
10	904.1151	Rear wheel 24", Starec hub R, handrim, Tread		2
11	1416884	Rear wheel 24", Starec hub X, handrim, light wall		2
12	904.1125	Rear wheel 24", Starec hub X, handrim, Tread		2
13	904.1135	Rear wheel 24", Starec hub X, handrim, Indoor sports		2

## 24" Küschall rear wheel accessories

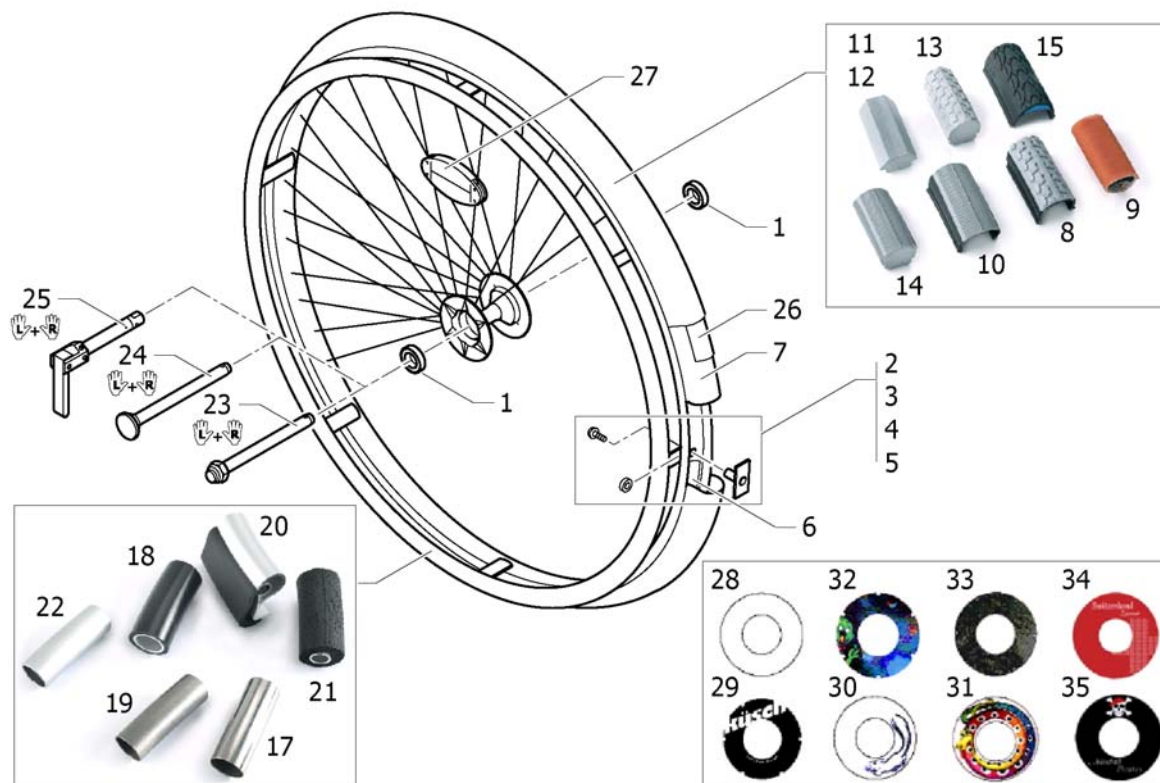


Pos.	Part number	Description	Valid from → until	Qty
1	270.1003	R8-2RS 1/2"/1.1/8"x5/16"		4
2	1495958	Fixation kit handrim, 3 cm		2
3	1495959	Fixation kit handrim, 4 cm		2
4	1495960	Fixation kit handrim, 3 cm with spokes guard		2
5	1495961	Fixation kit handrim, 4 cm with spokes guard		2
6	726.1008.000	Rim protection 24" Large		2
7	727.1002	Inner Tube 24" x 1"		2
8	728.1002	Treaded Tyre 24"		2
9	729.1001	Indoor sports tyre 24" / 10 bar		2
10	1424749	Tyre Light wall 24"		2
11	728.1015	KIK-Type 24", black, fail-safe		2
12	728.1023	KIK-Type 24", grey, fail-safe		2
13	728.1029	Solid tyre 24", profile		2
14	728.1032	Solid tyre 24", light, grey		2
15	1438521	Tyre 24" Marathon Evolution		2
17	457.1009	Handrim 24" steel polished		2
18	457.1017.429	Handrim 24" Super-Grip		2
19	457.1051	Handrim 24" Titanium		2
20	457.1052.000	Handrim 24" Tetra 2000		2
21	905.5002	Handrim 24" Maxgrip		2
22	1432287-0005	Handrim 24" colourless anod. light V2003 <i>Replaces 1432286-0005</i>		2
23	1512031	Quick release axles long, pair		1
24	1509137-0407	Quick release axles long with cap, pair, colourless	20080101 →	1
24	1509137-0400	Quick release axles long with cap, pair, tecno anthracite	20080101 →	1
24	1509137-0401	Quick release axles long with cap, pair, lemon	20080101 →	1
24	1509137-0402	Quick release axles long with cap, pair, apricot	20080101 →	1
24	1509137-0404	Quick release axles long with cap, pair, b-blue	20080101 →	1
24	1509137-0440	Quick release axles long with cap, pair, comet red	20080101 →	1
25	1493755	Tetra clip with quick release axle, pair, long		1
26	1424917	Rear wheel inset anti-flat for 20,22,24,26" tyre		2
27	220.1033	Spokereflection		2

**24" Küschall rear wheel accessories**

Pos.	Part number	Description	Valid from → until	Qty
28	736.1004	Spokes guard Transparent 24"		2
29	736.1009	Spokes guard Cosmos Brownie 24"	→ 20081231	2
30	736.1014	Spokes guard Carbon 24"		2
31	1497108	Spokes guard Switzerland 24"		2
32	736.1002	Spokes guard Küschall, black 24"		2
33	736.1017	Spokes guard Nimbo 24"	→ 20061231	2
34	736.1015	Spokes guard Boa Boa 24"		2
35	1497109	Spokes guard Pirates 24"		2

## High Performance wheel

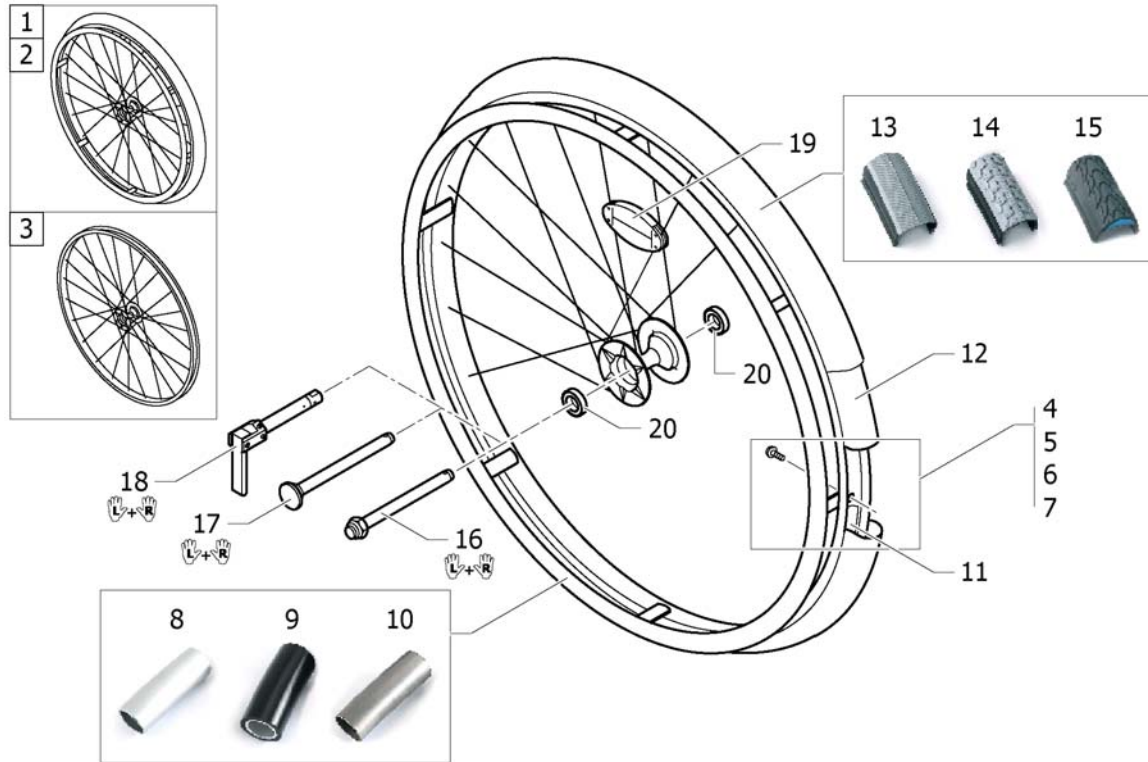

Pos.	Part number	Description	Valid from → until	Qty
1	1420958	Wheel 24", High Performance, colourless, handrim alu, tyre light wall		2
1	1420958-0400	Wheel 24", High Performance, techno-anthracite, handrim alu, tyre light		2
1	1420958-0401	Wheel 24", High Performance, lemon, handrim alu, tyre light wall		2
1	1420958-0402	Wheel 24", High Performance, apricot, handrim alu, tyre light wall		2
1	1420958-0404	Wheel 24", High Performance, b-blue, handrim alu, tyre light wall		2
1	1420958-0440	Wheel 24", High Performance, comet red, handrim alu, tyre light wall		2
2	1527046	Wheel 24", High Performance, colourless, handrim alu, tyre Marathon Evolution		2
2	1527046-0400	Wheel 24", High Performance, techno-anthracite, handrim alu, tyre Marathon Evolution		2
2	1527046-0401	Wheel 24", High Performance, lemon, handrim alu, tyre Marathon Evolution		2
2	1527046-0402	Wheel 24", High Performance, apricot, handrim alu, tyre Marathon Evolution		2
2	1527046-0404	Wheel 24", High Performance, b-blue, handrim alu, tyre Marathon Evolution		2
2	1527046-0440	Wheel 24", High Performance, comet red, handrim alu, tyre Marathon Evolution		2
3	1517723-0407	Wheel 24", High Performance, colourless, without handrim, without tyre		2
3	1517723-0400	Wheel 24", High Performance, techno-anthracite, without handrim, without tyre		2
3	1517723-0401	Wheel 24", High Performance, lemon, without handrim, without tyre		2
3	1517723-0402	Wheel 24", High Performance, apricot, without handrim, without tyre		2
3	1517723-0404	Wheel 24", High Performance, b-blue, without handrim, without tyre		2
3	1517723-0440	Wheel 24", High Performance, comet red, without handrim, without tyre		2
4	1496019	Fixation kit handrim for High Performance wheel, 3 cm		2
5	1496020	Fixation kit handrim for High Performance wheel, 4 cm		2
6	1497110	Fixation kit handrim for High Performance wheel, 3 cm with spokes guard		2
7	1497111	Fixation kit handrim for High Performance wheel, 4 cm with spokes guard		2
8	1432287-0005	Handrim 24" colourless anod. light V2003		2
9	457.1017.429	Handrim 24" Super-Grip		2
10	457.1051	Handrim 24" Titanium		2
11	1495962	Rim Protection for High Performance / Spinergy wheel		2

## High Performance wheel

Pos.	Part number	Description	Valid from → until	Qty
12	727.1002	Inner Tube 24" x 1"		2
13	1424749	Tyre Light wall 24"		2
14	728.1002	Treaded Tyre 24"		2
15	1438521	Tyre 24" Marathon Evolution		2
16	1512031	Quick release axles long, pair		1
17	1509137-0407	Quick release axles long with cap, pair, colourless		1
17	1509137-0400	Quick release axles long with cap, pair, tecno anthracite		1
17	1509137-0401	Quick release axles long with cap, pair, lemon		1
17	1509137-0402	Quick release axles long with cap, pair, apricot		1
17	1509137-0404	Quick release axles long with cap, pair, b-blue		1
17	1509137-0440	Quick release axles long with cap, pair, comet red		1
18	1493755	Tetra clip with quick release axle, pair, long		1
19	220.1033	Spokereflection		2
20	270.1003	R8-2RS 1/2"/1.1/8"x5/16"		4




# Mountainbike wheel




Pos.	Part number	Description	Valid from → until	Qty
1	1517875	Rear wheel 24", Mountain Bike, handrim alu, tyre MTB <i>Replaces 1420959</i>		2
2	1496015	Fixation kit handrim for Mountain bike wheel, 3 cm		2
3	1496016	Fixation kit handrim for Mountain bike wheel, 4 cm		2
4	1496017	Fixation kit handrim for Mountain bike wheel, 3 cm with spokes guard		2
5	1496018	Fixation kit handrim for Mountain bike wheel, 4 cm with spokes guard		2
6	457.1023	Handrim 22" Alu		2
7	457.1045.429	Handrim 22" Super-Grip		2
8	1420881	Rim protection for Mountain bike		2
9	1420884	Inner tube for Mountain bike		2
10	1420887	Tyre Mountain bike, 24"		2
11	1512031	Quick release axles long, pair		1
12	1509137-0407	Quick release axles long with cap, pair, colourless		1
13	1493755	Tetra clip with quick release axle, pair, long		1
14	220.1033	Spokereflection		2
15	270.1003	R8-2RS 1/2"/1.1/8"x5/16"		4
16	1529327	Rear wheel Mountain Bike 24", without handrim, without tyre <i>Replaces 1439420</i>		2

# Spider wheel


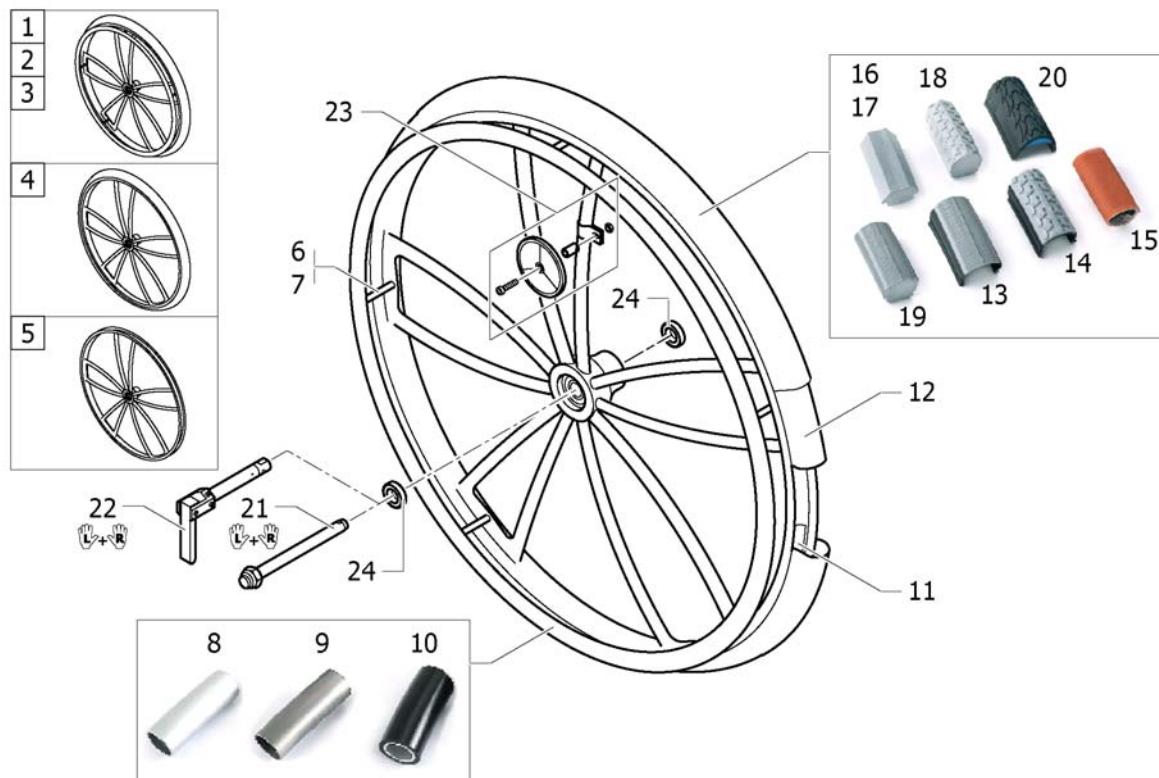


Pos.	Part number	Description	Valid from → until	Qty
1	904.1201.000	Rear wheel 24", Spiderwheel, handrim alu, tyre light wall		2
2	1416977	Rear wheel 24", Spider wheel, handrim Titan, tyre light wall		2
3	1416978	Rear wheel 24", Spider wheel, handrim Super-Grip, tyre light wall		2
4	1420961	Rear wheel 24", Spiderwheel, without handrim, tyre light wall		2
5	1424944	Rear wheel 24", Spiderwheel without handrim, without tyre		2
6	1418931	Fixation kit handrim for Spider wheel, 3 cm		2
7	1418932	Fixation kit handrim for Spider wheel, 4 cm		2
8	457.1053.000	Handrim 24" Alu for Spider wheel		2
9	1416801	Handrim 24" Titan for Spider Wheel		2
10	1416802	Handrim 24" Super-Grip for Spider wheel		2
11	726.1008.000	Rim protection 24" Large		2
12	727.1002	Inner Tube 24" x 1"		2
13	1424749	Tyre Light wall 24"		2
14	728.1002	Treaded Tyre 24"		2
15	729.1001	Indoor sports tyre 24" / 10 bar		2
16	728.1015	KIK-Type 24", black, fail-safe		2
17	728.1023	KIK-Type 24", grey, fail-safe		2
18	728.1029	Solid tyre 24", profile		2
19	728.1032	Solid tyre 24", light, grey		2
20	1438521	Tyre 24" Marathon Evolution		2
21	1512031	Quick release axles long, pair		1
22	1493755	Tetra clip with quick release axle, pair, long		1
23	1439352	Spoke reflection set for Spider wheel		2
24	1416804	Bearing for Spider wheel		4

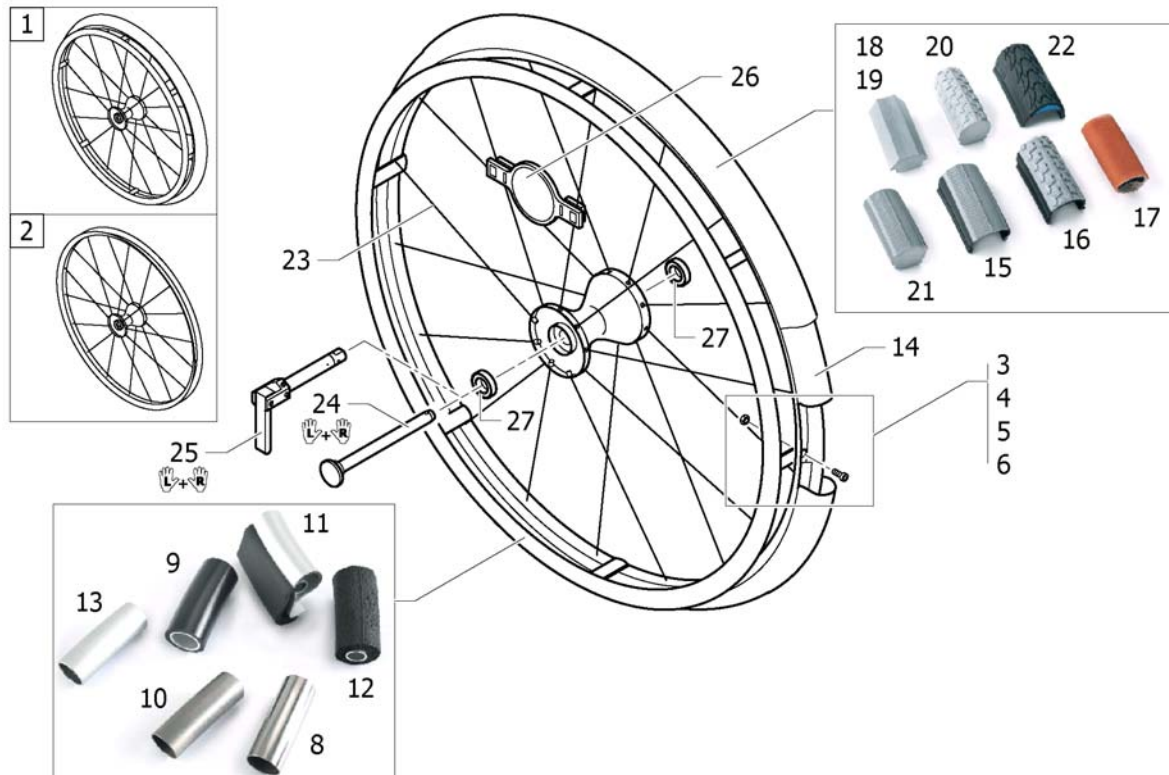


Pos.	Part number	Description	Valid from → until	Qty
1	1485805	Rear wheel 24", Spinergy, handrim Alu, tyre light wall		2
2	1438747	Rear wheel 24", Spinergy, without handrim, without tyre		2
3	1495967	Fixation kit handrim for Spinergy wheel, 3 cm		2
4	1495968	Fixation kit handrim for Spinergy wheel, 4 cm		2
5	1495969	Fixation kit handrim for Spinergy wheel, 3 cm with spokes guard		2
6	1495970	Fixation kit handrim for Spinergy wheel, 4 cm with spokes guard		2
8	457.1009	Handrim 24" steel polished		2
9	457.1017.429	Handrim 24" Super-Grip		2
10	457.1051	Handrim 24" Titanium		2
11	457.1052.000	Handrim 24" Tetra 2000		2
12	905.5002	Handrim 24" Maxgrip		2
13	1432287-0005	Handrim 24" colourless anod. light V2003 <i>Replaces 1432286-0005</i>		2
14	727.1002	Inner Tube 24" x 1"		2
15	1424749	Tyre Light wall 24"		2
16	728.1002	Treaded Tyre 24"		2
17	729.1001	Indoor sports tyre 24" / 10 bar		2
18	728.1015	KIK-Type 24", black, fail-safe		2
19	728.1023	KIK-Type 24", grey, fail-safe		2
20	728.1029	Solid tyre 24", profile		2
21	728.1032	Solid tyre 24", light, grey		2
22	1438521	Tyre 24" Marathon Evolution		2
23	1493648	Spoke for Spinergy wheel 24"		36
24	1507627	Quickrelease axle long with cap for Spinergy wheel, pair		1
25	1512029	Tetra clip with quick release axle, long, for Spinergy wheel, pair		1
26	1451931	Spoke reflection for Spinergy wheel		2

# BRAKES

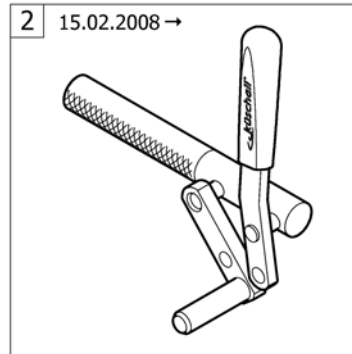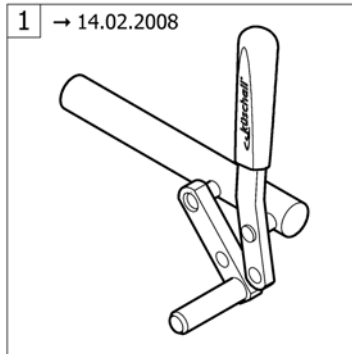


Pos.	Part number	Description	Valid from → until	Qty
1	-	Brakes =>14.02.2008	→ 20080214	1
2	-	Brakes 15.02.2008=>		1

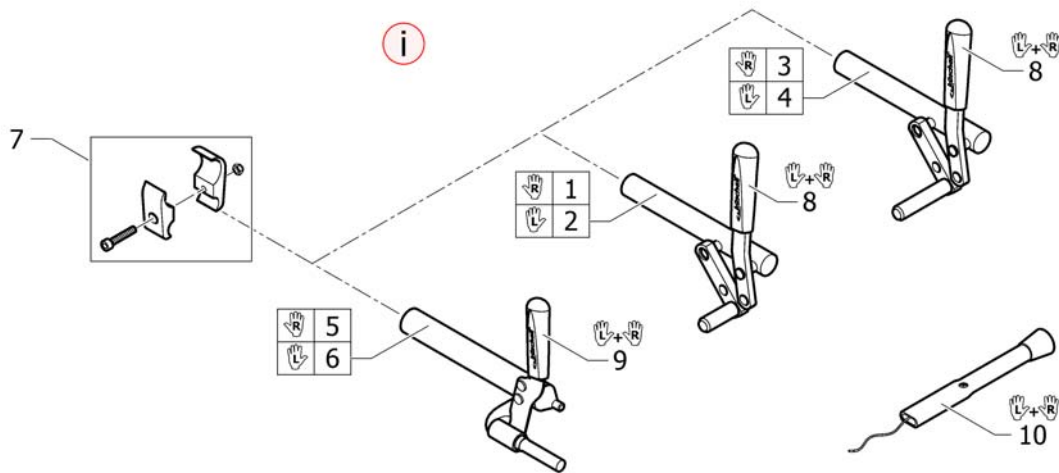


Pos.	Part number	Description	Valid from → until	Qty
i	-	Old design: Solid, plain brake holder without knurling		0
1	1438663	Brake, bent lever, right		1
2	1438664	Brake, bent lever, left		1
3	1488179	Brake, lever bent, long shoe, right		1
4	1488180	Brake, lever bent, long shoe, left		1
5	1438665	Performance brake right		1
6	1438666	Performance brake left		1
7	1513176	Clamping piece to brake, set <i>Replaces 901.1075</i>		2
8	1513177	Brake handle standard, pair <i>Replaces 1432056</i>		1
9	1513178	Brake handle for performance brake, pair <i>Replaces 1432055</i>		1
10	1513179	Brake lever extension, pair <i>Replaces 901.1095</i>		1

**Brakes 15.02.2008=>**

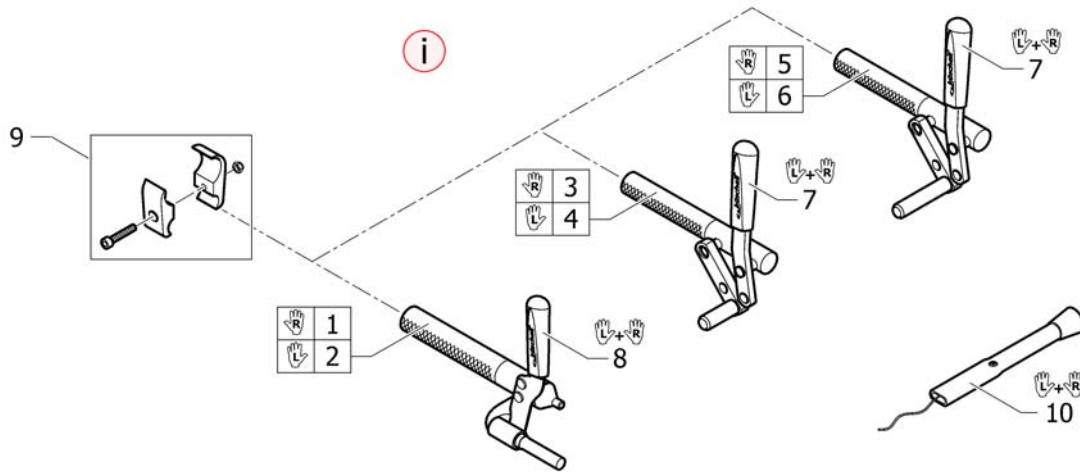


Pos.	Part number	Description	Valid from → until	Qty
i	-	New design: Tubular, knurled brake holder		0
1	1512132	Performance brake right		1
2	1512131	Performance brake left		1
3	1512126	Brake, bent lever, right		1
4	1512125	Brake, bent lever, left		1
5	1517655	Brake, lever bent, long shoe, right		1
6	1517656	Brake, lever bent, long shoe, left		1
7	1513177	Brake handle standard, pair		1
8	1513178	Brake handle for performance brake, pair		1
9	1513176	Clamping piece to brake, set		2
10	1513179	Brake lever extension, pair		1

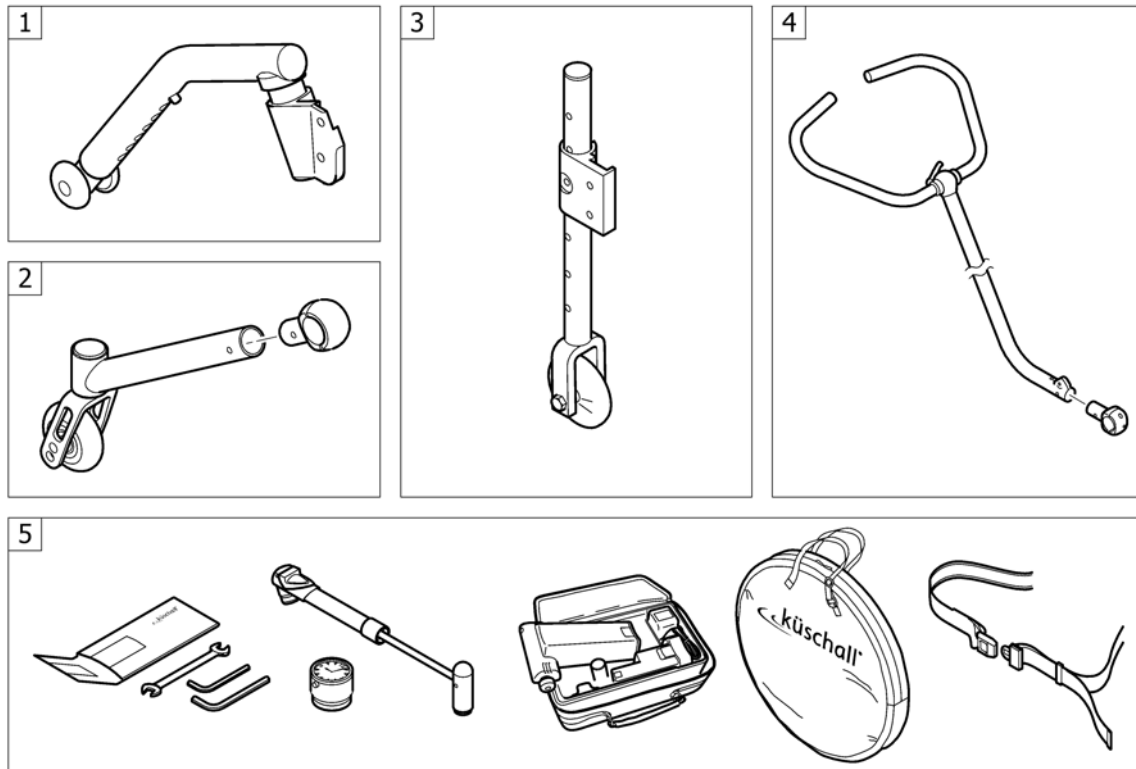**OPTIONS & ACCESSORIES**


Pos.	Part number	Description	Valid from → until	Qty
1	-	Antitipper		1
2	-	Active antitipper		1
3	-	Transit wheels		1
4	-	Buggy push handle		1
5	-	Chair accessories		1

# Antitipper

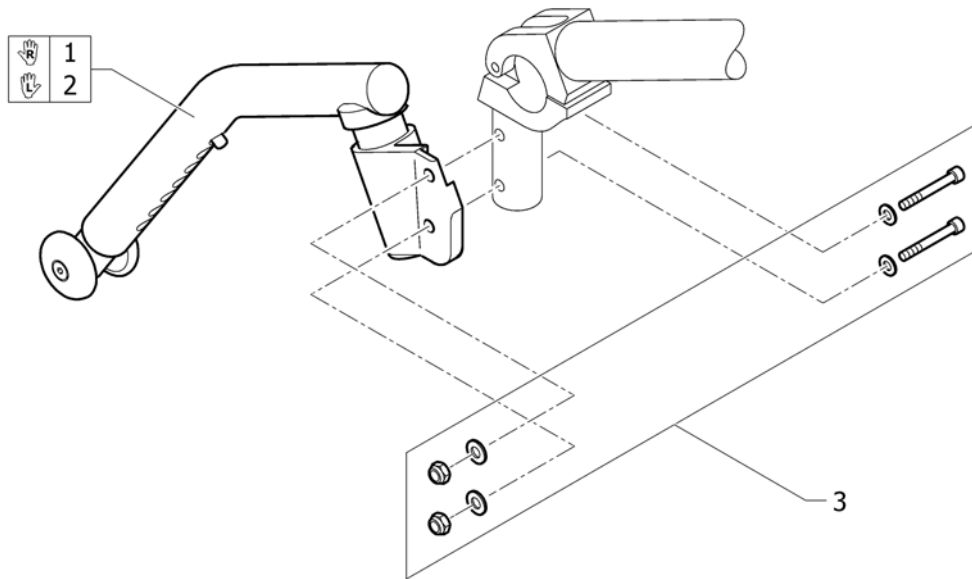


Pos.	Part number	Description	Valid from → until	Qty
1	1488928	Antitipper short right complete		1
2	1488927	Antitipper short left complete		1
3	1492194	Fixation kit for adaptations		1



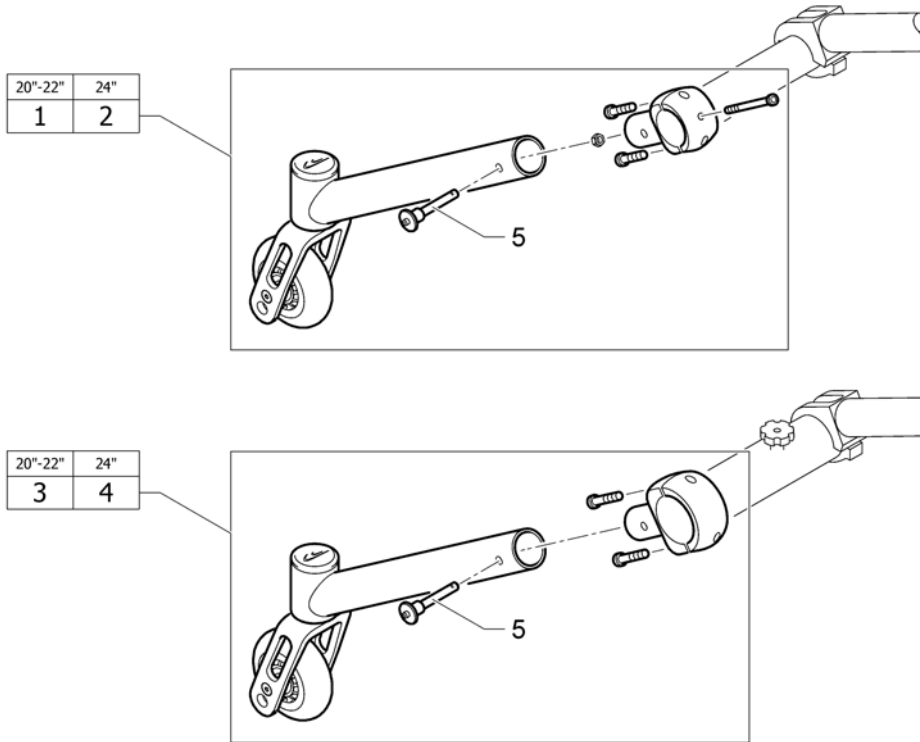


Pos.	Part number	Description	Valid from → until	Qty
1	1517871	Active Antitipper, for standard axle Ø30 for 20" and 22" wheels		1
2	1517873	Active Antitipper, for standard axle Ø30 for 24" and 25" wheels <i>Replaces 906.1029</i>		1
3	1517872	Active Antitipper, for Vario-Ax Ø40 for 20" and 22" wheels		1
4	1517874	Active Antitipper, for Vario-Ax Ø40 for 24" and 25" wheels <i>Replaces 906.1030</i>		1
5	1517877	Quickpin fixture <i>Replaces 907.1125</i>		1

# Transit wheels

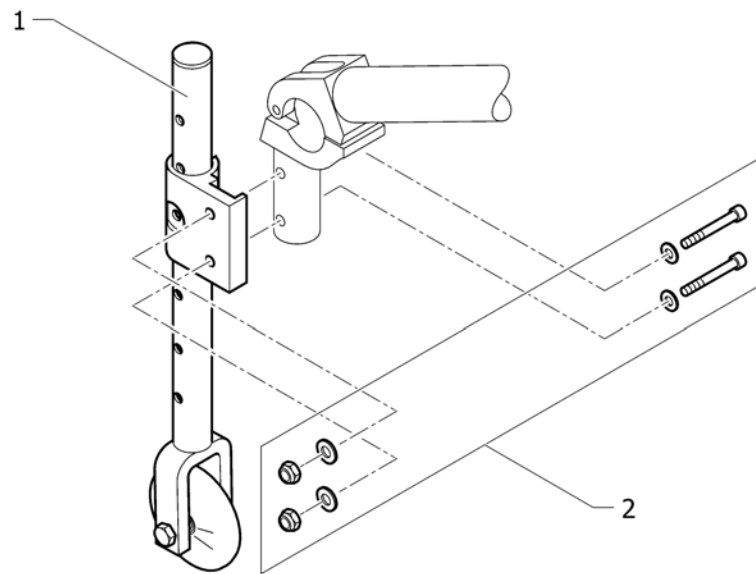


Pos.	Part number	Description	Valid from → until	Qty
1	1451791	Transit wheels, pair		1
2	1492194	Fixation kit for adaptations		1

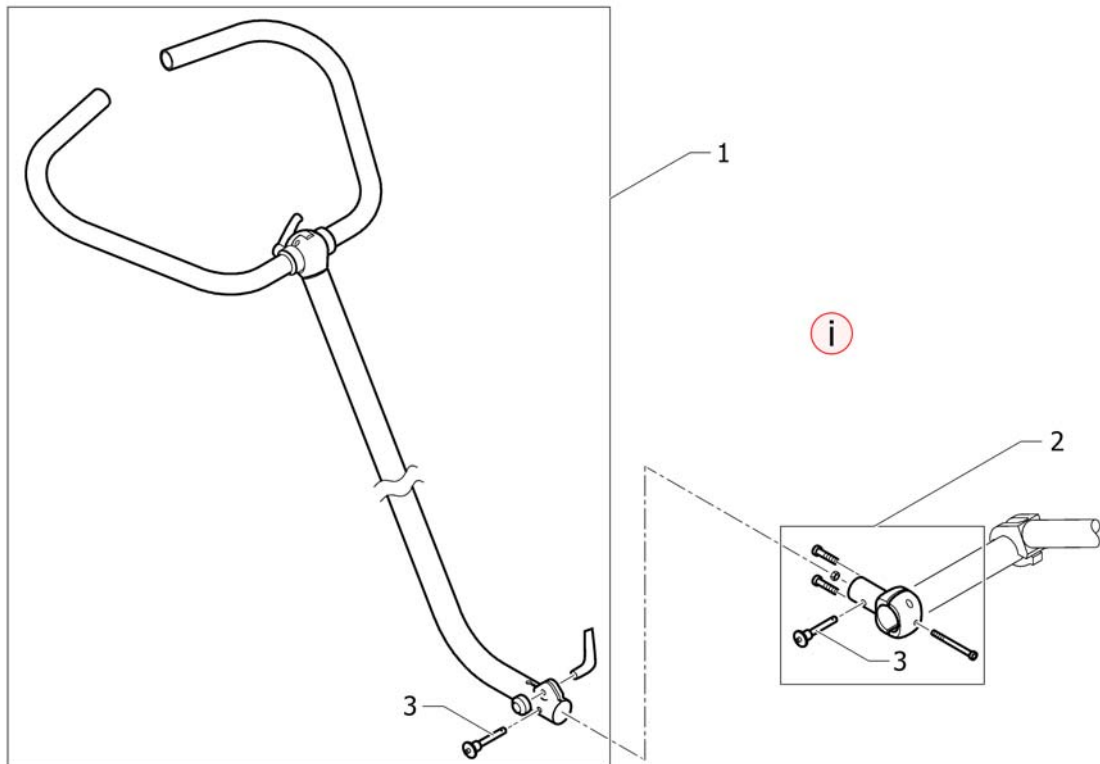**Buggy push handle**

Pos.	Part number	Description	Valid from → until	Qty
i	-	Not possible with rear wheel extension or with Vario-AX		0
1	1486022	Buggy push handle complete		1
2	1486023	Fixation kit D30 for Buggy push handle		1
3	1517877	Quickpin fixture		1

## Chair accessories

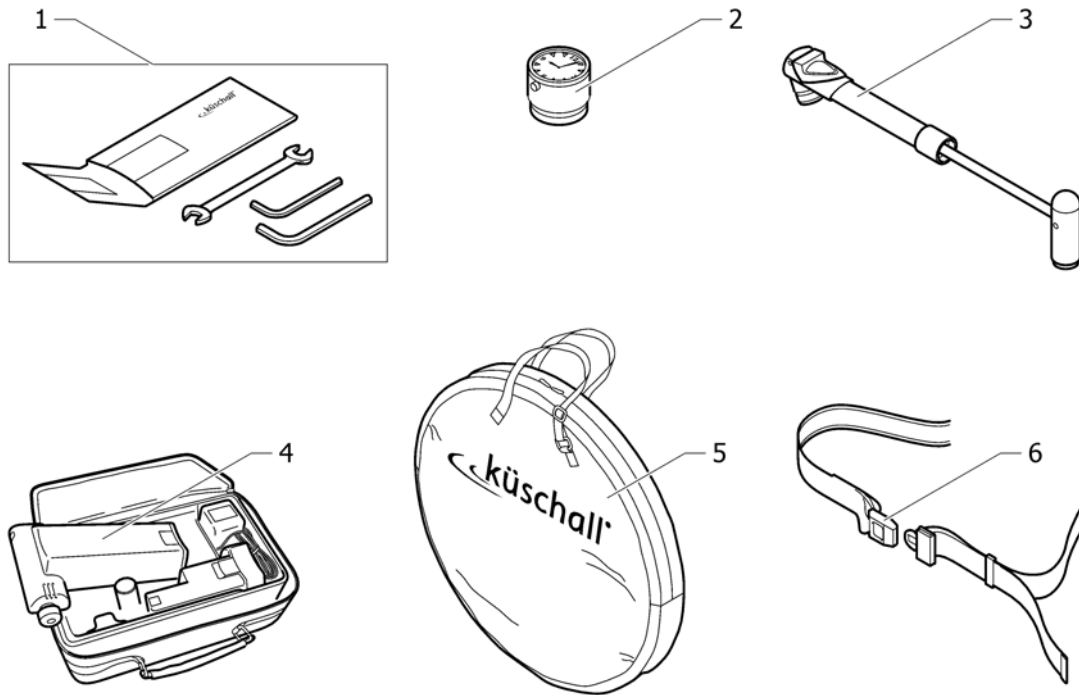

Pos.	Part number	Description	Valid from → until	Qty
1	1518285	Tool Kit <i>Replaces 800.1013</i>		1
2	1517925	Küschall watch		1
3	1517924	Air pump <i>Replaces 801.1004</i>		1
4	1518286	Air man <i>Replaces 801.1006.000</i>		1
5	1518287	Wheel bag 24" - 26" <i>Replaces 1420922</i>		1
6	1518288	Seat belt		1





**küschall<sup>®</sup> distributors**

**Belgium & Luxemburg:** Invacare nv • Autobaan 22 • B-8210 Loppem  
Tel: (32) (0)50 83 10 10 • Fax: (32) (0)50 83 10 11 • [belgium@invacare.com](mailto:belgium@invacare.com)

**Danmark:** Invacare A/S • Sdr. Ringvej 37 • DK-2605 Brøndby  
Tel: (45) (0)36 90 00 00 • Fax: (45) (0)36 90 00 01 • [denmark@invacare.com](mailto:denmark@invacare.com)

**Deutschland:** Invacare Aquatec GmbH • Alemannenstraße 10 • D-88316 Isny  
Tel: (49) (0)75 62 7 00 0 • Fax: (49) (0)75 62 7 00 66 • [info@invacare-aquatec.com](mailto:info@invacare-aquatec.com)

**European Distributor Organisation:** Invacare • Kleiststraße 49 • D-32457 Porta Westfalica  
Tel: (49) (0)57 31 754 540 • Fax: (49) (0)57 31 754 541 • [edo@invacare.com](mailto:edo@invacare.com)

**España:** Invacare SA • c/Areny s/n • Poligon Industrial de Celrà • E-17460 Celrà (Girona)  
Tel: (34) (0)972 49 32 00 • Fax: (34) (0)972 49 32 20 • [contactsp@invacare.com](mailto:contactsp@invacare.com)

**France:** Invacare Poirier SAS • Route de St Roch • F-37230 Fondettes  
Tel: (33) (0)2 47 62 64 66 • Fax: (33) (0)2 47 42 12 24 • [contactfr@invacare.com](mailto:contactfr@invacare.com)

**Ireland:** Invacare Ireland Ltd • Unit 5 Seatown Business Campus • Seatown Road • Swords • County Dublin – Ireland  
Tel: (353) 1 810 7084 • fax: (353) 1 810 7085 • [ireland@invacare.com](mailto:ireland@invacare.com)

**Italia:** Invacare Mecc San s.r.l. • Via dei Pini 62 • I-36016 Thiene (VI)  
Tel: (39) 0445 38 00 59 • Fax: (39) 0445 38 00 34 • [italia@invacare.com](mailto:italia@invacare.com)

**Nederland:** Invacare BV • Celsiusstraat 46 • NL-6716 BZ Ede  
Tel: (31) (0)318 695 757 • Fax: (31) (0)318 695 758 • [nederland@invacare.com](mailto:nederland@invacare.com)

**Norge:** Invacare AS • Grensesvingen 9 • Postboks 6230 • Etterstad • N-0603 Oslo  
Tel: (47) (0)22 57 95 00 • Fax: (47) (0)22 57 95 01 • [norway@invacare.com](mailto:norway@invacare.com) • [island@invacare.com](mailto:island@invacare.com)

**Österreich:** Mobitec Mobilitätshilfen GmbH • Herzog Odilostrasse 101 • A-5310 Mondsee  
Tel.: (43) 6232 5535 0 • Fax.: (43) 6232 5535 4 • [office@mobitec-austria.com](mailto:office@mobitec-austria.com) • [austria@invacare.com](mailto:austria@invacare.com)

**Portugal:** Invacare Lda • Rua Estrada Velha • 949 • P-4465-784 Leça do Balio  
Tel: (351) (0)225 1059 46/47 • Fax: (351) (0)225 1057 39 • [portugal@invacare.com](mailto:portugal@invacare.com)

**Sverige & Suomi:** Invacare AB • Fagerstagatan 9 • S-163 91 Spånga  
Tel: (46) (0)8 761 70 90 • Fax: (46) (0)8 761 81 08 • [sweden@invacare.com](mailto:sweden@invacare.com) • [finland@invacare.com](mailto:finland@invacare.com)

**Switzerland:** Küschall AG • Benkenstrasse 260 • CH-4108 Witterswil  
Tel.: (41) (0)61 487 70 80 • Fax.: (41) (0)61 487 70 81 • [kueschall@invacare.com](mailto:kueschall@invacare.com)

**United Kingdom:** Invacare Limited • Pencoed Technology Park • Pencoed, Bridgend CF35 5HZ  
Tel: (44) (0)1656 776200 • Fax: (44) (0)1656 776201 • [UK@invacare.com](mailto:UK@invacare.com)