
# Invacare® Revato

H335 / H345 / R7762 / R7763 / R7765 / R7766 / R7772 / R7780

en	<b>Foldable Grab Bar</b> User Manual .....	<b>3</b>
de	<b>Klappbarer Haltegriff</b> Gebrauchsanweisung .....	<b>6</b>
fr	<b>Barre de maintien relevable</b> Manuel d'utilisation .....	<b>10</b>
it	<b>Barra di supporto pieghevole</b> Manuale d'uso .....	<b>14</b>
es	<b>Barra de apoyo abatible</b> Manual del usuario .....	<b>17</b>
pt	<b>Barra de apoio rebatível</b> Manual de utilização .....	<b>21</b>
nl	<b>Inklapbare handgreep</b> Gebruiksaanwijzing .....	<b>25</b>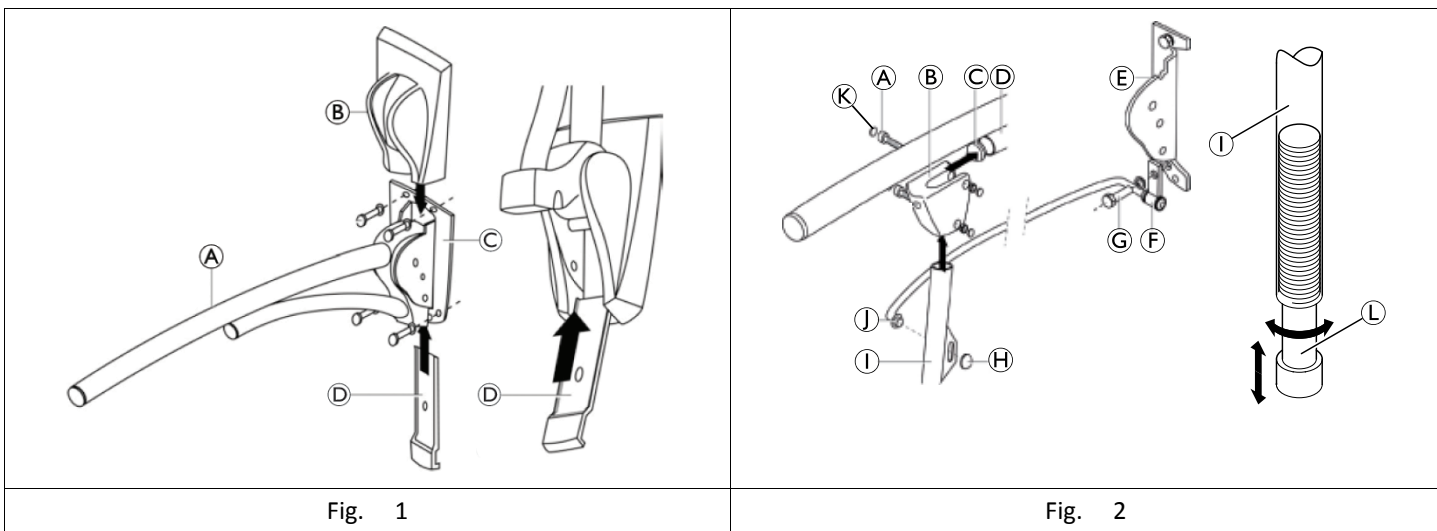

This manual MUST be given to the user of the product.  
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®

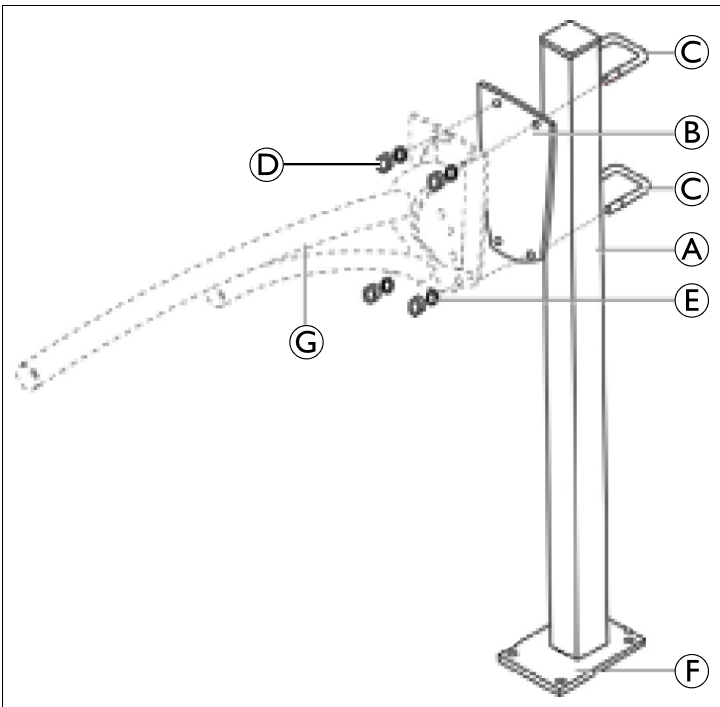
Fig. 3

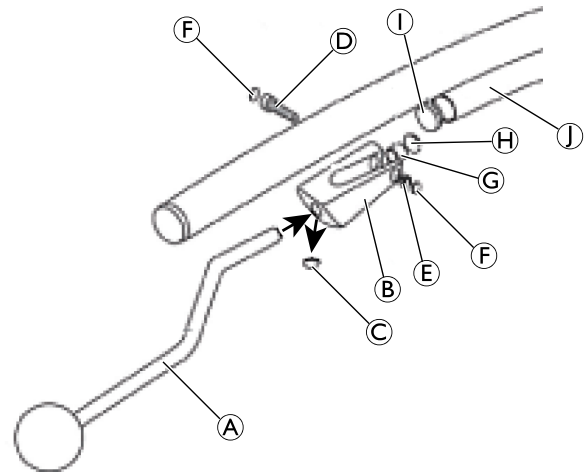
Fig. 4

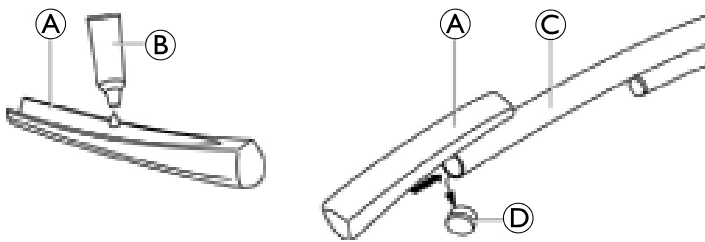
Fig. 5

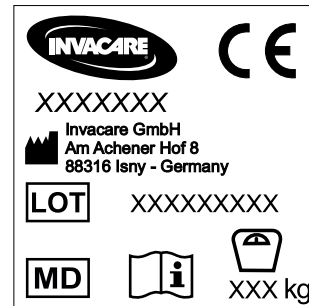
Fig. 6

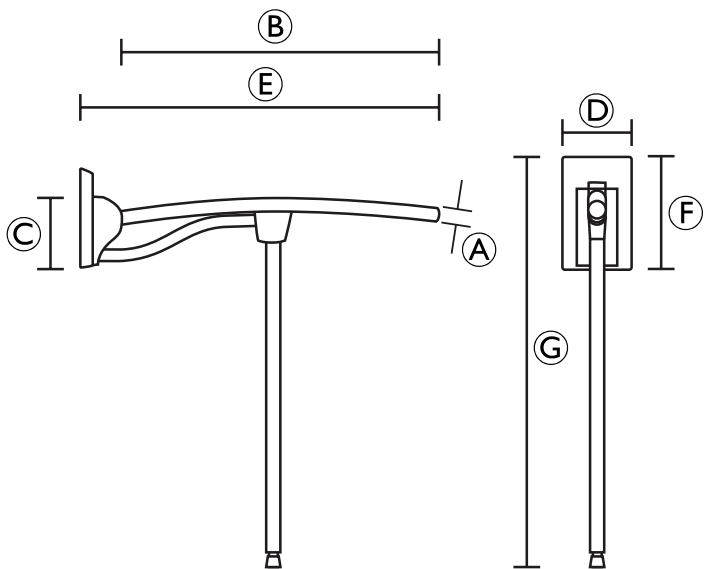
Fig. 7

## 1 General

### 1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

### 1.2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

### 1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

### Product-Specific Standards

The product meets the requirement of the standard DIN EN 12182 (Assistive products for persons with disability) and related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

### 1.4 Service life

The expected service life of this product is five years when used daily in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

### 1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

## 2 Safety

### 2.1 Safety Information

#### WARNING!

#### Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

#### WARNING!

#### Risk of Falling!

- Do not use if defective.
- Do not use as sports equipment or climbing aid.
- Do not make any unauthorised alterations or modifications to the product.

#### WARNING!

#### Risk of Falling!

Excessive or intermittent loads can lead to breaking.

- Do not exceed the maximum user weight, see 8 *Technical Data, page 6*.

#### CAUTION!

#### Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalogue or website for available options or contact your Invacare distributor. See addresses at the end of this document.

### 2.2 Labels and Symbols on Product

#### Identification Label (Fig. 6)

	Conformity mark		Maximum safe load
	Lot number		Read user manual
	Manufacturer		Medical Device

The identification label is attached to the bottom side of the foldable grab bar.

## 3 Product Overview

### 3.1 Intended Use

Invacare Revato are grab bars made of rilsanized steel to be installed on wall or floor with screws and can be flipped up. Additional floor installation or a support leg are available to increase stability.

This product is intended to be used as transfer and support aid when e.g. sitting down and standing up from a toilet seat.

### Intended User

Users are any adults and adolescents with restricted or limited mobility or limited muscle strength. If used without an assistant, the users must be able to keep their own balance.

### Indications

- Restricted or limited mobility (e.g. legs and/or hips) or
- having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended. For the maximum user weight, see 8 *Technical Data, page 6*.

## 3.2 Scope of Delivery

 Not all products, models or accessories / options may be available in every country.

Overview — Complete Products	
H335	Foldable grab bar (700 mm)
R7762	Foldable grab bar (500 – 1000 mm)
H345	Foldable grab bar with support leg (700 mm)
R7772	Foldable grab bar with support leg (600 – 1000 mm)
R7780	Foldable grab bar with support leg, armpad and paper holder (600 – 1000 mm)
Overview — Accessories / Options	
R7763	Floor mounted console
R7765-W	Toilet paper holder
R7766	Armpad

The following items are included in the scope of delivery depending on the product combination resp. accessory / option ordered:

### Foldable Grab Bar (Fig. 1)

Ⓐ	Grab bar (1x)	Ⓒ	Wall attachment socket (pre-mounted to bracket) (1x)
Ⓑ	Guard (1x)	Ⓓ	Adjustment plate 2° (1x)

### Support Leg (Fig. 2)

Ⓐ	Set of screw and nut (2x)	Ⓖ	Fixing screw (1x)
Ⓑ	Hinge piece (1x)	Ⓗ	Starlock retainer ring (1x)
Ⓒ	End cap (1x)	Ⓘ	Support leg (1x)
Ⓓ	Grab bar (1x)	Ⓝ	Guide (1x)
Ⓔ	Socket (1x)	Ⓚ	Protective cap (4x)
Ⓕ	Wall hinge (1x)	Ⓛ	Foot assembly (1x)

### R7763 – Floor Mounted Console (Fig. 3)

Ⓐ	Upright (1x)	Ⓓ	Nut, DIN 934, M8 (4x)
Ⓑ	Attachment plate (1x)	Ⓔ	Washer, DIN 125, Ø 8,4 x Ø 16 (4x)
Ⓒ	Clamp (2x)	Ⓕ	Plate, pre-mounted to upright Ⓐ (1x)

### R7765-W – Toilet Paper Holder (Fig. 4)

Ⓐ	Roll holder (1x)	Ⓔ	Nut, DIN 934, M5 (1x)
Ⓑ	Hinge piece (1x)	Ⓕ	Protective cap (2x)
Ⓒ	Protective cap (1x)	Ⓖ	Spacer Ø 8,2 x Ø 12 x 10 (1x)
Ⓓ	Screw, DIN 912, M5 x 25 (1x)	Ⓗ	Fastener, Ø 8 (1x)

### R7766 – Armpad (Fig. 5)

Ⓐ	Armpad, PUR (1x)	Ⓑ	Glue (1x)
---	------------------	---	-----------

## 4 Setup

### 4.1 Installing Foldable Grab Bar

#### WARNING!

#### Risk of Falling!

– Mount only to walls that have a sufficient load-bearing capacity.

#### Installing the Foldable Grab Bar (Fig. 1)

1. With the user standing upright, without shoes, measure the height of the thumb's upper joint. This height corresponds to the grab bar's highest point Ⓐ (middle of grab bar).
2. Insert the 2° adjustment plate Ⓓ.
3. Mark out the location of the holes on the wall (2 above and 2 below outside) taking into account the previously measured height.
4. Remove the 2° adjustment plate.
5. Position the wall attachment socket Ⓒ for the foldable grab bar accordingly.
6. Drill and insert 4 dowels appropriate for the type of wall.
7. Fix the socket to the wall using suitable screws, washers.
8. Snap the guard Ⓑ onto the socket.
9. Insert the 2° adjustment plate from below again.

#### Installing the Support Leg (Fig. 2)

1. Install the front end of the guide Ⓘ into the oblong hole in the support leg Ⓘ using a plastic mallet and starlock retainer ring Ⓗ.
2. Remove the lower left-hand fixing screw Ⓖ from the socket Ⓔ.
3. Position the guide Ⓘ on the outside of the foldable grab bar, the end with the wall hinge Ⓕ pointing towards the support wall or socket Ⓔ.
4. Fasten the wall hinge Ⓕ using the socket's lower, left-hand side fixing screw Ⓖ.
5. Insert the upper end of the support leg Ⓘ into the hinge piece Ⓑ. Fasten it using the screw and position two protective caps Ⓚ.
6. Install the socket's guard.

To decrease the height, turn the foot Ⓛ counterclockwise. To increase the height, turn the foot Ⓛ clockwise.

#### Installing the Floor Mounted Console (R7763) (Fig. 3)

This option, when sealed to the floor, enables a foldable grab bar to be fastened in place when the wall is unsuitable for this.

1. Position the upright Ⓐ.
2. The plate Ⓕ resting on the floor must have its longest side under the foldable grab bar Ⓒ.
3. Mark out the location of the 4 screws.
4. Drill and insert 4 dowels appropriate to the type of floor.

5. Fix the plate to the floor using suitable screws.
6. Position the plate **B** with the rounded section towards the bottom and the foldable grab bar **C**.
7. Fasten the plate **B** and the foldable grab bar **C** with the two clamps **C**, washers **E** and nuts **D** supplied but without tightening them.
8. With the user standing upright, without shoes, measure the height of the upper joint of the user's thumb and move the foldable grab bar to this height.
9. Tighten the nuts **D** on the two clamps **C**.
10. Install the socket's guard.

## 4.2 Installing Options

### Installing the Toilet Paper Holder (R7765-W) (Fig. 4)

1. If fitted, remove the end cap **I** from the lower foldable grab bar **I**.
2. Remove the protective cap **C** from the hinge piece **B** using a screwdriver
3. Insert the toilet roll support **A** into the hinge piece **B**.
4. Position the spacer **G** and lock the assembly in place with the fastener **H**.
5. Attach the hinge piece **B** by pressing or by tapping with a plastic mallet to the lower foldable grab bar **I** and fasten with screw **D** and nut **E**. Attach a protective cap **F** on each side.

### Installing the Armpad (R7766) (Fig. 5)

1. If fitted, remove the black end cap **D** from the foldable grab bar's upper tube **C**.
2. Clean the parts to be glued together with acetone before applying the glue **B**.
3. Apply the whole tube of glue to the inside face of the armpad **A** and position it onto the foldable grab bar **C** being aligned lengthwise, ensure the armpad to point straight upward.
4. Press it down for approximately 30 seconds.

 Full adhesion is only obtained after approximately 24 hours. Invacare is not responsible for any damage caused by improper fixing or inappropriate use.

## 5 Usage

### WARNING!

#### Risk of Falling or Slipping!

- Before each use, check that the foldable grab bar is securely fastened and sits firmly.

1. Grab the foldable grab bar with one hand and carry out the desired body movement.
2. Lift the foldable grab bar from the horizontal position to its maximum vertical position. It locks automatically in this position.
3. Lift the foldable grab bar approx. 10 mm to unlock it and hold it as it drops down to the horizontal position.

## 6 Maintenance

### 6.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

### 6.2 Cleaning and Disinfection

#### General Safety Information

### CAUTION!

#### Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

### NOTICE!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

### Cleaning Intervals

### NOTICE!

Regular cleaning and disinfection enhance smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product:

- regularly while in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

### Cleaning

### NOTICE!

Maximum cleaning temperature:

- 60 °C by hand

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

### Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

## 7 After Use

### 7.1 Reconditioning

### WARNING!

#### Risk of Falling

Worn fastening elements can suddenly come loose and cause falling.

- Always use new wall plugs and bolts when repositioning the product.

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *6 Maintenance, page 5*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

## 7.2 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

## 8 Technical Data

### 8.1 Dimensions, Weights and Materials (Fig. 7)

Product	Foldable Gar Bar without Support Leg	Foldable Grab Bar with Support Leg
Ⓐ	Æ 32 mm	
Ⓑ	420 – 920 mm <sup>1)</sup>	
Ⓒ	135 mm	
Ⓓ	135 mm	
Ⓔ	500 – 1000 mm <sup>1)</sup> / (H335/H345 = 700 mm)	
Ⓕ	220 mm	
Ⓖ	-	750 – 850 mm
Product weight <sup>1)</sup>	H335: 3.0 kg	H345: 3.5 kg
Maximum user weight	125 kg	
Color	White	
Material <sup>2)</sup>	Steel, Plastic, PU (armpad option)	

<sup>1)</sup> Depends on grab bar length of the product ordered.

<sup>2)</sup> All product components are corrosion resistant and do not contain natural rubber latex.

### 8.2 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare Poirier SAS
Invacare AG
Invacare GmbH