
Replacement Kit Invacare® Alegio™ NG

en	Replacement kit for base and scissor screws Service Manual	2
es	Kit de sustitución de los tornillos de la base y la tijera Manual de servicio.....	2
fr	Kit de remplacement pour les vis de la base et des ciseaux Manuel de maintenance.....	3
pt	Kit de substituição para os parafusos da base e da tesoura Manual de assistência.....	3

PROVIDER: Keep this manual.
The procedures in this manual MUST be performed by a qualified technician.

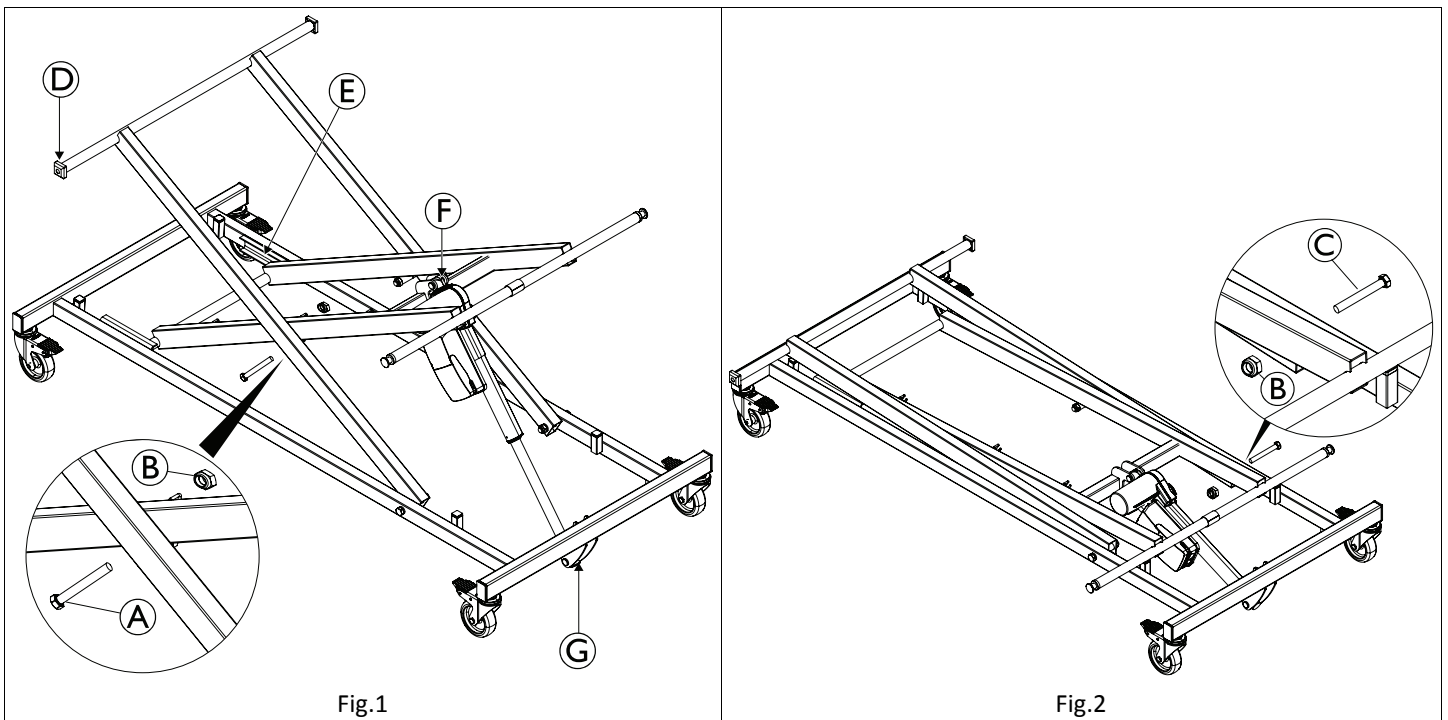Yes, you can.®

Safety Information

WARNING!

Risk of injury

Improper installation may affect the function and safety of the bed.

- The procedures in this manual **MUST** be performed by a qualified technician.
- Read this manual and the manual of the bed carefully and follow the safety instructions.
- After each assembly, check that all fittings are properly tightened and that all parts have the correct function.
- If you have any questions regarding the installation, contact your local Invacare representative.

WARNING!

Risk of injury or damage to property

There is a risk of entrapment or squeezing, while assembling or disassembling.

- Pay attention to your fingers.
- Follow instructions carefully.

Scope of delivery

- 2x M12 x80 screws **A**
- 2x M12 x90 screws **C**
- 4x M12 locknuts **B**
- 1x Grease (NEVASTANE XMF 2)

Assembly Instructions

1. Lock all castors and remove all accessories, mattress, side rails and bed ends from the bed.
2. Put the base in its lowest position.
3. Disconnect the mattress support actuators from the control box and remove the mattress support.
4. Replacing the scissor screws (Fig. 1)
 - a. Put the base in its highest position.
 - b. With two 19 mm wrenches, remove screw **A** and nut **B**.
 - c. Apply grease at the body of a new screw **A**.
 - d. Insert the new screw in the joint and tighten with a torque of 40 Nm using a new nut **B**.
 - e. Repeat steps b – d for the other side.
5. Replacing the base screws (Fig. 2)
 - a. Put the base in the lowest position.
 - b. With two 19 mm wrenches, remove screw **C** and nut **B**.
 - c. Apply grease at the body of a new screw **C**.
 - d. Insert the new screw in the joint and tighten with a torque of 40 Nm using a new nut **B**.
 - e. Repeat steps b – d for the other side of the bed.
6. Apply grease to the pivot points **D**, **E** (both sides of the bed), **F** and **G**.
7. Remove any grease that may be on the outside of the joints, with a soft cloth.
8. Move the base up and down to make sure that the connections rotate smoothly.
9. Reattach the control box and the mains cable to the base.
10. Attach the mattress support to the base and connect the mattress support actuators to the control box.
11. Move the bed up and down to ensure that it moves smoothly.
12. Reattach all accessories and check for correct function.

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Invacare S.A.
Invacare Poirier SAS