Matrx®	Libra	2.0 kg	136; 227 kg	355 - 560 mm	305 - 560 mm	85 mm	80 °C
	Contour Visco NG	2.5 kg	80 - 140 kg	355 - 555 mm	405 - 530 mm	100 mm	90 °C
Flo-Tech®	Lite	0.9 kg	127 kg	255 - 510 mm	255 - 535 mm	80 mm	80 °C
	Lite Visco	1.6 kg	32 - 127 kg	255 - 510 mm	255 - 535 mm	80 mm	80 °C
	Contour	1.5 kg	152; 191 kg	355 - 610 mm	330 - 560 mm	115 mm	80 °C
	Contour Visco	1.8 kg	51 - 127 kg	355 - 510 mm	330 - 535 mm	115 mm	80 °C
	Plus	2.4 kg	152; 191 kg	355 - 610 mm	330 - 560 mm	115 mm	80 °C
	Image	1.9 kg	140 kg	355 - 510 mm	330 - 535 mm	80 mm	80 °C
	Solution	3.5 kg	140 kg	355 - 610 mm	330 - 560 mm	80 mm	80 °C
	Solution Xtra	3.5 kg	140 kg	355 - 610 mm	330 - 560 mm	80 mm	80 °C

United Kingdom & Ireland:
Invacare Limited
Pencoed Technology Park,
Pencoed
Bridgend CF35 5AQ
Tel: (44) (0) 1656 776 200
uk@invacare.com
www.invacare.co.uk

France:
Invacare Poirier SAS
Route de St Roch
F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66
Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com
www.invacare.fr

Danmark:
Invacare A/S
Sdr. Ringvej 37
DK-2605 Brøndby
Tel: (45) (0)36 90 00 00
Fax: (45) (0)36 90 00 01
denmark@invacare.com
www.invacare.dk

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Invacare UK Operations Limited
Pencoed Technology Park
Pencoed Bridgend CF35 5AQ
UK

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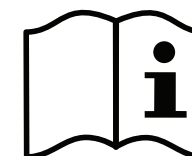
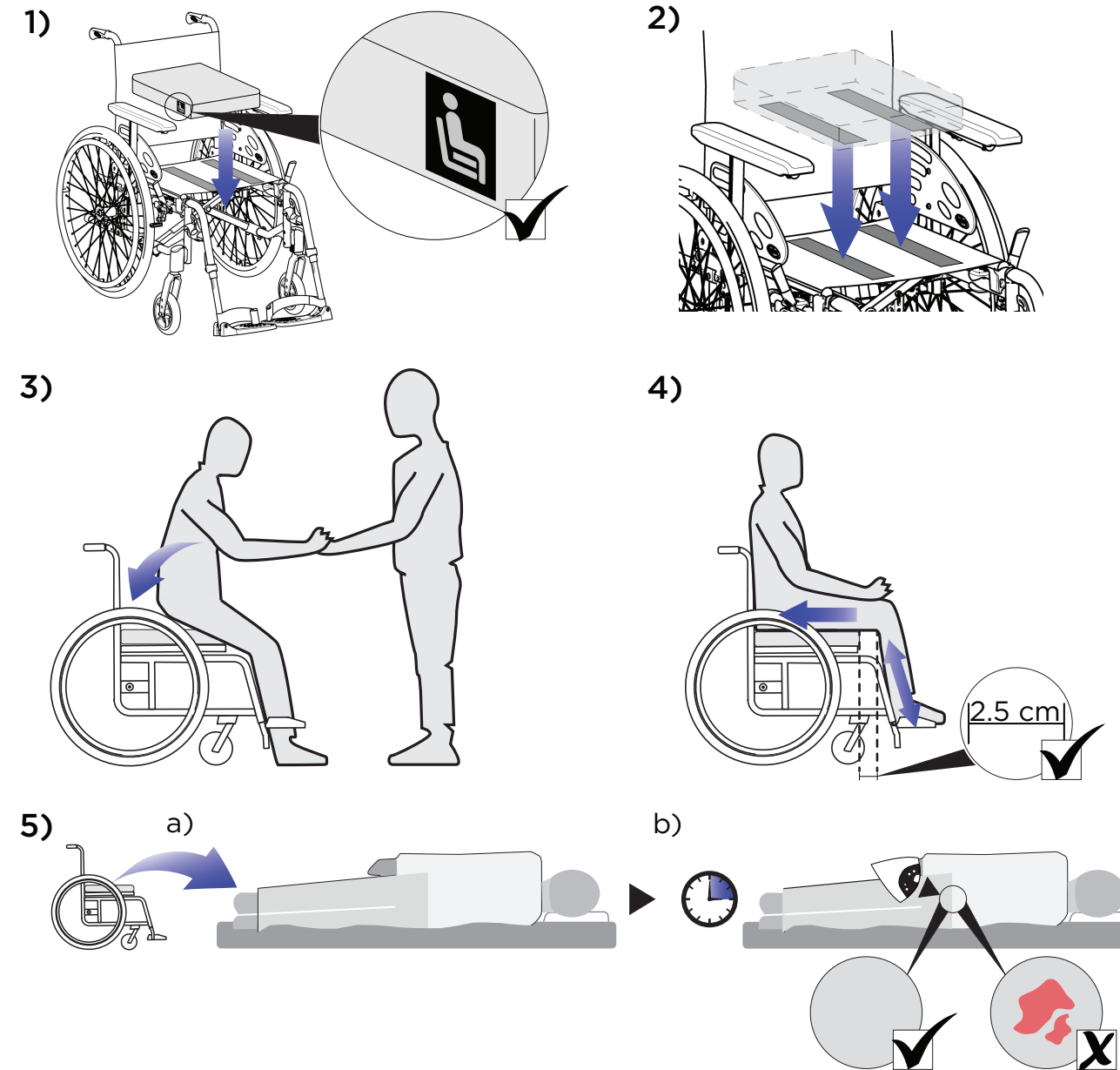
Yes, you can.

Introduction

This user manual contains important information about the handling of the product. For more detailed information, you find the latest version as a PDF on the Invacare website.

Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485. This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. This product features the UKCA mark, in compliance with Part II UK MDR 2002 (as amended) Class I. We only use REACH compliant materials and components.

Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the product was obtained.

Product Description

The Invacare cushion range is made of a unique, pressure reducing, ergonomically designed foam, which is designed to provide comfort, postural support and pelvic stability. The foam has different shapes, densities and depending on the cushion model, it can be also equipped with non-toxic silicone fluid to optimise pressure distribution.

Intended Use/User

This cushion is intended to be used in an armchair or wheelchair as part of an overall pressure injury prevention program of care. This cushion can be used for users who require pelvic stability and are at a low to very high risk (depending on model) of developing pressure ulcers.

Safety Information

Cushions must be used with the cover at all times. Invacare advises that the patient be assessed by a trained healthcare professional to ensure the correct size and seating requirements. It is very important for the patient to reposition themselves, or to be repositioned, on a regular basis. This relieves pressure which helps prevent both tissue compression and potential pressure injury formation. Skin redness is one clinical indicator of skin breakdown. Immediately contact a healthcare professional if your skin develops redness.

Maintenance

Maintenance should be done at least once a year. Do not use a defective product. Do not attempt repairs yourself. For repairs, contact your provider.

Inspection

Check cushion (foam and cover) for strike-through (this may include fluid ingress, stains, rips or damage) after the use of each patient and on a minimum monthly basis (depending on which occurs first) by a suitably qualified and competent person.

Cleaning and Disinfection

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures. Never use corrosive fluids (alkaline, acid, etc.), abrasive cleaning agents or solvents (cellulose thinner, acetone, etc.). We recommend a mild detergent. Regular cleaning and disinfection enhance smoothe operation, increases the service life and prevents contamination.

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